
IPS Inflatable Prone support

Ergotrics® nv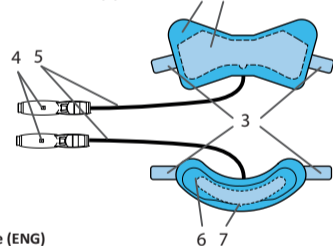

Instruction for Use (ENG) IPS (Inflatable Prone Support)

Description
The IPS is a disposable inflatable cushion. Inflate and deflate with the Hummingbird (HMB) using compressed medical air, the IPS lifts and positions the patient in prone position. For best result, use Ergotrics' inflatable Board (IBO) for the proning of the patient.

The IPS consists of:

- | | |
|-----------------------------|-----------------------|
| Thoracic cushion | Pelvic cushion |
| 1. cushion | 6. cushion |
| 2. body contact sheet (BCS) | 7. pelvic tape |
| 3. side tape | 3. kleefstrip zijkant |
| 4. valve | 4. klep |
| 5. tube | 5. tube |

Intended purpose
Positioning patients during a surgical procedure in prone position. Compatible with HMB.

Operating instructions

- Position the patient onto the bed, spread the arms of the patient (Fig A).
- Apply cushions on the patient using the bones as reference (Fig B).
- Apply the thoracic cushion onto the rib cage of the patient. The round extension of the BCS (2) should be aligned with the metasternum (Fig B). The thoracic part should not extend below the rib cage or above the clavicles.
- Apply the pelvic cushion onto the pelvis of the patient. The round extension of the 'pelvic tape' (7) should be near the pubic bone and the tube should be pointing to the belly button (Fig B). The pelvic part should not extend over the groin.
- Apply the side tape (3) tightly to the patient's body (Fig C).
- Arrange the tube and valve as illustrated (Fig C).
- Put the arm of the patient over the tubing (Fig D).
- Bring the patient into prone position.
- Put the arms of the patient onto the arm supports and release the side tapes (Fig E).
- Inflate first the thoracic cushion using the Ergotrics Hummingbird. Stop inflation of the IPS when the whistle sounds (Fig F.1) and close the pinch clamp (Fig F.2).

Remark: Hold the IPS valve with one hand while pushing the HMB against it with the other hand.

11. Inflate the pelvic cushion using the HMB. Stop inflation of the IPS when the whistle sounds (Fig G.1) and close the pinch clamp (Fig G.2).

AFTER SURGERY: return patient in supine position and remove the cushions. Cut the inflation tubes to deflate the cushions.

Warnings
The IPS has been designed for single use. Reuse will cause disfunctions of the valve and cushions. Only inflate and deflate using the HMB with compressed medical air at 4-6bar/58-87psi. Higher pressures may cause damage. See table for maximum patient weight. Store in normal warehouse conditions. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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