Instruction for Use (ENG)
IxS (Inflatable Support for multiple use)

Description
The IxS is a single-use or single-patient inflatable pillow. (Part of) the patient's body is positioned by inflating and deflating the cushion with the Hummingbird (HMB) using medical compressed air.

- The IxS-R consists of:**
1. cushion
 2. body contact sheet (BCS)*
 3. side tape*
 4. valve
 5. tube
 6. Cover (COV)*

Intended purpose
The multifunctional IxS pillows are designed to position patients for a surgical or medical procedure, treatment or examination. Compatible with HMB.

Operating instructions

1. Affix the IxS to the body part of the patient to be moved or positioned.
- 2a. When using the variant with tape, adhere the body contact sheet (BCS) and side strips tightly to the patient's body so that the IxS does not move during the procedure or treatment or examination.
CAUTION: Always detach the side strips before inflation.
- 2b. When using the variant with cover, first stick the cover to the patient and then stick the IxS to the cover.
3. Place the inflation hose and valve on the bed next to the patient so that neither this patient nor a healthcare professional will disturb or touch during the procedure or treatment or examination.
4. Inflate the pillow using the nose of the Ergotrics Hummingbird (Fig A). Stop inflating the IxS when you hear the whistle (Fig B) or until (the body part of) the patient is in the desired position. During the procedure, treatment or examination you can inflate the IxS even further (Fig A) but always stop at the whistle. (Fig B)
- Note: Hold the cover of the IxS with one hand while pushing the HMB against it with your other hand.
5. Use the Hummingbird's horn to release air from the IxS during or after surgery, treatment or research (Fig C).
6. Close with the pinch clamp (Fig D) when the patient is in the desired position and every time you are involved in the procedure, treatment or examination.

AFTER THE OPERATION / TREATMENT / EXAMINATION: Loosen the cushions and cut the hose to deflate the cushions.

Warnings
The IxS is designed for single use or single patient use. Inflate only and deflate using the Hummingbird with medical compressed air at 4-6bar/58-87psi. Higher pressures can cause damage. See size chart for the maximum load. Storage in normal warehouse conditions. Any serious incident related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient are/is established.

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