

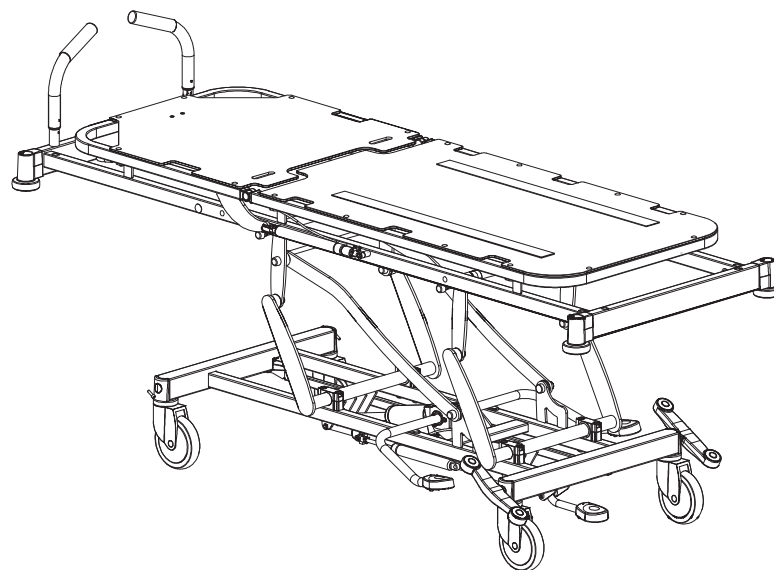



Explosionszeichnung ST20

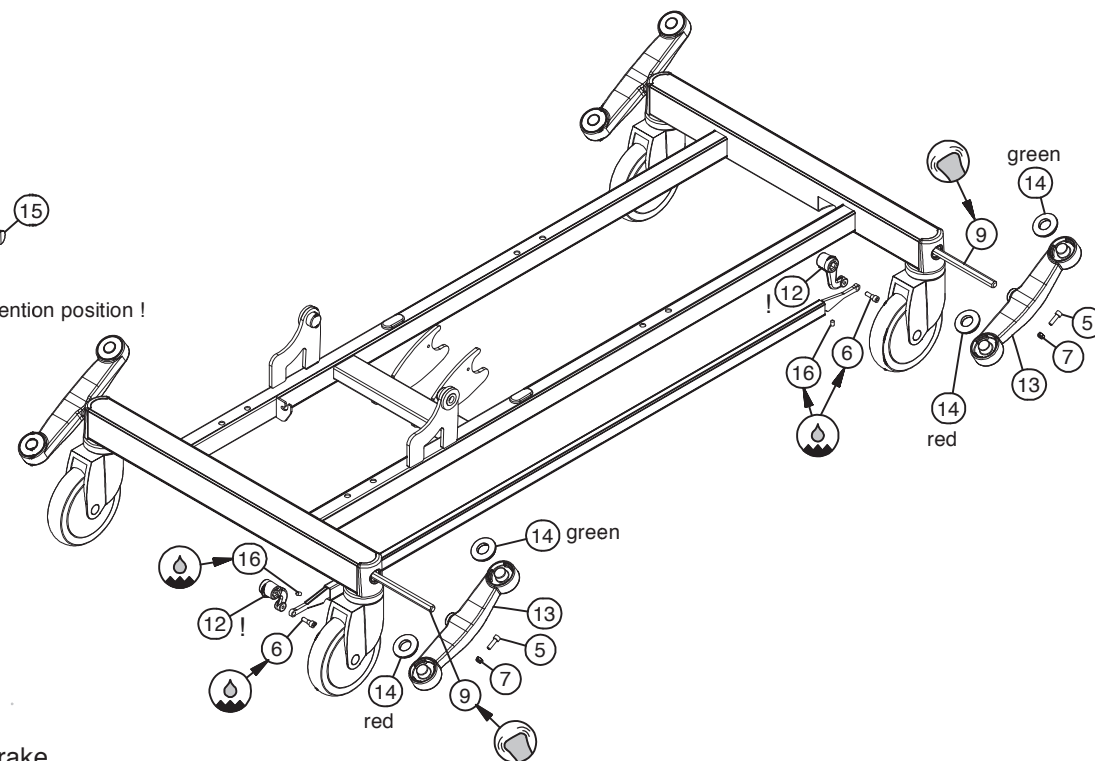

CE nordiska⁺

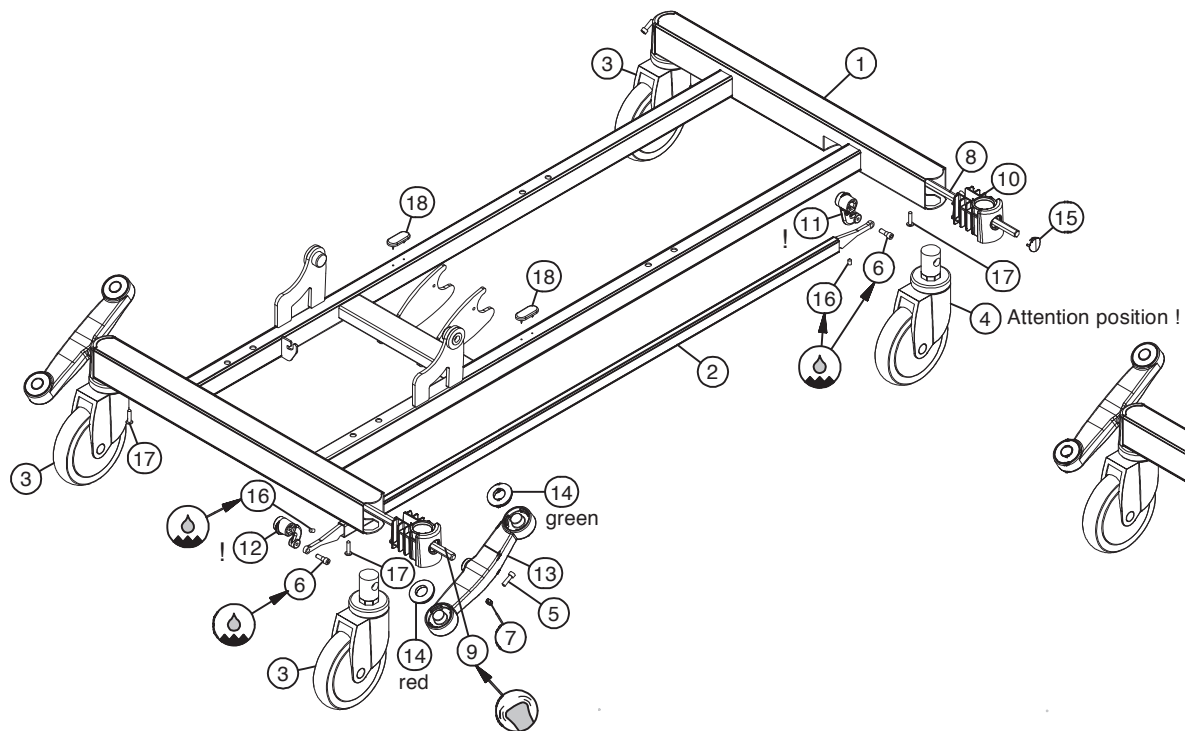

Position drive central brake & direction castor

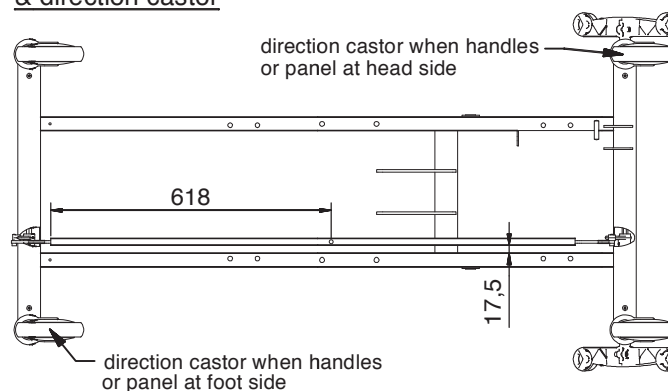


Standard

	ID	Description	Q
1	A-011634	frame carrier Vico	1
2	A-012342	drive central brake Vico	1
3	A-013244	castor	3
4	A-013695	direction castor	1
5	B-000652	bolt Din912-M6x20x20-8.8 GV white	4
6	B-015008	bolt central brake	2
7	M-000137	check nut Din985-M6	4
8	P-014380	hexagonal rod head Vico	1
9	P-014381	hexagonal rod double brake pedal Vico	1
10	P-014396	castor fixation 32-CB	4
11	P-014621	head lever central brake (S)	1
12	P-014717	foot lever central brake (L)	1
13	P-015472	double brake pedal	2
14	P-015477	button double brake pedal	4
15	P-015672	cap castor fixation	2
16	S-001939	set screw Din914-M6x10	2
17	S-015945	screw Din7981C-5.5x25	4
18	St000047	slide with nails	2


Option : pedals at head and foot side


	ID	Description	Q
5	B-000652	bolt Din912-M6x20x20-8.8 GV white	4
6	B-015008	bolt central brake	2
7	M-000137	check nut Din985-M6	4
9	P-014381	hexagonal rod double brake pedal Vico	2
12	P-014717	foot lever central brake (L)	2
13	P-015472	double brake pedal	4
14	P-015477	button double brake pedal	8
16	S-001939	set screw Din914-M6x10	2

	ID	Description	Q
1	A-011639	bottom lever head	1
2	A-011644	bottom lever foot	1
3	A-013377	hydraulic pump 9500N-IL252-SL140-pressure relief	1
4	A-013447	hydraulic pump 9500N-IL252-SL140	1
5	B-006313	bolt Iso7380-M10x90x90-10.9 GV white	8
6	Dc000449	axle motor	1
7	GI017646	bearing 101216-12	2
8	M-000136	check nut Din985-M10	8
9	P-002384	fork hydraulic pump	2
10	P-005972	bearing 35 - top	4
11	P-014284	bearing 35-bottom side type 2	4
12	S-002061	screw Din84-M6x16-4.8GV	2
13	St010662	axle locking ring 14/7x1.3	2
14	VI000469	washer Din125A-M10-PA	4
15	VI002143	washer Din6798A-M6	2

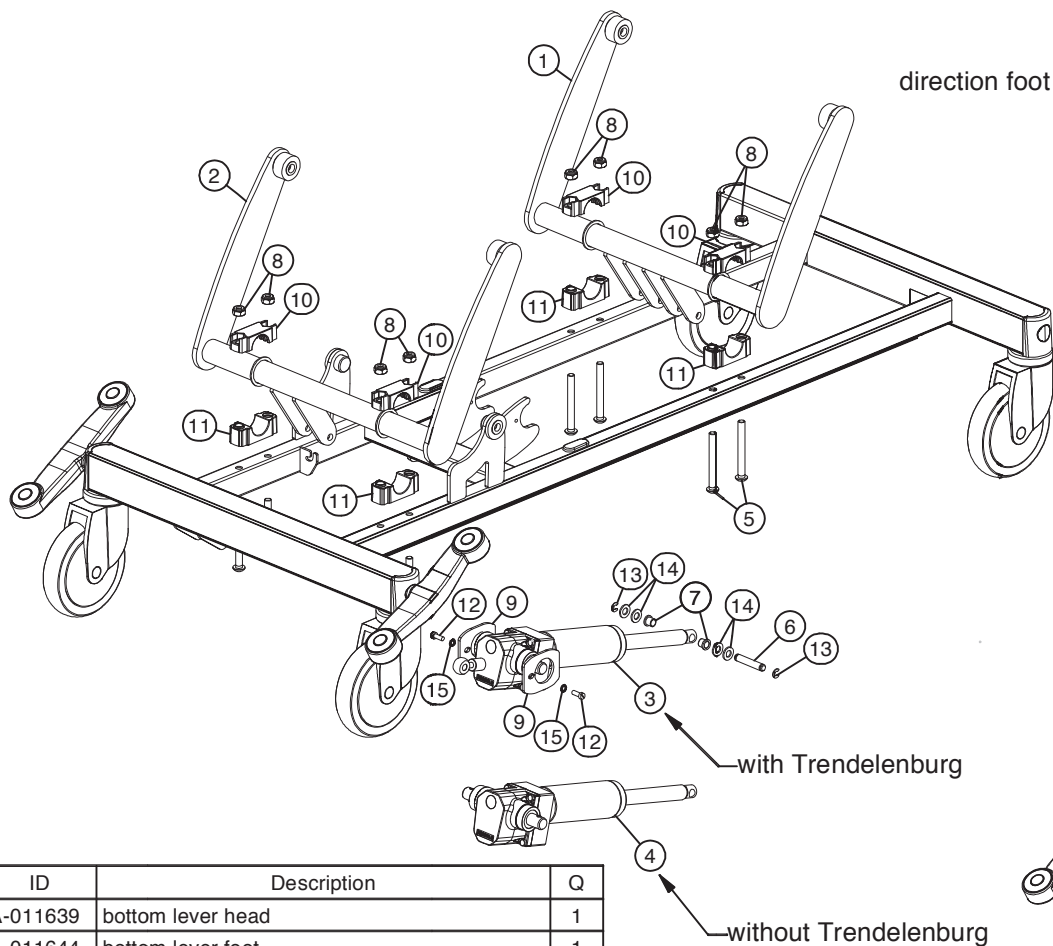


	ID	Description	Q
16	A-012864	top lever foot stretcher	1
17	A-012866	top lever head stretcher	1
18	A-013539	stabilisator	1
19	Ad003053	cap M12	8
20	B-003235	bolt Din912-M12x30x14-8.8 GV white	8
21	GI000620	bearing 1214-06	8
22	St015252	saddle base 28-30	2
23	VI016293	bearing flange 2238-015	8

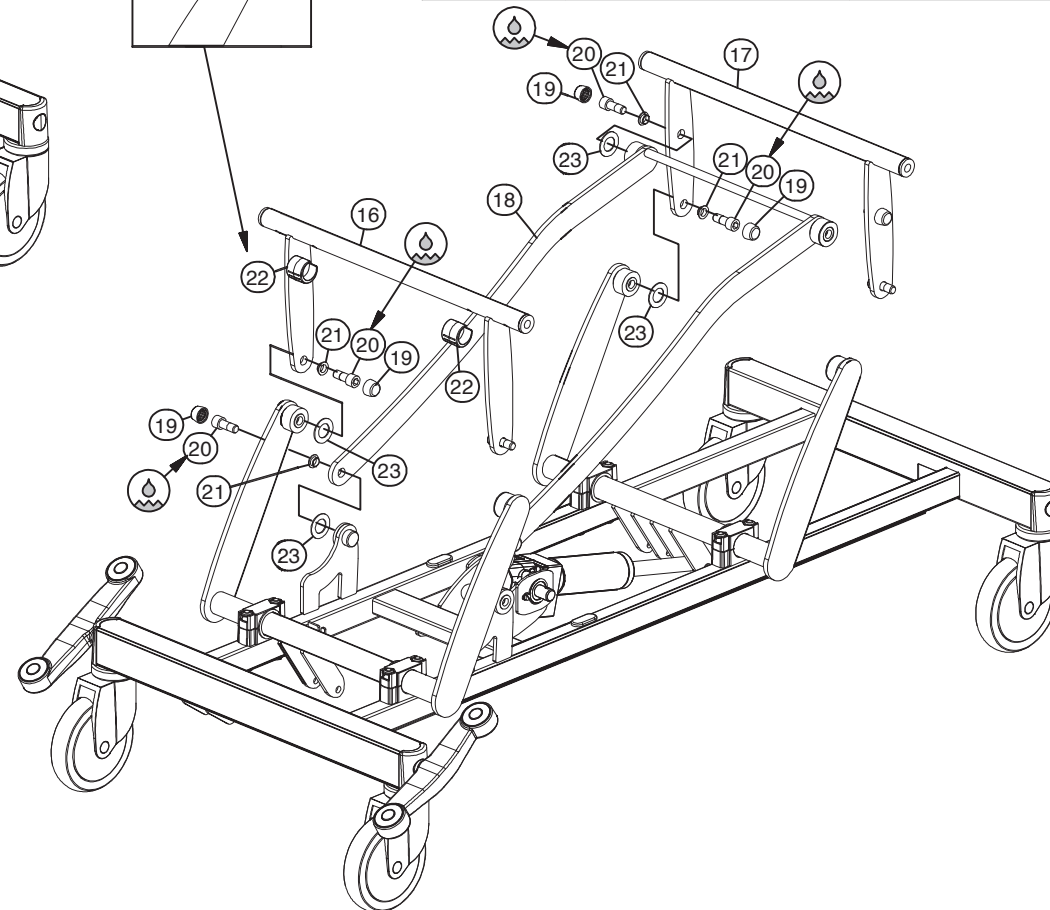

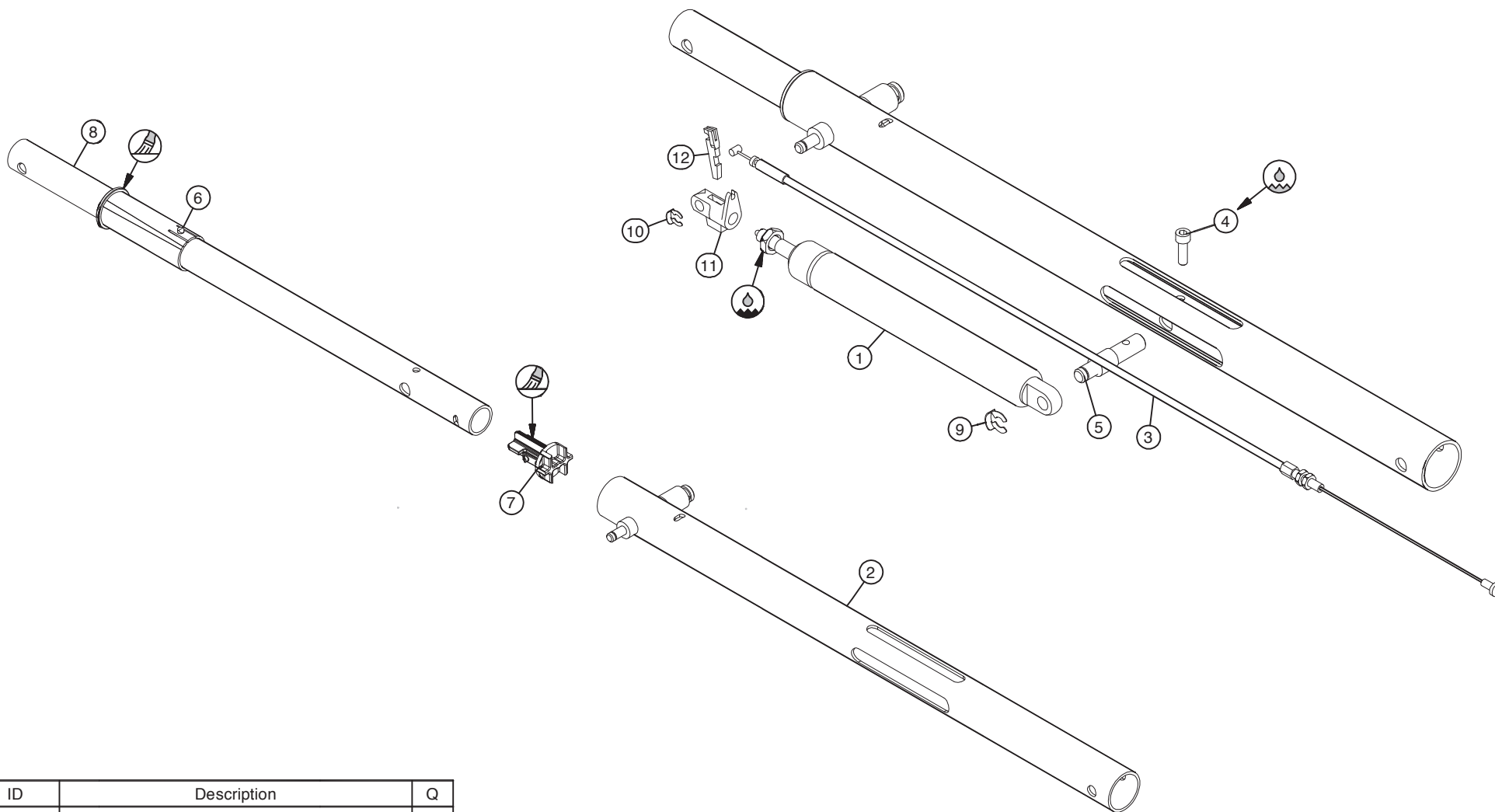


	ID	Description	Q
1	A-000723	gas spring 28B-SL100-IL240-900N	1
2	A-012377	guide trendelenburg	1
3	A-012450	Bowden cable trend Vico	1
4	B-000652	bolt Din912-M6x20x20-8.8 GV white	1
5	Dc014932	fixing axle gas spring	1
6	P-012936	guiding cage Calandro bottom	1
7	P-013614	guiding tap Calandro top	1
8	P-014931	tube Trendelenburg	1
9	St000465	springfix diam.10	1
10	St000797	springfix diam.8	1
11	St005073	release head M10 dia8	1
12	St005074	release lever	1

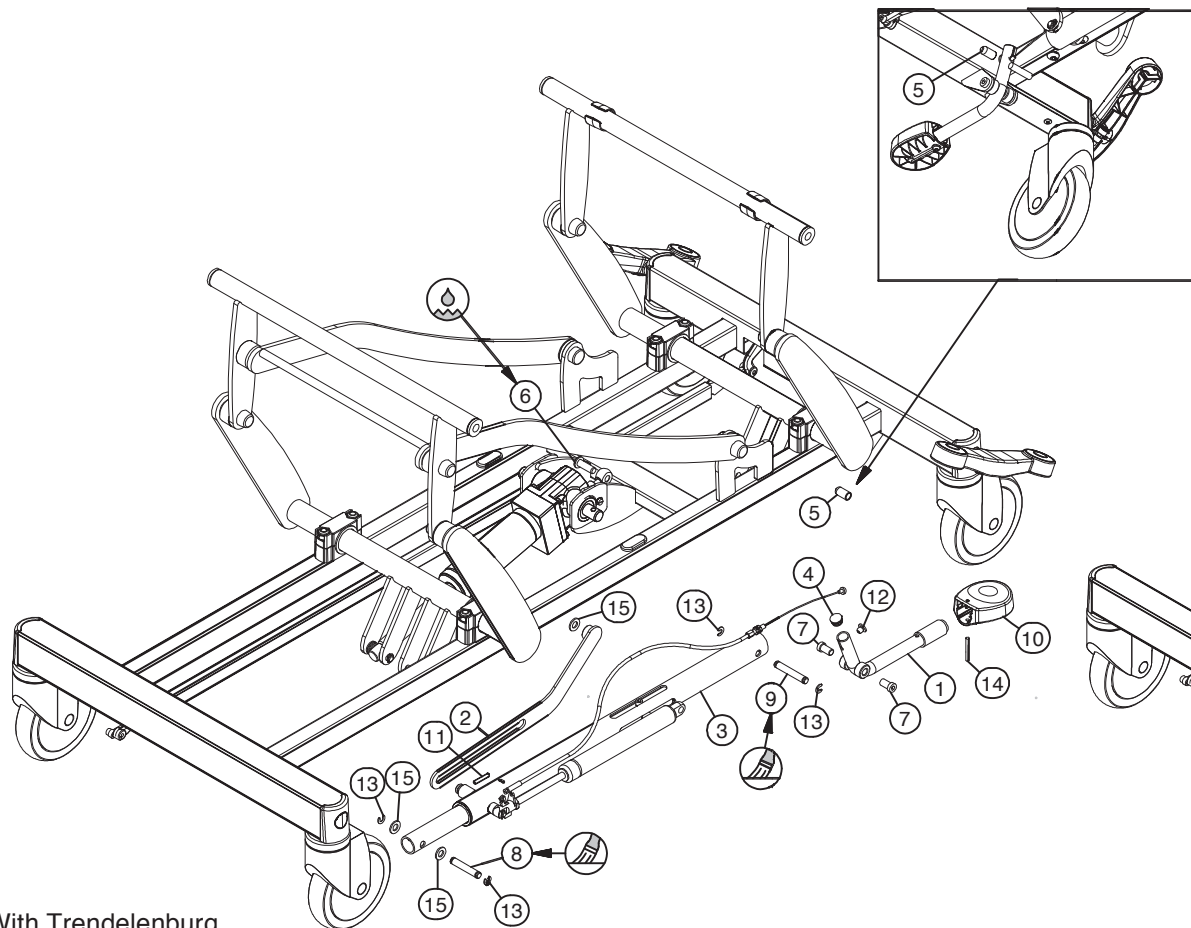


With Trendelenburg

	ID	Description	Q
1	A-012270	pedal Trendelenburg	1
2	A-012352	pulling fork pressure relief	1
3	A-013383	trendelenburg mechanism	1
4	Ad004473	cap PR 18mm	1
5	Ad016158	cap solid 10mm L19	1
6	B-000818	bolt Iso7380-M10x30x30-10.9 GV white	1
7	B-003317	bolt-M10x18x12-GV white	2
8	Dc000449	axle motor	1
9	Dc014939	axle trendelenburg	1
10	P-015611	treadle	1
11	S-014286	set screw Din914-M5x30	1
12	St000141	buffer 14x2/8x7	1
13	St010662	axle locking ring 14/7x1.3	4
14	St015495	grooved pin Din1471-5x40-St	1
15	VI000469	washer Din125A-M10-PA	3

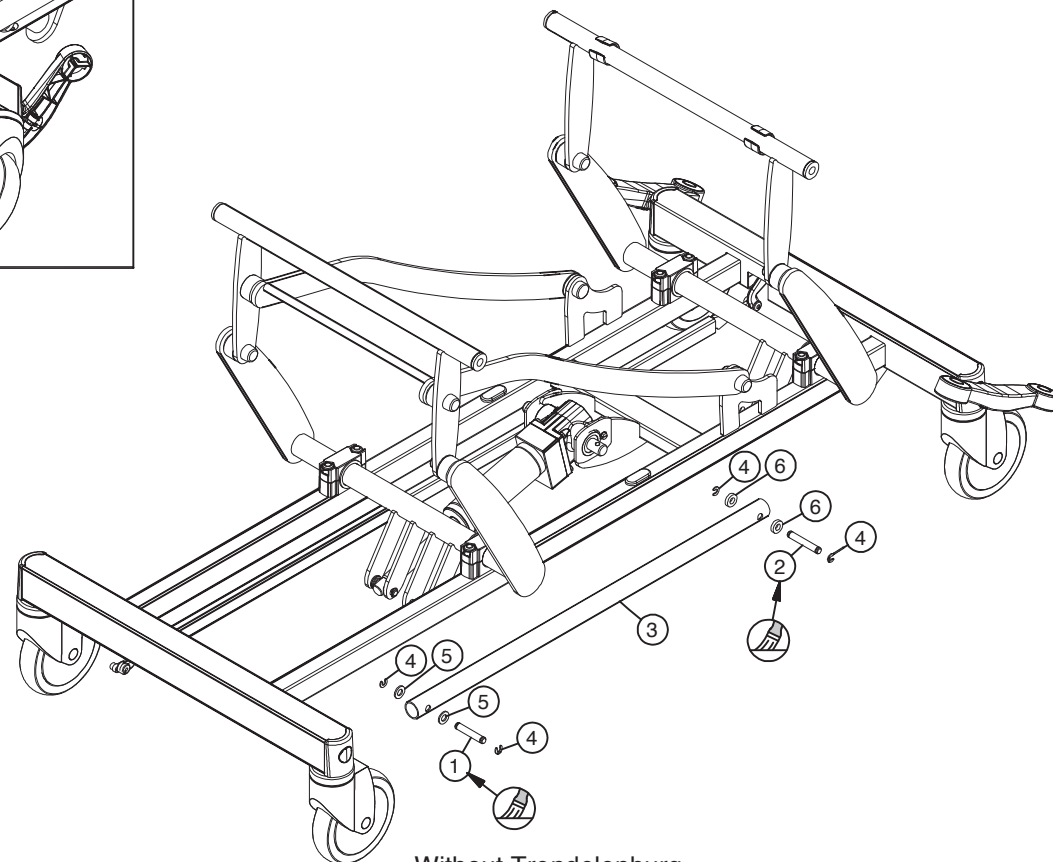


Without Trendelenburg

	ID	Description	Q
1	Dc000449	axle motor	1
2	Dc014939	axle trendelenburg	1
3	P-015864	drive HL	1
4	St010662	axle locking ring 14/7x1.3	4
5	VI000469	washer Din125A-M10-PA	2
6	VI004120	synthetic washer 20x10x5	2

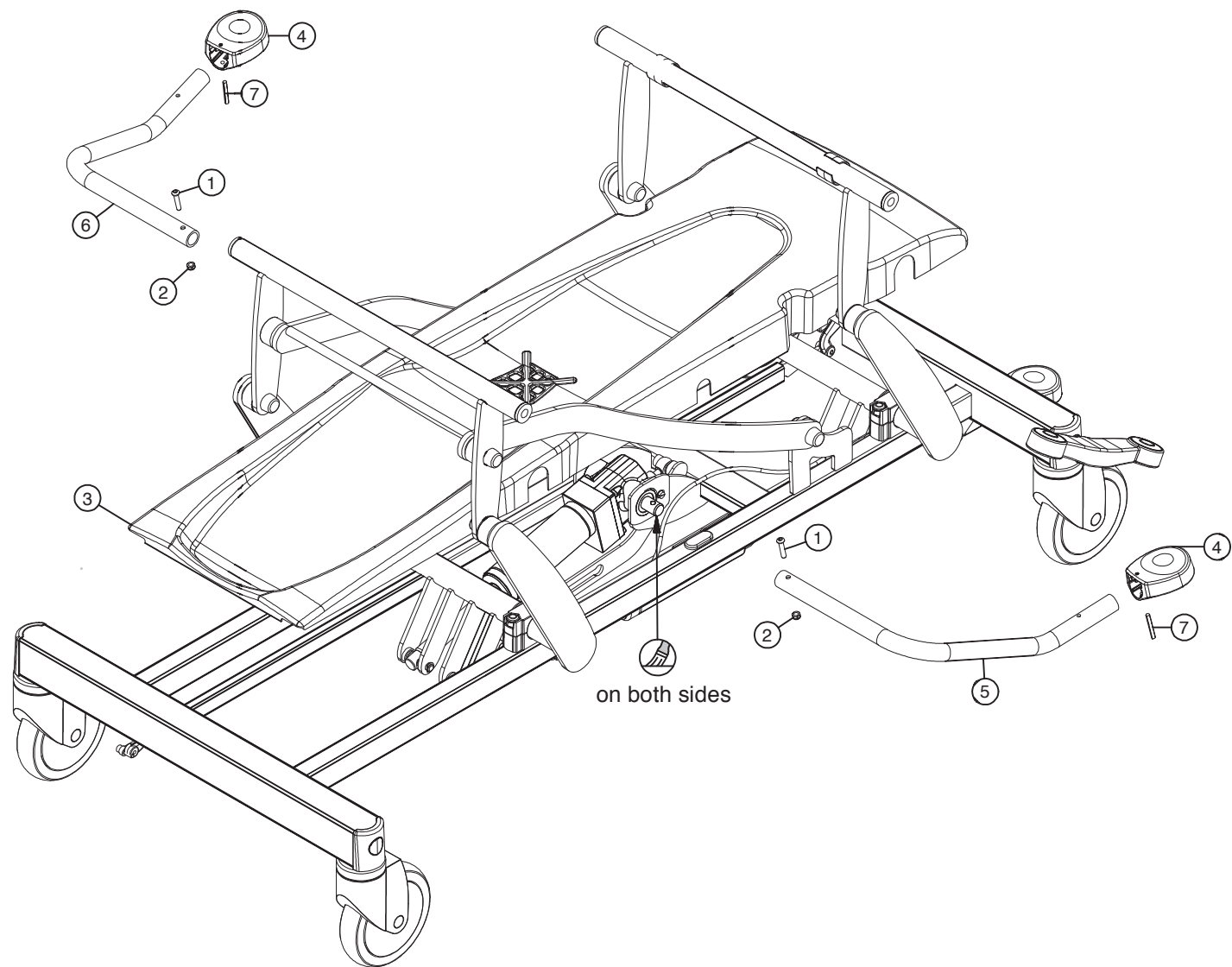

	ID	Description	Q
1	B-006389	bolt Iso7380-M6x30x30-10.9 GV white	2
2	M-000137	check nut Din985-M6	2
3	P-014775	covering Vico	1
4	P-015611	treadle	2
5	P-015821	arm treadle L	1
6	P-015853	arm treadle R	1
7	St015495	grooved pin Din1471-5x40-St	2

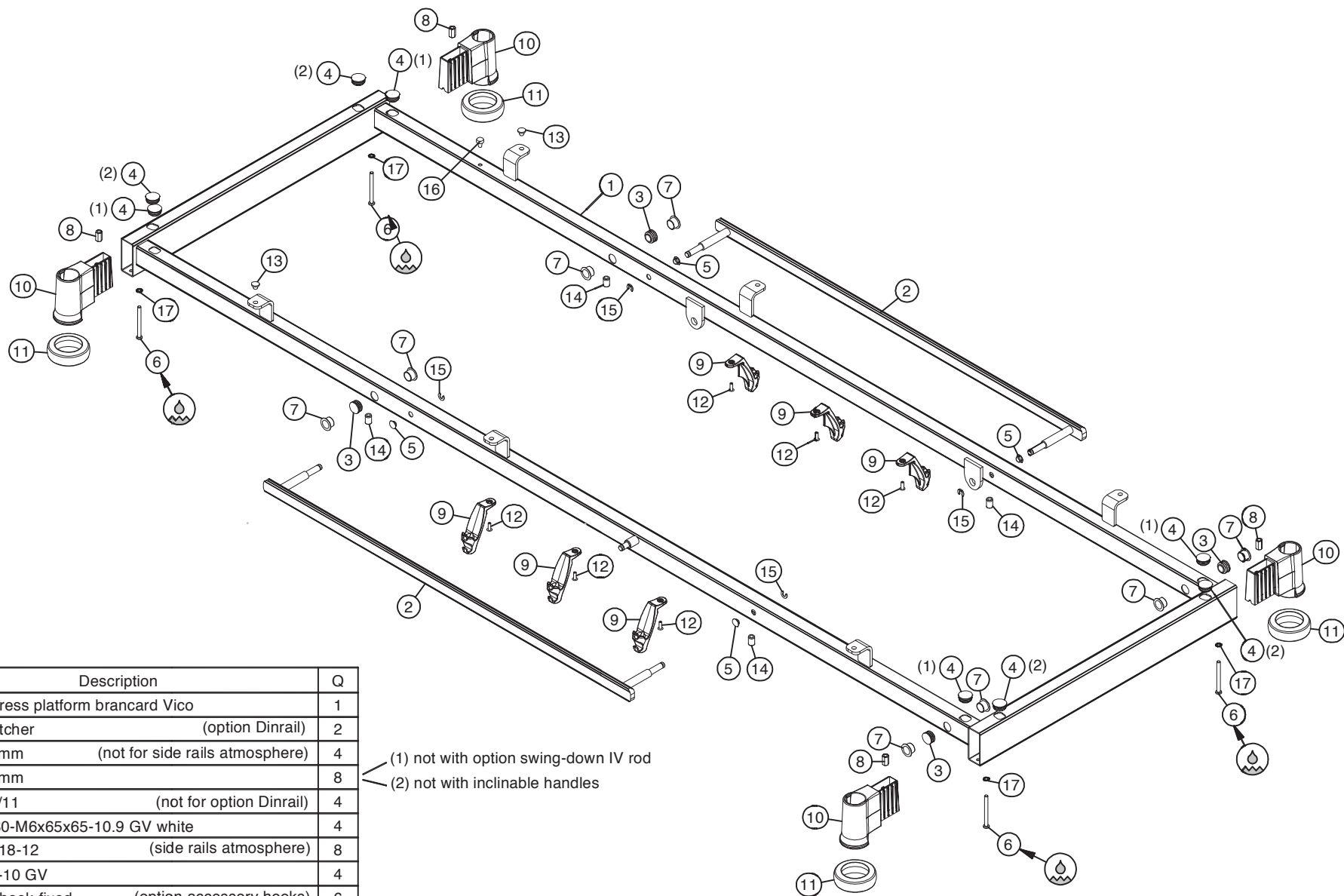


ID	Description	Q
1	A-012860 frame mattress platform brancard Vico	1
2	A-012872 Dinrail stretcher (option Dinrail)	2
3	Ad000637 cap PR 22mm (not for side rails atmosphere)	4
4	Ad000890 cap PR 25mm	8
5	Ad012577 cap PR 15/11 (not for option Dinrail)	4
6	B-010240 bolt Iso7380-M6x65x65-10.9 GV white	4
7	GI000553 bearing 1618-12 (side rails atmosphere)	8
8	M-015222 nut M6x18-10 GV	4
9	P-012677 accessory hook-fixed (option accessory hooks)	6
10	P-014412 bumper wheel support 32	4
11	P-014413 bumper wheel 75	4
12	R-002201 rivet Alu/St 4.8x14 (option accessory hooks)	6
13	St000141 buffer 14x2/8x7	2
14	St010602 insert CSK M8x16 (only for stretcher fixed height)	4
15	St010662 axle locking ring 14/7x1.3 (option Dinrail)	4
16	St016561 grommet foot	1
17	VI002143 washer Din6798A-M6	4

(1) not with option swing-down IV rod
(2) not with inclinable handles

Set of inclinable handles

optional at foot side


	ID	Description	Q
1	B-016544	bolt Iso7380-M8x70x70-10.9 GV white	2
2	B-017743	bolt Iso7380-M4x8x8-10.9 GV wit	2
3	Db014239	threaded bush-M4x25-GV	2
4	P-015131	coupling - case	2
5	P-015176	tube handle	2
6	P-015191	handle (yellow)	2
7	P-015799	coupling - pin	2
8	R-001694	rivet A2/A2 4.8x14	4

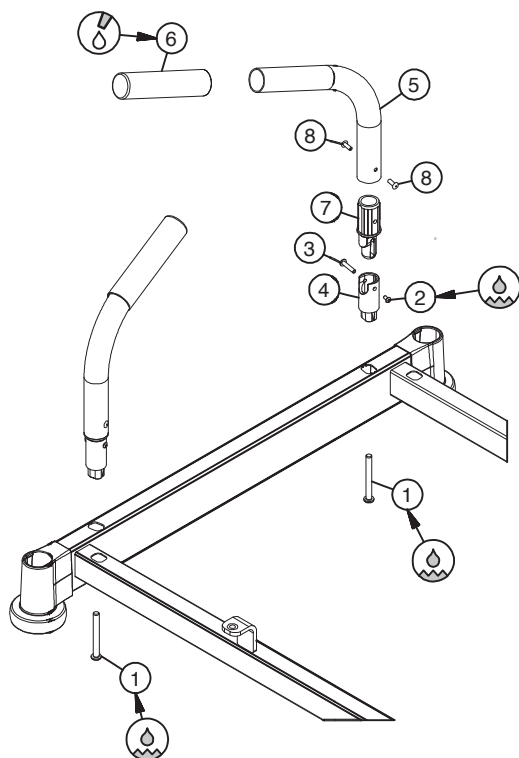


Option : Lockable panel


instead of handles at head and/or foot side

	ID	Description	Q
9	P-016267	lock	1
10	P-016534	brace Vela stretcher	1
11	P-016535	panel Vela strecher	1
12	R-000459	rivet Alu/St 3.9x10	1

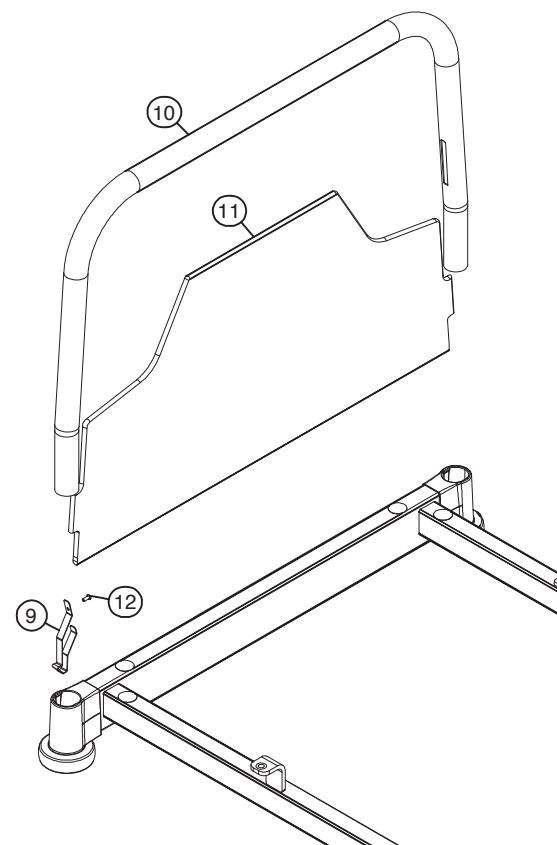

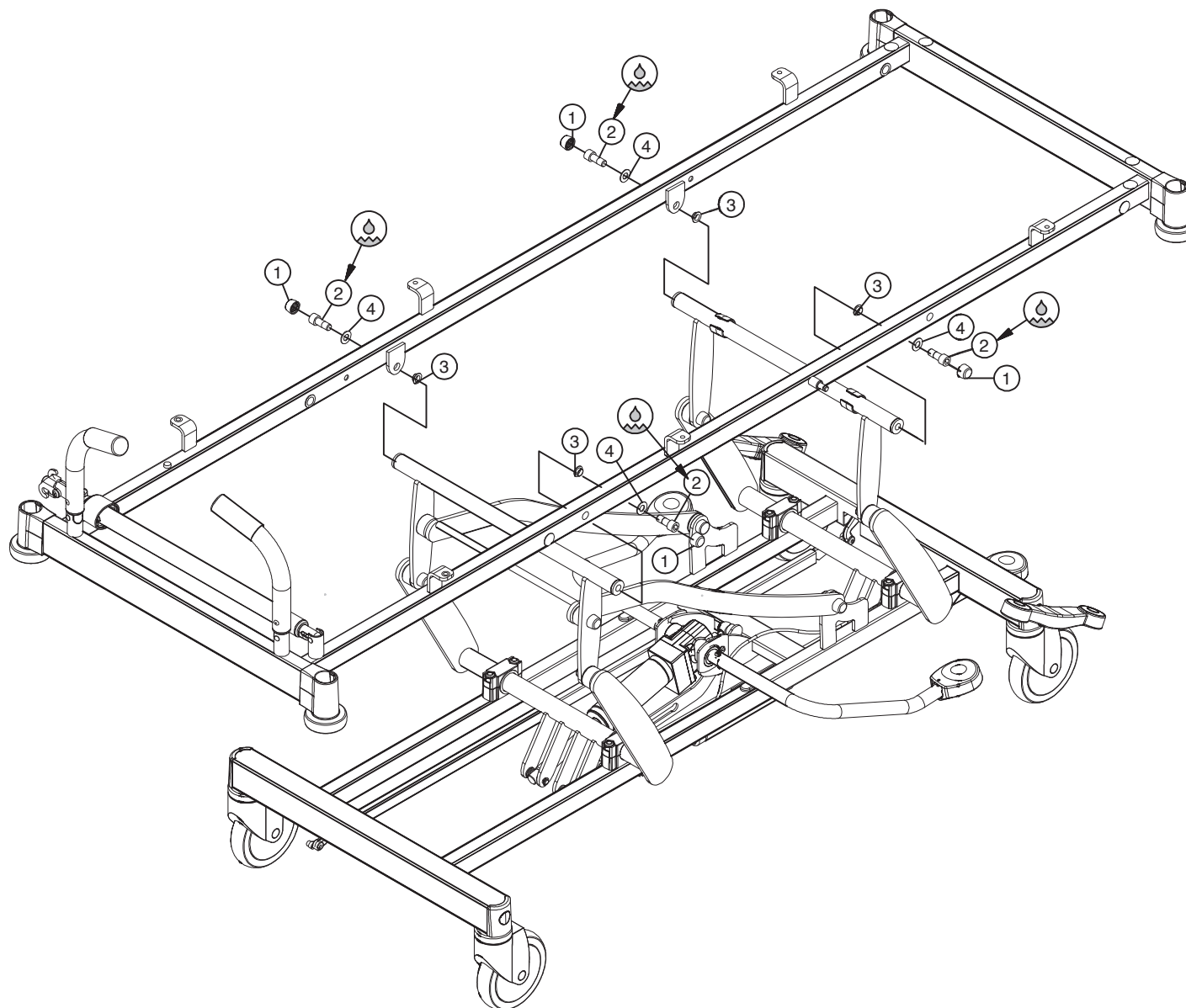



	ID	Description	Q
1	Ad003053	cap M12	4
2	B-003235	bolt Din912-M12x30x14-8.8 GV white	4
3	GI000620	bearing 1214-06	4
4	VI000638	bearing flange 1224-015	4

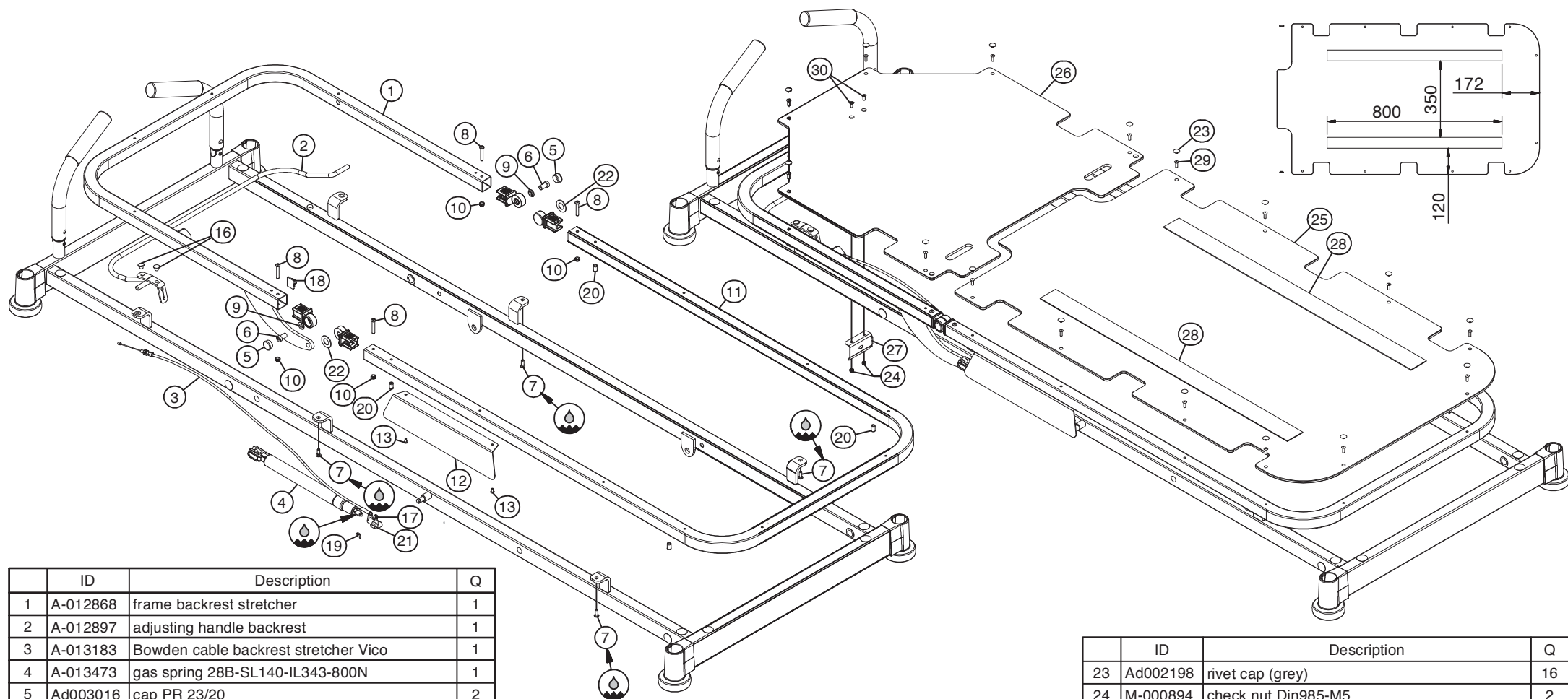


	ID	Description	Q
1	A-012868	frame backrest stretcher	1
2	A-012897	adjusting handle backrest	1
3	A-013183	Bowden cable backrest stretcher Vico	1
4	A-013473	gas spring 28B-SL140-IL343-800N	1
5	Ad003016	cap PR 23/20	2
6	B-003317	bolt-M10x18x12-GV white	2
7	B-005873	bolt Iso7380-M6x16x16-10.9 GV white	4
8	B-012413	bolt Iso7380-M6x35x35-10.9 GV white	4
9	GI000618	bearing 1012-04	2
10	M-000137	check nut Din985-M6	4
11	P-015062	tube fixed part	1
12	P-017474	cover plate gas spring	1
13	S-004413	screw Din7500 (TT85TX)-M4x8	2
14	Sc002558	matress platfrom hinge 1	2
15	Sc002560	matress platform hinge 2	2
16	St000141	buffer 14x2/8x7	2
17	St005074	release lever	1
18	St006353	retaining clip 7-8 (self-adhesive)	1
19	St010662	axle locking ring 14/7x1.3	1
20	St013341	insert small countersunk-M6x14.5-St GV	4
21	St016171	release head M10 dia10	1
22	VI002705	bearing flange 1630-015	2

	ID	Description	Q
23	Ad002198	rivet cap (grey)	16
24	M-000894	check nut Din985-M5	2
25	P-015063	perforated plate fixed part	1
26	P-015074	board backrest	1
27	P-016459	locking plate	1
28	P-016594	self-adhesive velcro 50mm (hook)	2
29	R-002201	rivet Alu/St 4.8x14	16
30	S-016477	srew Din965Z-M5x12-4.8 GV	2


OPTIONEN

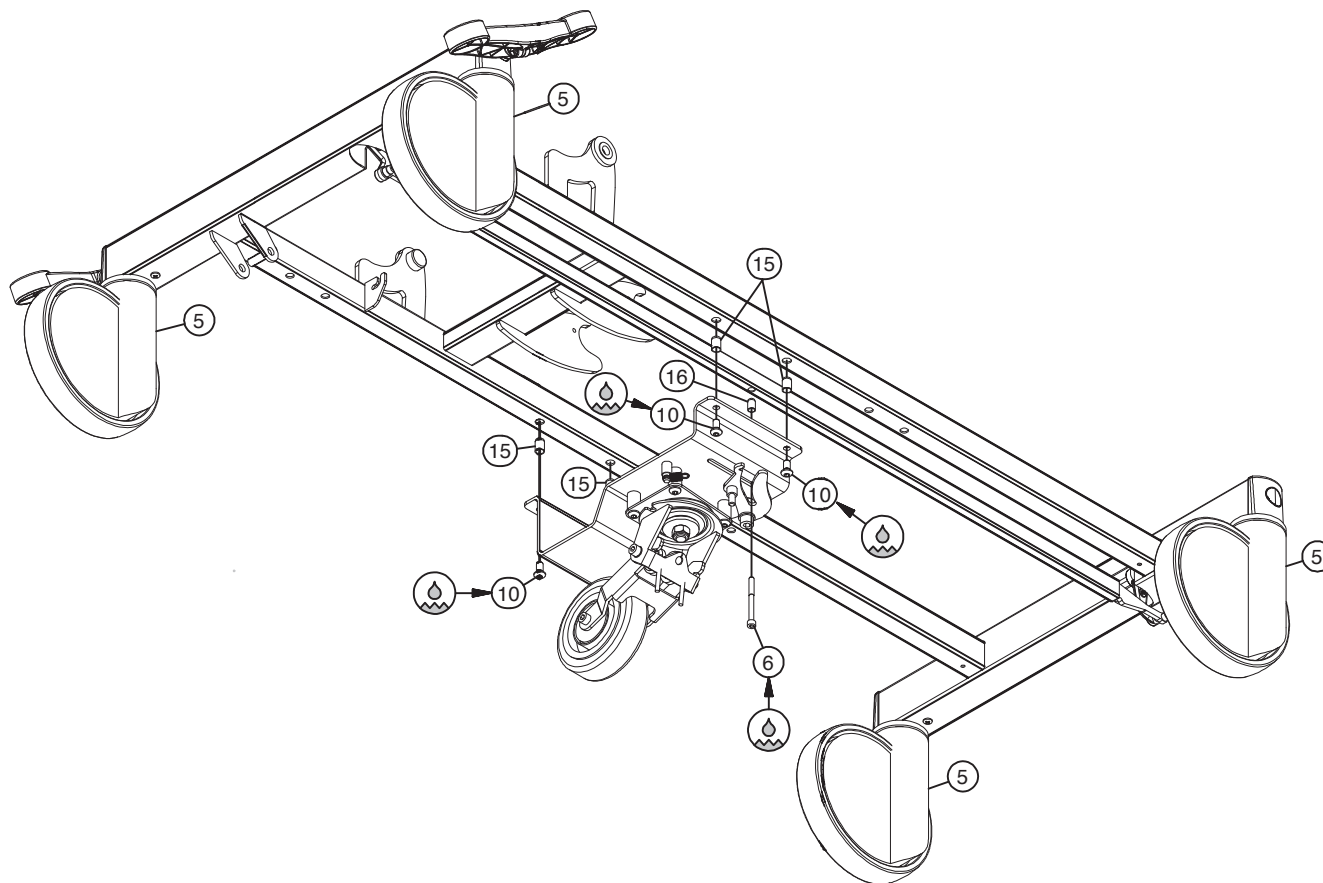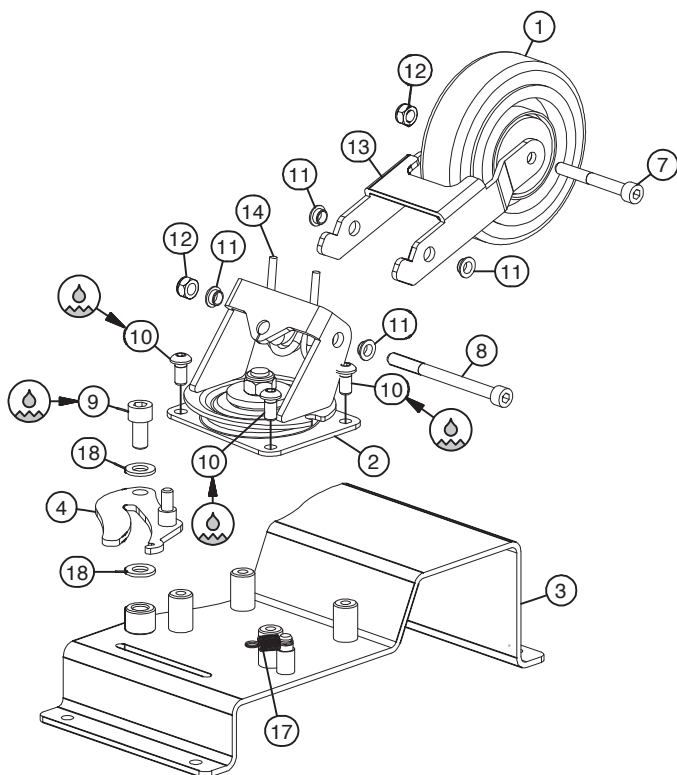

	ID	Description	Q
1	A-012132	wheel dia 125	1
2	A-012410	turning plate 5th wheel	1
3	A-013677	mounting plate 5th wheel stretcher	1
4	A-013679	locking device Vico	1
5	A-013696	castor dia 200	4
6	B-000661	bolt Din912-M6x65-8.8 GV white	1
7	B-000742	bolt Din912-M8x65-8.8 GV white	1
8	B-000747	bolt Din912-M8x100-8.8 GV white	1
9	B-000787	bolt Din912-M10x20x20-8.8 GV white	1
10	B-010844	bolt Iso7380-M8x16x16-10.9 GV white-LC	8
11	GI000510	bearing 081014-05	4
12	M-000189	check nut Din985-M8	2
13	P-013632	fork wheel	1
14	P-013833	spring direction wheel	1
15	St010602	insert CSK M8x16	4
16	St013341	insert CSK M6x14.5	1
17	St014684	spring 10x0.8x16/0.8 (2x double German eye)	1
18	VI000469	washer Din125A-M10-PA	2

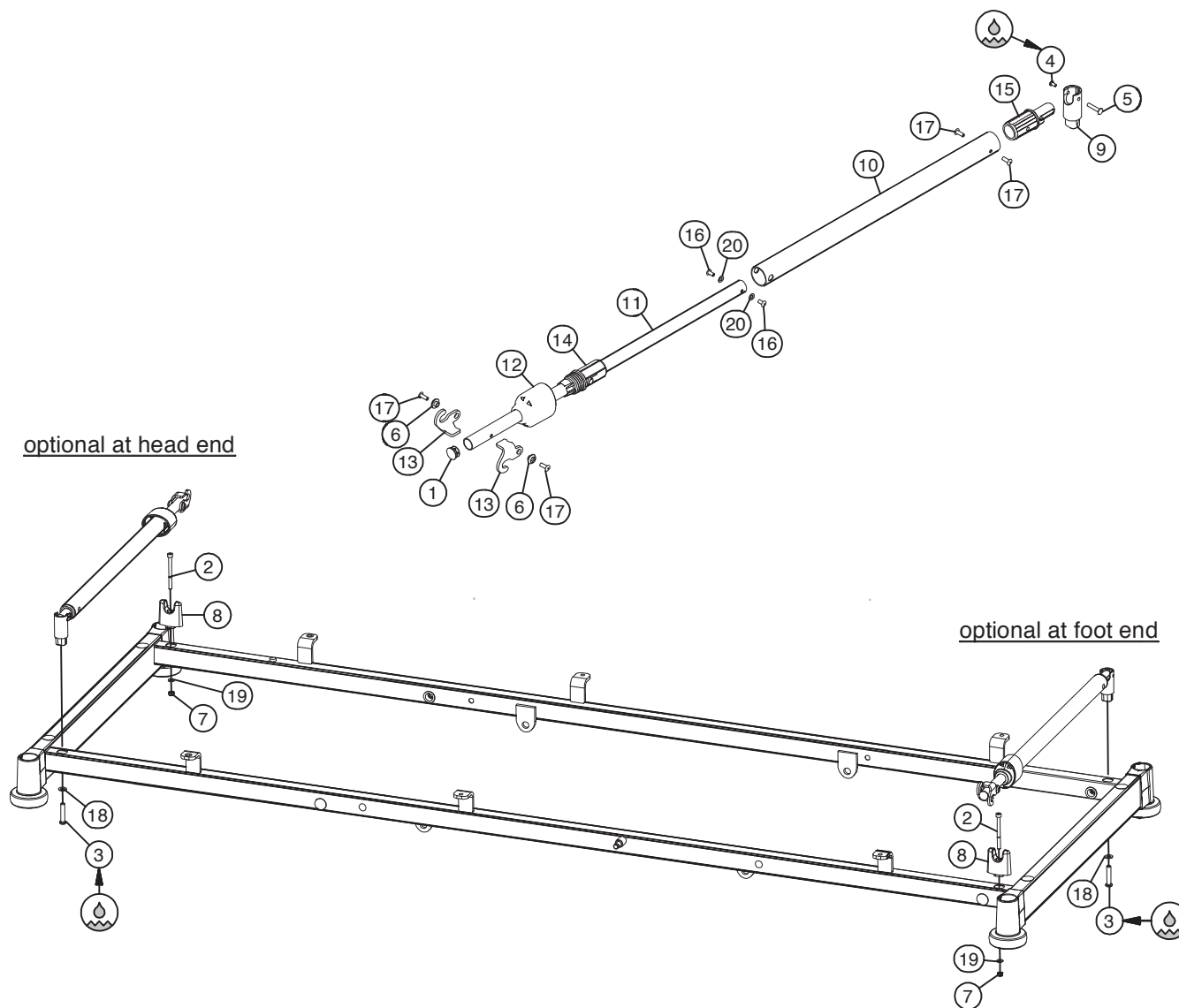


	ID	Description	Q
1	Ad016592	cap PR 19mm + chrome cap	2
2	B-000662	bolt Din912-M6x70-8.8 GV white	2
3	B-013424	bolt Iso7380-M8x40x40-10.9 GV wit	2
4	B-017743	bolt Iso7380-M4x8x8-10.9 GV wit	2
5	Db014239	threaded bush-M4x25-GV	2
6	Dc005468	fixation bush foot-rest	4
7	M-000137	check nut Din985-M6	2
8	P-015130	tube support dia 19	2
9	P-015131	coupling - case	2
10	P-015132	outer tube IV rod stretcher	2
11	P-015133	inner tube IV rod stretcher	2
12	P-015172	knob sleeve coupling 32-19	2
13	P-015174	hook IV rod	4
14	P-015732	sleeve coupling 32-19	2
15	P-015799	coupling - pin	2
16	R-000461	rivet Alu/St 4.8x10	4
17	R-001694	rivet A2/A2 4.8x14	8
18	VI000337	washer Din125A-M8-GV	2
19	VI000421	washer Din125A-M6-GV	2
20	VI001148	washer Din125A-M5-GV	4

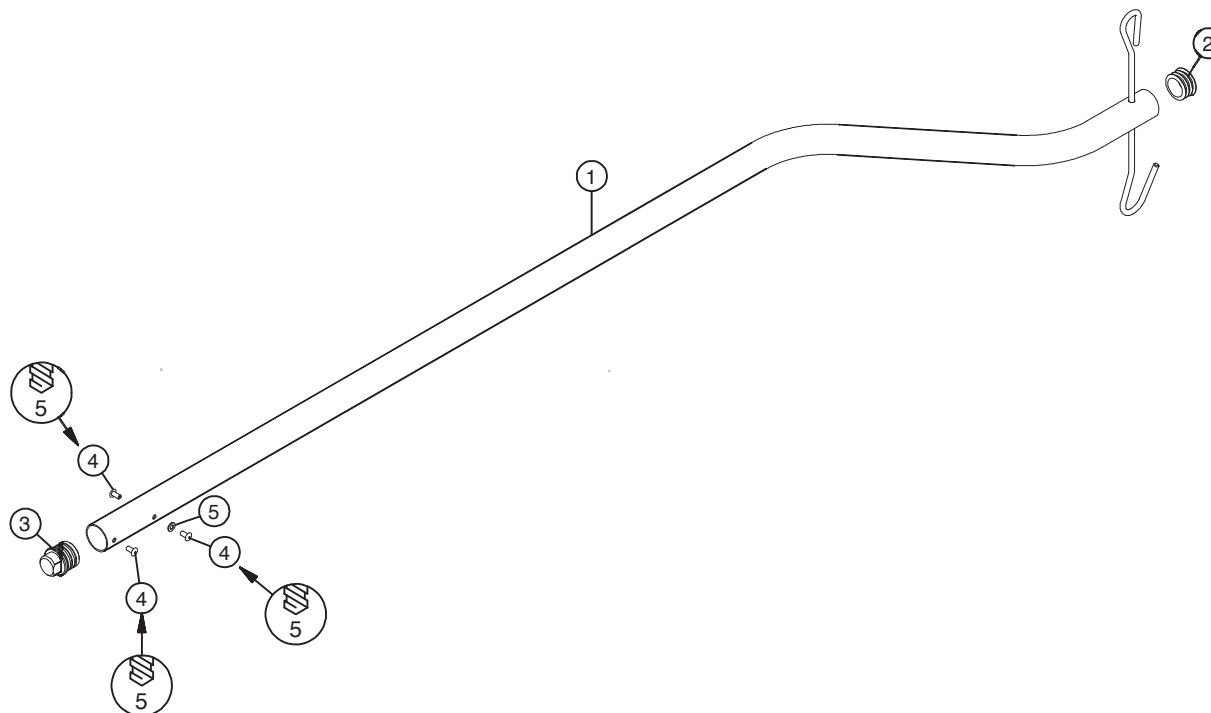


	ID	Description	Q
1	A-013954	IV-rod/diam.32/bent - stretcher	1
2	Ad005995	cap PR 32mm + chrome cap	1
3	Dc017091	positioning piece	1
4	R-000461	rivet Alu/St 4.8x10	3
5	VI001148	washer Din125A-M5-GV	1

assembly X-ray cassette holder


	ID	Description	Q
1	A-004014	spring button pin38	1
2	A-014267	position strip	1
3	Ad016638	blanking plug dia 9.5mm	4
4	M-011064	nut Din439-M12-04 GV white	1
5	P-015066	plate X-ray cassette stretcher	1
6	S-010982	screw Din965Z-M4x10-4.8 GV white	4
7	St016776	knob dia15-H21-M4	4
8	St017057	clamp handle dia50-M10x25	1

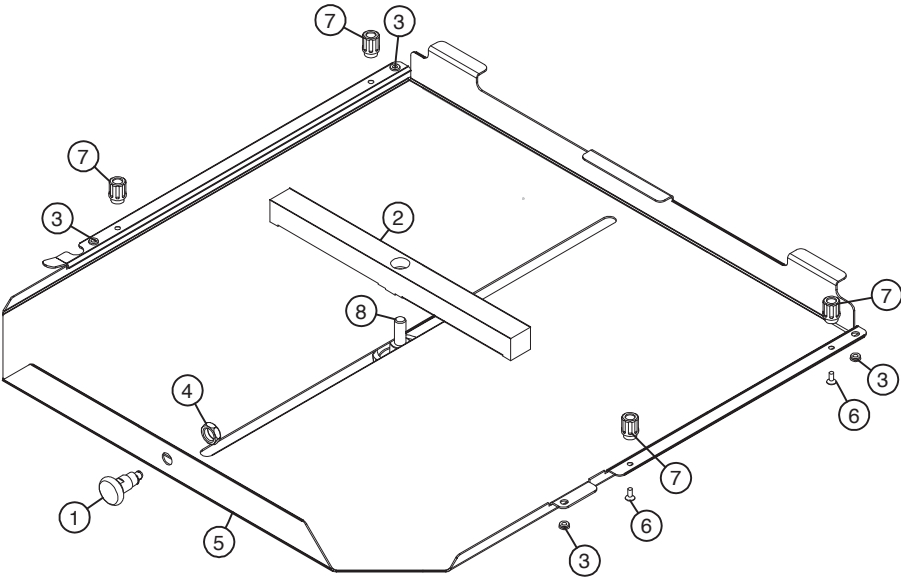


placement on mattress platform

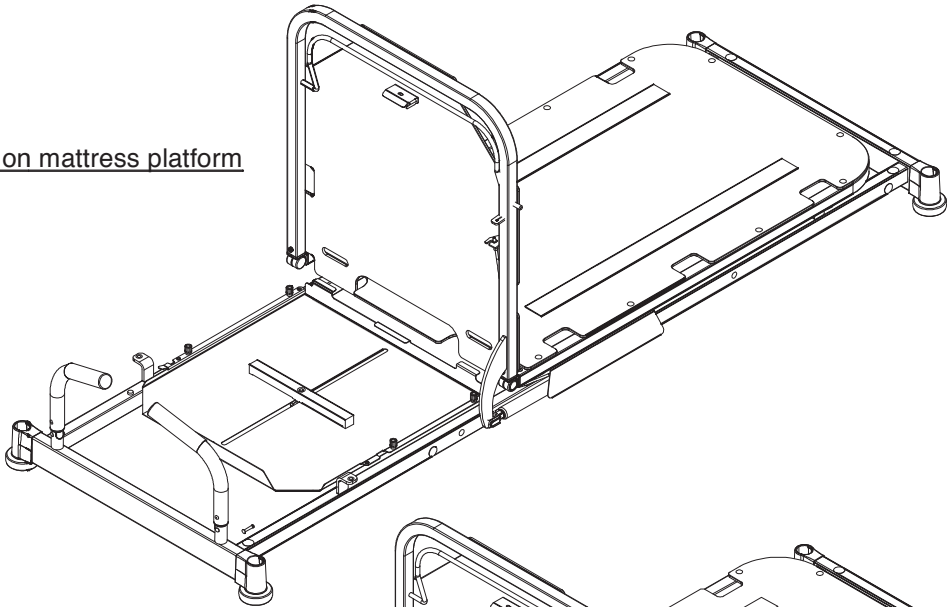


placement on backrest

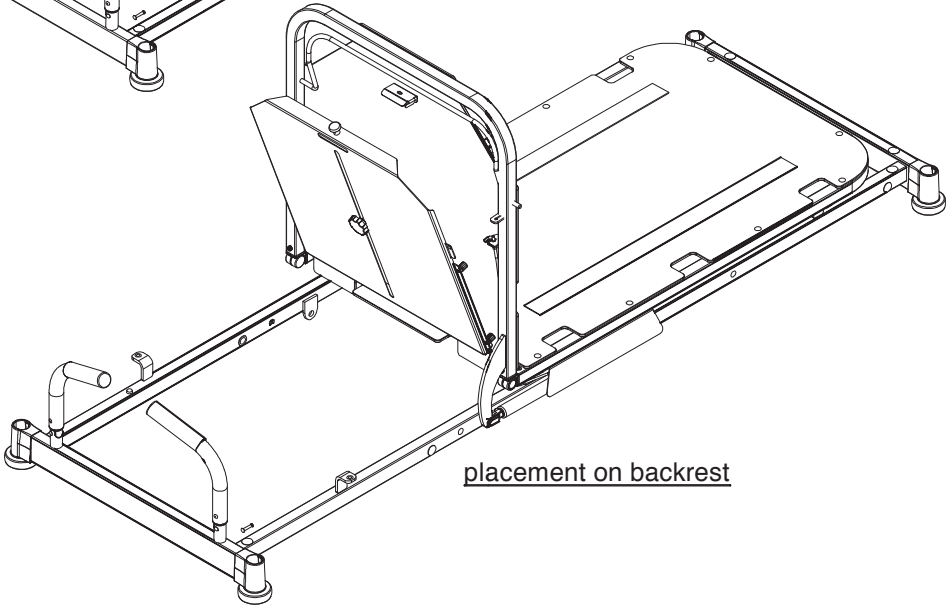


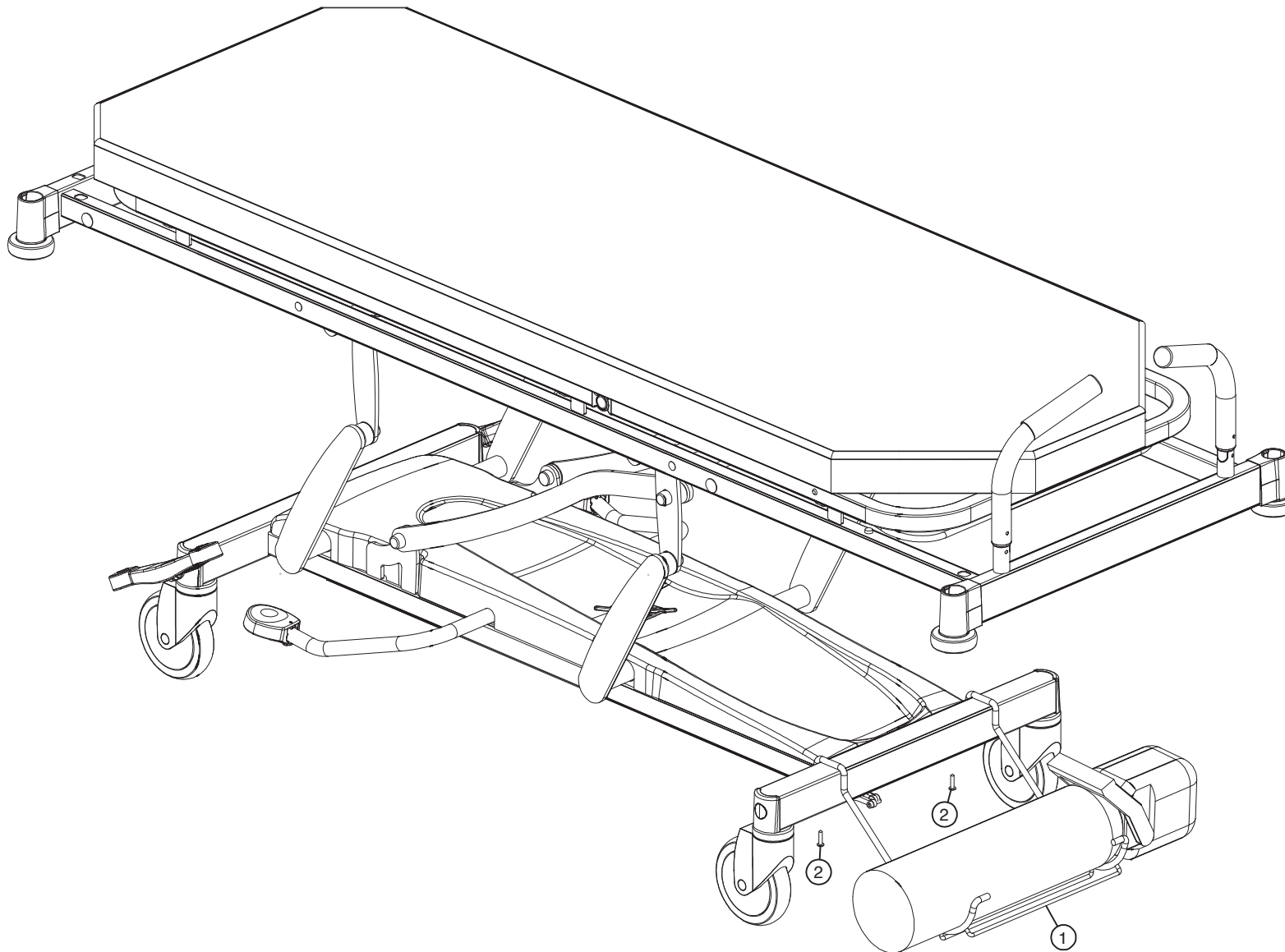


	ID	Description	Q
1	A-013647	oxygen cylinder holder carrier	1
2	S-015945	screw Din7981C-5.5x25	2

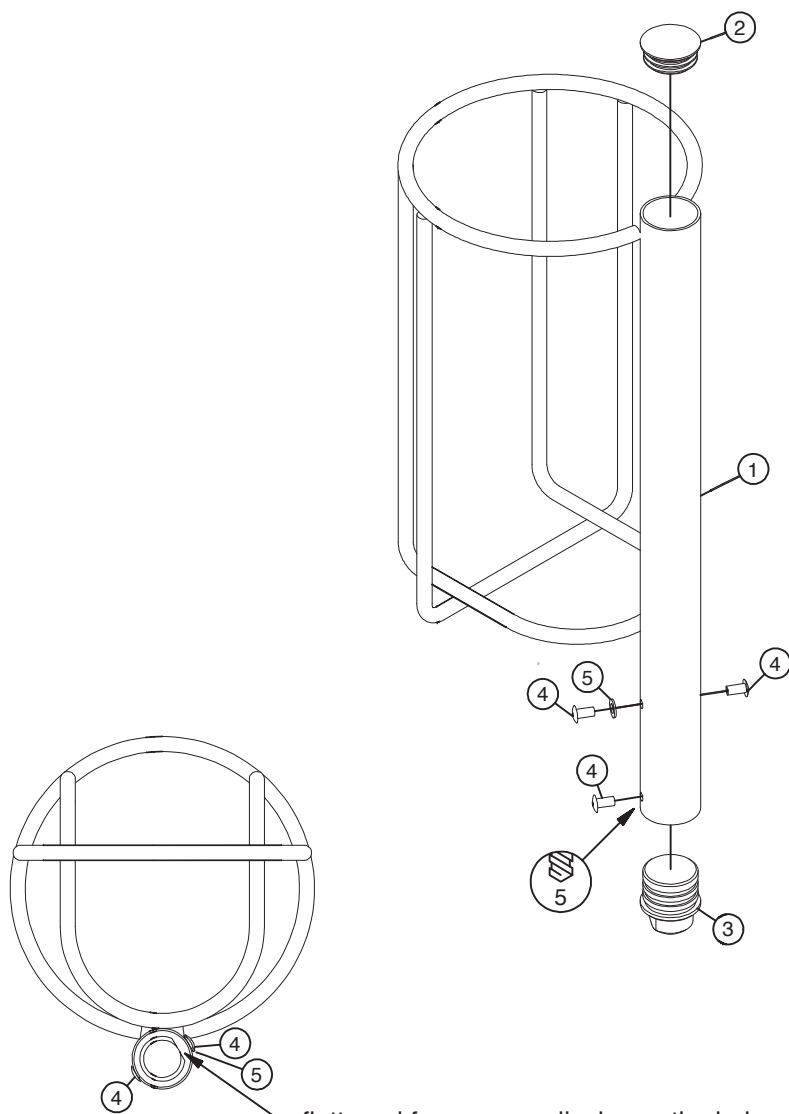



flattened face perpendicular on the holes
+ in the direction of the basket

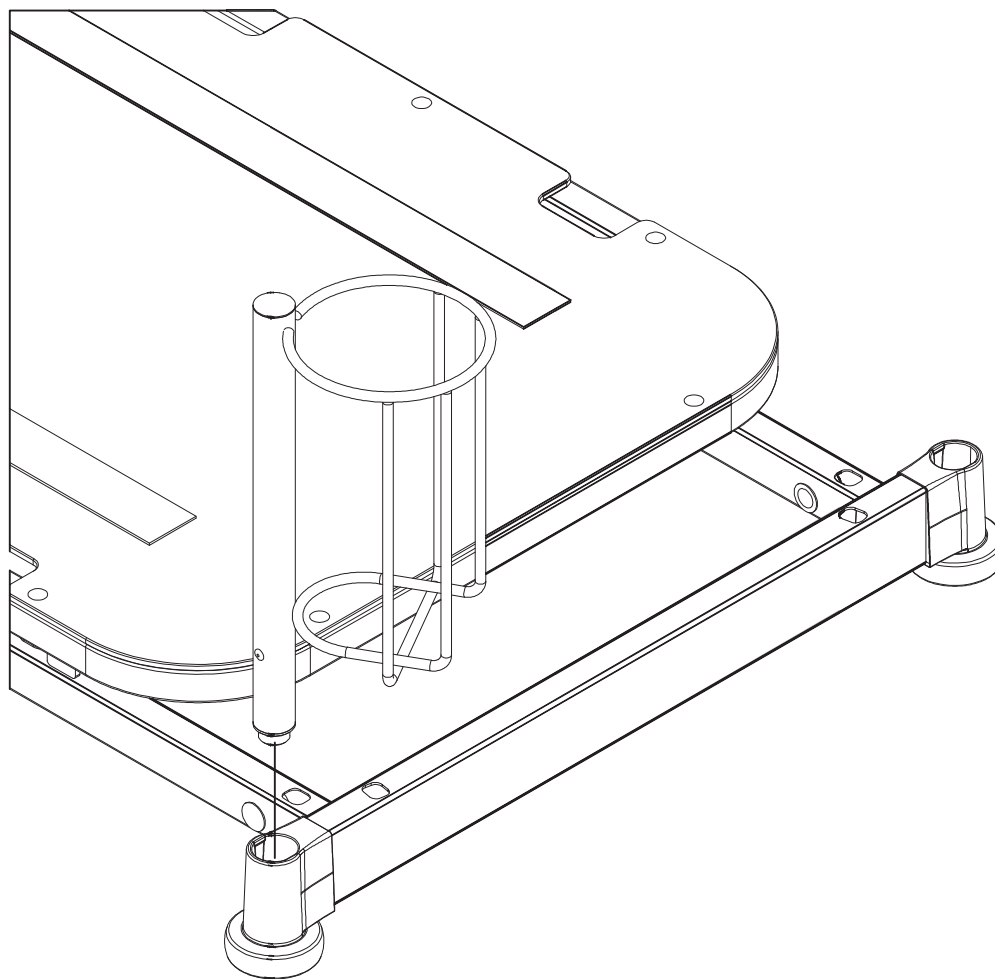


	ID	Description	Q
1	A-013920	oxygen cylinder holder stretcher	1
2	Ad000495	cap PR 32mm	1
3	Dc017091	positioning piece	1
4	R-000461	rivet Alu/St 4.8x10	3
5	VI001148	washer Din125A-M5-GV	1

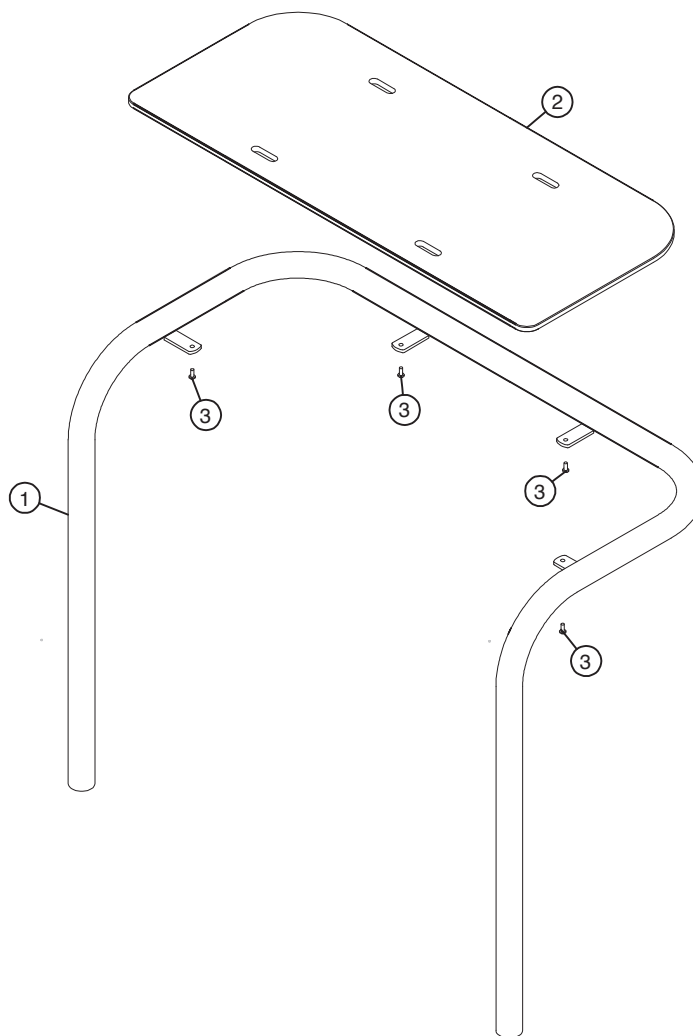

	ID	Description	Q
1	A-014291	equipment holder stretcher Vico	1
2	P-017937	board equipment holder stretcher Vico	1
3	S-004400	screw PLASTITE 7985-4x8-GV	4

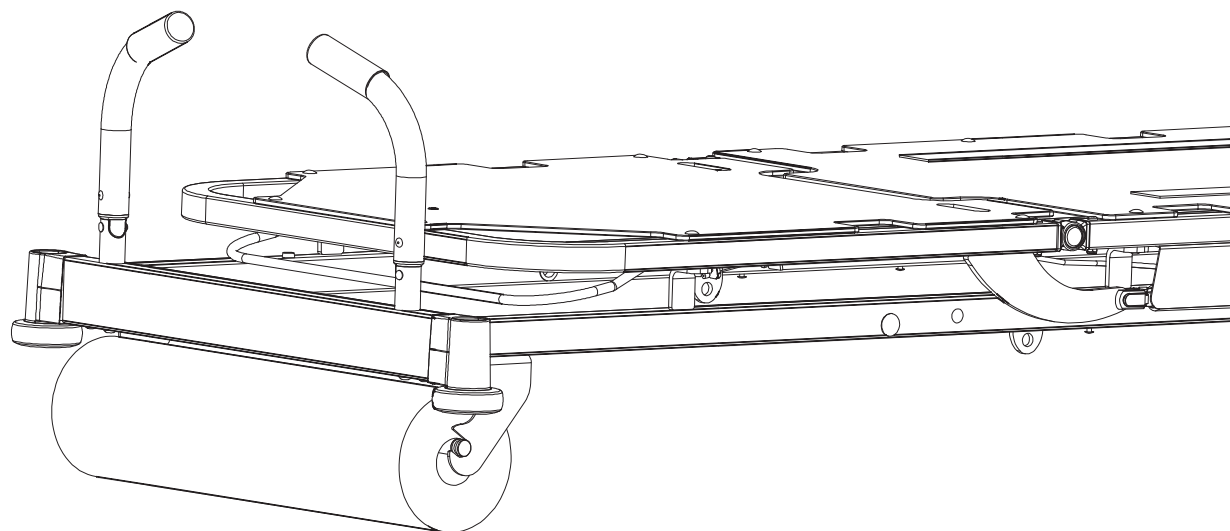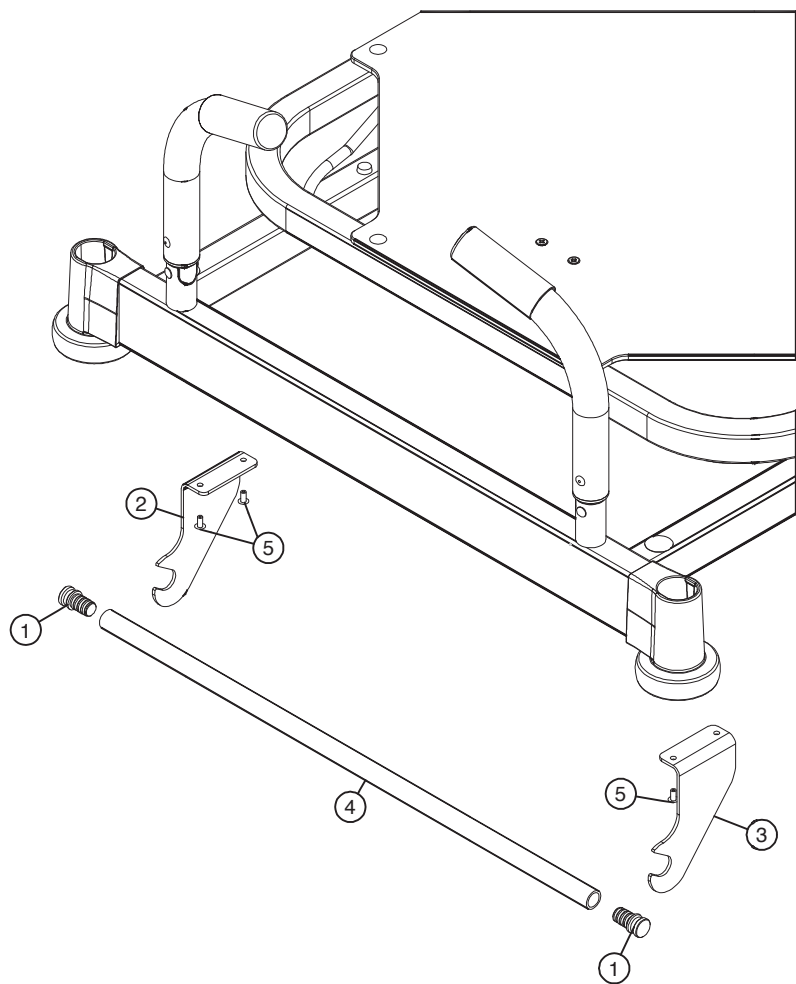

	ID	Description	Q
1	Dc017639	cap paper roll holder	2
2	P-017637	fork paper roll holder L	1
3	P-017638	fork paper roll holder R	1
4	P-017644	tube paper roll holder 60cm	1
5	R-000461	rivet Alu/St 4.8x10	4

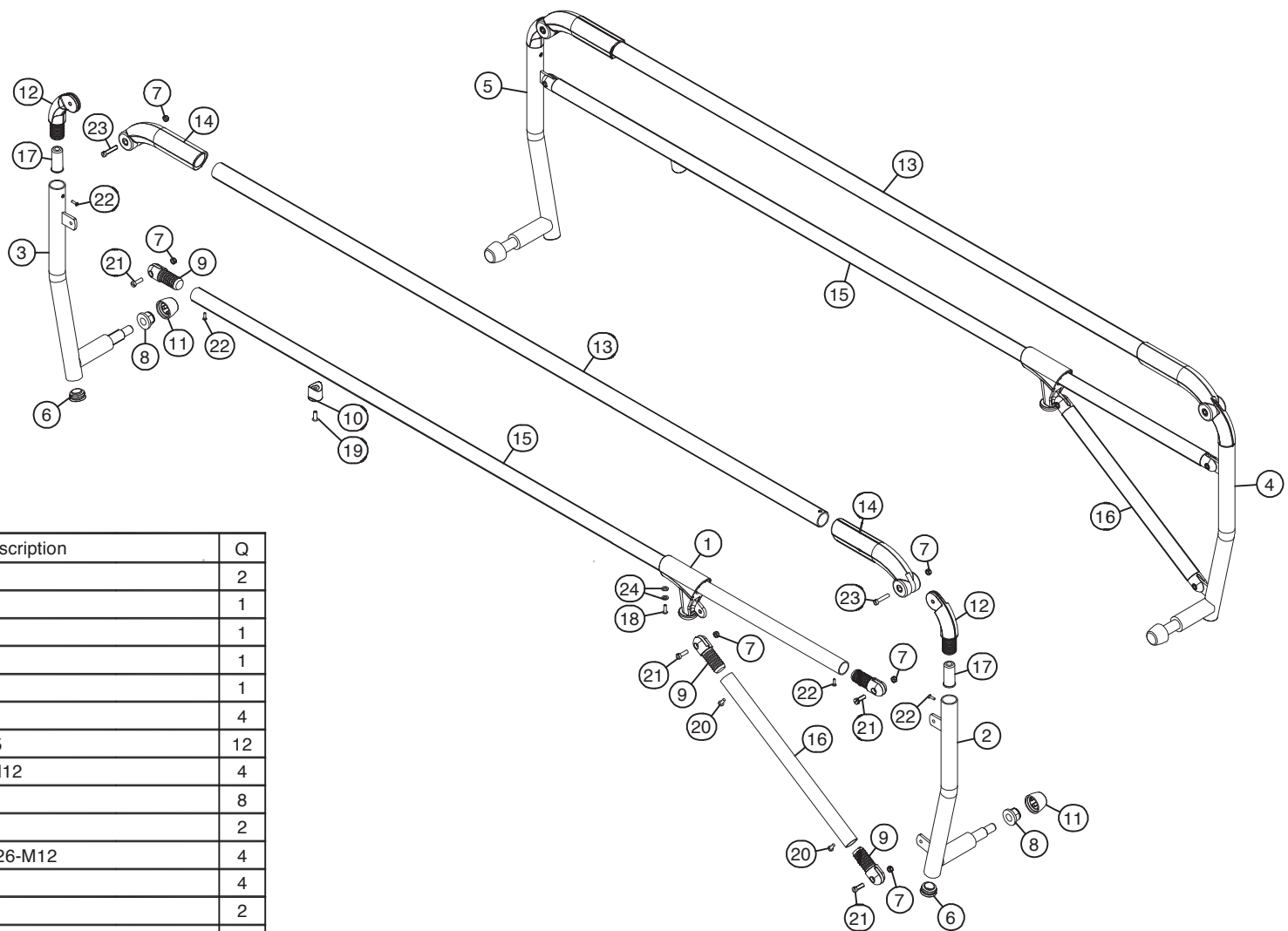

	ID	Description	Q
1	A-003049	sliding piece side rail	2
2	A-012856	post L foot side	1
3	A-012857	post L head side	1
4	A-012858	post R foot side	1
5	A-012859	post R head side	1
6	Ad000890	cap PR 25mm	4
7	M-000894	check nut Din985-M5	12
8	M-001151	check nut Din6926-M12	4
9	P-000892	joint 22mm	8
10	P-006434	spacer	2
11	P-010973	cap check nut Din6926-M12	4
12	P-013703	support cap side rail	4
13	P-013742	top tube Aron	2
14	P-014283	handle side rail Atmosphere	4
15	P-015068	bottom tube stretcher Vico	2
16	P-015069	inclinated tube stretcher Vico	2
17	P-015851	insert support	4
18	R-001149	rivet A2/A2 4,0x13	2
19	R-001694	rivet A2/A2 4.8x14	2
20	S-000830	screw Din7981C-4.2x13	4
21	S-000897	screw Din84-M5x16-8.8GV	8
22	S-001139	screw Din7505A-3x12-GV	8
23	S-002989	screw Din84-M5x25-8.8GV	4
24	VI001148	washer Din125A-M5-GV	4