


## Bill of Material



---

### Avento HL

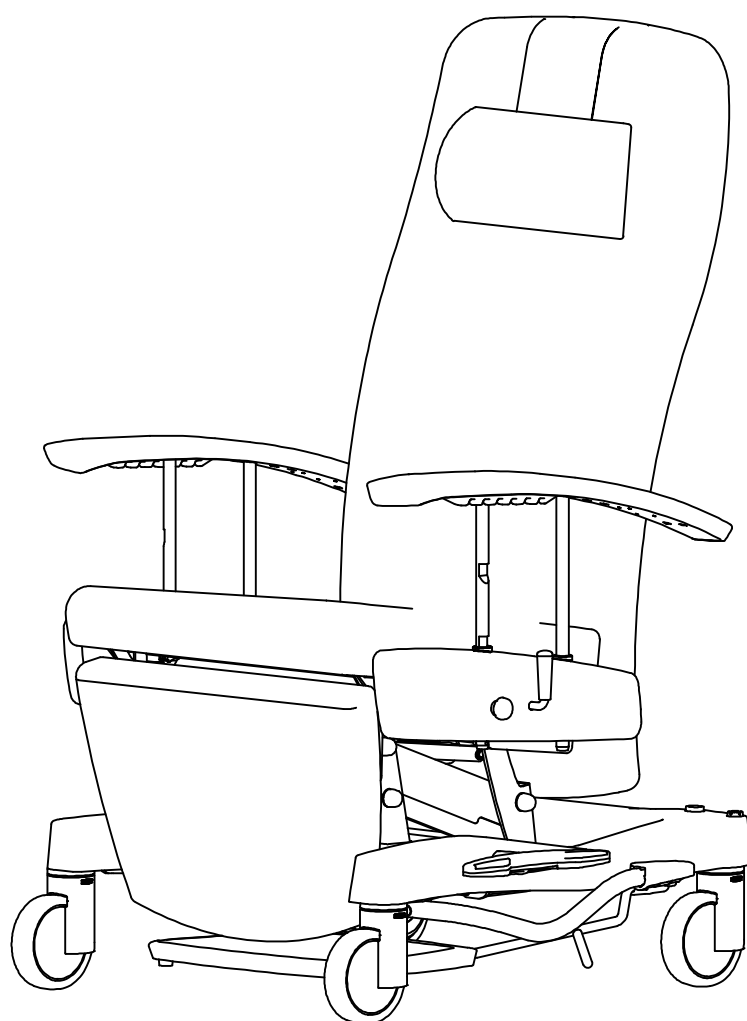


T-039373

a

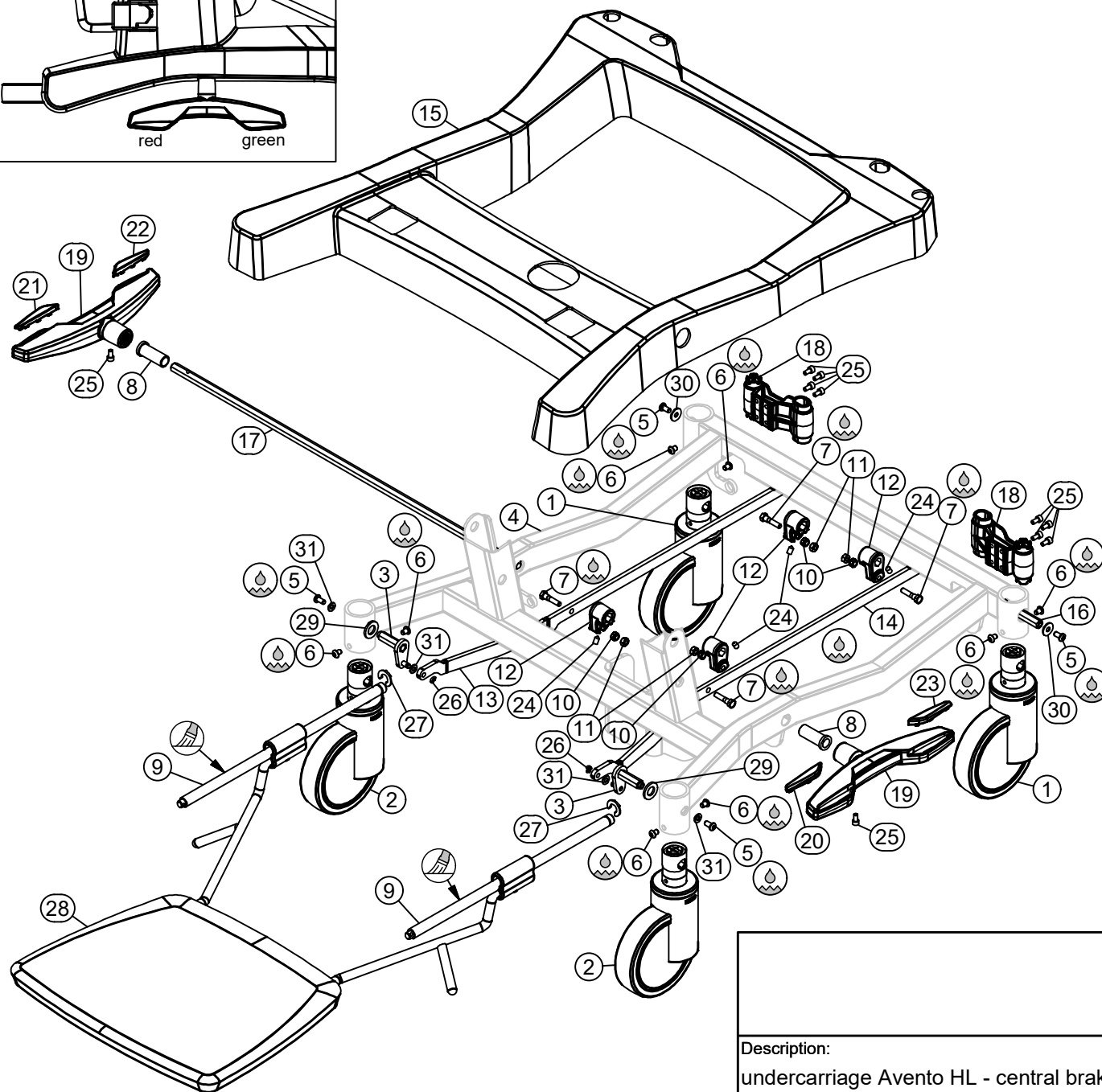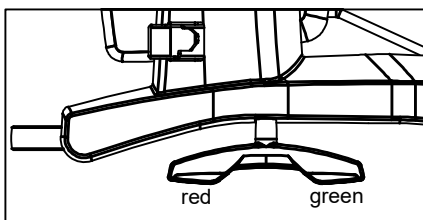


|    | ID       | Description                                       | Q  |
|----|----------|---------------------------------------------------|----|
| 1  | A-020494 | Integral S 125-R36-32S35-DC                       | 2  |
| 2  | A-021881 | Integral S 125-R36-32S35                          | 2  |
| 3  | A-022138 | brake lever                                       | 2  |
| 4  | A-026431 | undercarriage Avento HL central braked 201910     | 1  |
| 5  | B-004168 | bolt Iso7380-M6x12x12-10.9 GV white-LC            | 4  |
| 6  | B-006478 | bolt Iso7380-M6x8x8-10.9 GV white - LC            | 8  |
| 7  | B-013711 | bolt central brake                                | 4  |
| 8  | DC028090 | bearing central brake pedal                       | 2  |
| 9  | DC029328 | footrest sliding rail Avento HL                   | 2  |
| 10 | M-000137 | check nut Din985-M6                               | 4  |
| 11 | M-001357 | nut Din934-M6-8-GV                                | 4  |
| 12 | P-005676 | switch cam 24mm                                   | 4  |
| 13 | P-024955 | brake link R Avento HL                            | 1  |
| 14 | P-024994 | brake link L Avento HL                            | 1  |
| 15 | P-025170 | cover undercarriage Avento HL central brake       | 1  |
| 16 | P-025809 | hexagonal bar back Avento HL                      | 1  |
| 17 | P-025810 | hexagonal bar middle Avento HL                    | 1  |
| 18 | P-025832 | IV rod & headrest support                         | 2  |
| 19 | P-026154 | double brake pedal Avento HL                      | 2  |
| 20 | P-036141 | indicator B RAL3020 red double brake pedal 2016   | 1  |
| 21 | P-036142 | indicator A RAL3020 red double brake pedal 2016   | 1  |
| 22 | P-036143 | indicator B RAL6024 green double brake pedal 2016 | 1  |
| 23 | P-036144 | indicator A RAL6024 green double brake pedal 2016 | 1  |
| 24 | S-001939 | set screw Din914-M6x10                            | 4  |
| 25 | S-014995 | screw type TT12-M5x10-GV white                    | 10 |
| 26 | ST000464 | springfix diam.5                                  | 2  |
| 27 | ST010715 | axe locking ring 23/12x1.6                        | 2  |
| 28 | T-037524 | foot rest Avento HL 201910                        | 1  |
| 29 | VL002182 | washer Din125A-M12-PA                             | 2  |
| 30 | VL013506 | washer Din9021-M6-PA                              | 2  |
| 31 | VL020302 | washer Din125A-M6-PA                              | 4  |

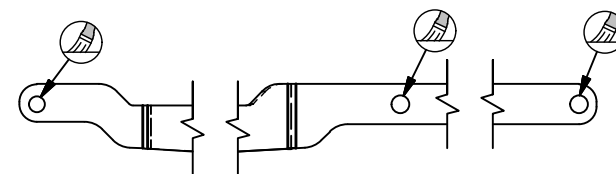


|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| Description:                                                                                                                                                                    |  | Designer:                                                                                      |   |
| undercarriage Avento HL - central brake 202005                                                                                                                                  |  | Bogdan O.                                                                                      |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 22/05/2020                                                                               | Scale: 1 : 8                                                                                                                                                                |
|                                                                                                                                                                                 |  | Drawing nr.: T-039373                                                                          | Revision nr.: a                                                                                                                                                             |

|          |                     |   |
|----------|---------------------|---|
| T-039376 |                     | a |
| Options  |                     |   |
| ID.      | Description         |   |
| A        | IV rod holder Ø32 R |   |
| B        | IV rod holder Ø32 L |   |

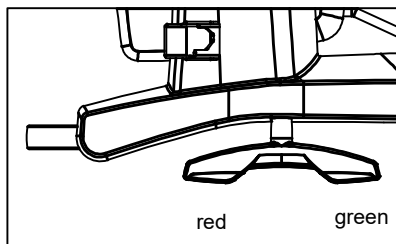

Orientation accessory cases

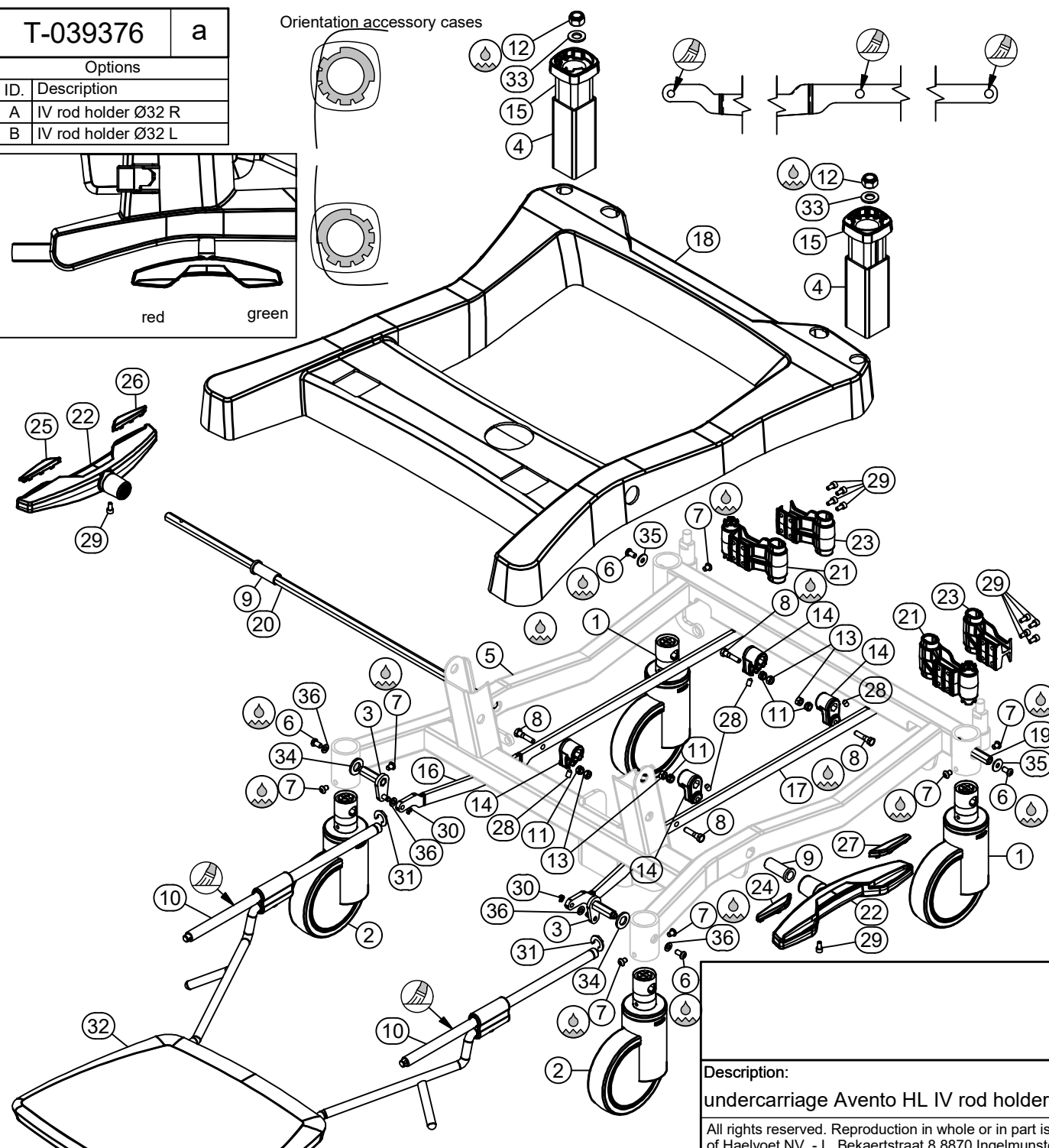

|    | ID       | Description                                         | Q         |
|----|----------|-----------------------------------------------------|-----------|
| 1  | A-020494 | Integral S 125-R36-32S35-DC                         | 2         |
| 2  | A-021881 | Integral S 125-R36-32S35                            | 2         |
| 3  | A-022138 | brake lever                                         | 2         |
| 4  | A-026435 | IV rod holder Avento HL 32 mm 201910                | A+1;B+1 0 |
| 5  | A-026436 | undercarriage Avento HL IV rod holders 32 mm 201910 | 1         |
| 6  | B-004168 | bolt Iso7380-M6x12x12-10.9 GV white-LC              | 4         |
| 7  | B-006478 | bolt Iso7380-M6x8x8-10.9 GV white - LC              | 8         |
| 8  | B-013711 | bolt central brake                                  | 4         |
| 9  | DC028090 | bearing central brake pedal                         | 2         |
| 10 | DC029328 | footrest sliding rail Avento HL                     | 2         |
| 11 | M-000137 | check nut Din985-M6                                 | 4         |
| 12 | M-001068 | check nut Din985-M12                                | A+1;B+1 0 |
| 13 | M-001357 | nut Din934-M6-8-GV                                  | 4         |
| 14 | P-005676 | switch cam 24mm                                     | 4         |
| 15 | P-017129 | accessory case                                      | A+1;B+1 0 |
| 16 | P-024955 | brake link R Avento HL                              | 1         |
| 17 | P-024994 | brake link L Avento HL                              | 1         |
| 18 | P-025170 | cover undercarriage Avento HL central brake         | 1         |
| 19 | P-025809 | hexagonal bar back Avento HL                        | 1         |
| 20 | P-025810 | hexagonal bar middle Avento HL                      | 1         |
| 21 | P-025832 | IV rod & headrest support                           | A-1;B-1 2 |
| 22 | P-026154 | double brake pedal Avento HL                        | 2         |
| 23 | P-030250 | IV rod support 32 mm Avento HL                      | A+1;B+1 0 |
| 24 | P-036141 | indicator B RAL3020 red double brake pedal 2016     | 1         |
| 25 | P-036142 | indicator A RAL3020 red double brake pedal 2016     | 1         |
| 26 | P-036143 | indicator B RAL6024 green double brake pedal 2016   | 1         |
| 27 | P-036144 | indicator A RAL6024 green double brake pedal 2016   | 1         |
| 28 | S-001939 | set screw Din914-M6x10                              | 4         |
| 29 | S-014995 | screw type TT12-M5x10-GV white                      | 10        |
| 30 | ST000464 | springfix diam.5                                    | 2         |
| 31 | ST010715 | axe locking ring 23/12x1.6                          | 2         |
| 32 | T-037524 | foot rest Avento HL 201910                          | 1         |
| 33 | VL000484 | washer Din125A-M12-GV                               | A+1;B+1 0 |
| 34 | VL002182 | washer Din125A-M12-PA                               | 2         |
| 35 | VL013506 | washer Din9021-M6-PA                                | 2         |
| 36 | VL020302 | washer Din125A-M6-PA                                | 4         |



Description:

undercarriage Avento HL IV rod holders 32 mm - central brake 202005


All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95

Designer:

Ionut I.

Date: 22/05/2020

Drawing nr.: T-039376

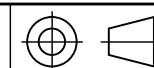


Scale: 1 :8

Revision nr.: a


T-037524

b

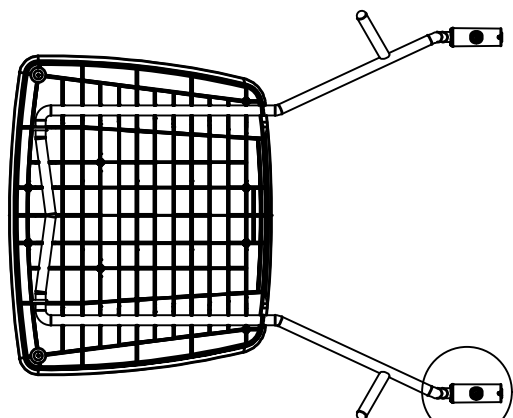


B

B ( 1 : 2 )

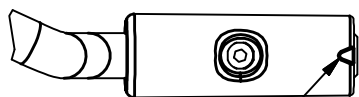


Arrow to the front

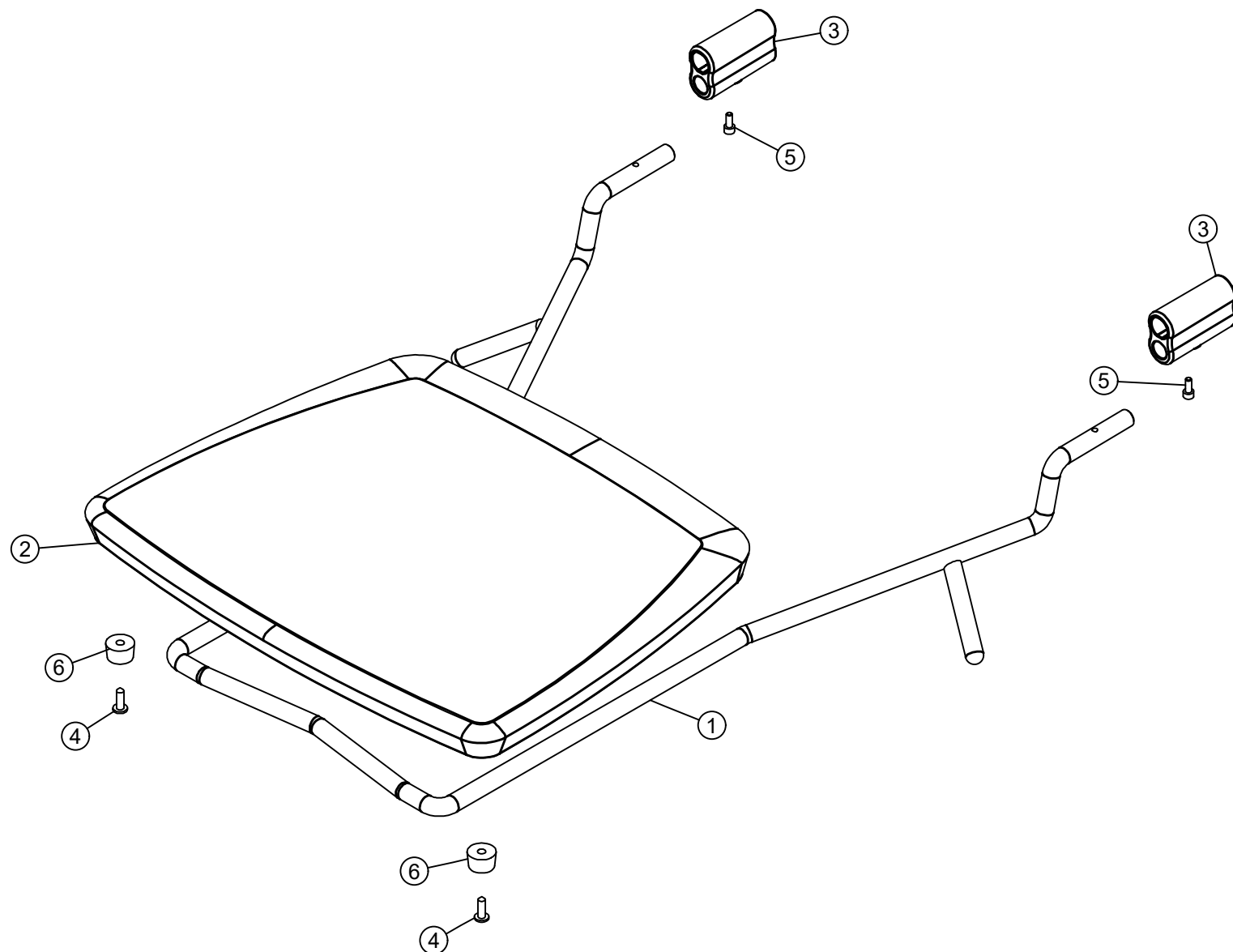

|   | ID       | Description                              | Q |
|---|----------|------------------------------------------|---|
| 1 | A-026432 | footrest support Avento 201910           | 1 |
| 2 | P-030832 | U-shape fixation standard footrest board | 1 |
| 3 | P-035649 | footrest guide                           | 2 |
| 4 | S-000382 | screw Din7981C-4.8x16                    | 2 |
| 5 | S-025842 | screw Din7500(TT12)-M4x10-GV             | 2 |
| 6 | ST005281 | buffer 19/11                             | 2 |



|                                                                                                                                                                                 |  |                                                          |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|----------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  <b>Haelvoet</b>                                                                           |  | <b>Description:</b><br><b>foot rest Avento HL 201910</b> |                                                                                                                                                                             |
|                                                                                                                                                                                 |  | <b>Designer:</b><br>Ionut I.                             |   |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | <b>Date:</b> 21/10/2019<br><b>Drawing nr.:</b> T-037524  | <b>Scale:</b> 1 : 4<br><b>Revision nr.:</b> b                                                                                                                               |

|    | ID       | Description                                  | Q  |
|----|----------|----------------------------------------------|----|
| 1  | A-022112 | HL arm Avento HL                             | 1  |
| 2  | A-022116 | stabilisator arm Avento HL                   | 1  |
| 3  | A-022122 | main support Avento HL                       | 1  |
| 4  | A-022416 | hydraulic pump 10000N-IL202-SL80             | 1  |
| 5  | AD000435 | buffer                                       | 2  |
| 6  | AD003053 | grey cap M12                                 | 10 |
| 7  | B-003235 | bolt Din912-M12x30x20-8.8 GV white           | 6  |
| 8  | B-012047 | bolt Iso7380-M8x30x30-10.9 GV white          | 2  |
| 9  | B-018125 | bolt Iso7380-M6x10x10-10.9 GV white          | 2  |
| 10 | B-027583 | bolt Din912-M12x20x15-8.8 GV white           | 4  |
| 11 | DC027515 | axle 10x42                                   | 1  |
| 12 | GL017646 | bearing 101216-12                            | 2  |
| 13 | GL025719 | bearing 1214-04                              | 6  |
| 14 | GI000620 | bearing 1214-06                              | 4  |
| 15 | M-000189 | check nut Din985-M8-8-GV                     | 2  |
| 16 | P-024987 | hydraulic pump fixation ring                 | 2  |
| 17 | P-028260 | high-low pedal                               | 2  |
| 18 | P-029582 | tube HL pedal L Avento HL                    | 1  |
| 19 | P-029584 | tube HL pedal R Avento HL                    | 1  |
| 20 | S-014995 | screw type TT12-M5x10-GV white               | 2  |
| 21 | ST010662 | axle locking ring 14/7x1.3                   | 2  |
| 22 | T-039373 | undercarriage Avento HL central brake 202005 | 1  |
| 23 | VL000469 | washer Din125A-M10-PA                        | 2  |
| 24 | VL002143 | washer Din6798A-M6                           | 2  |
| 25 | VL002364 | bearing flange 2036-015                      | 2  |
| 26 | VI000638 | bearing flange 1224-015                      | 8  |

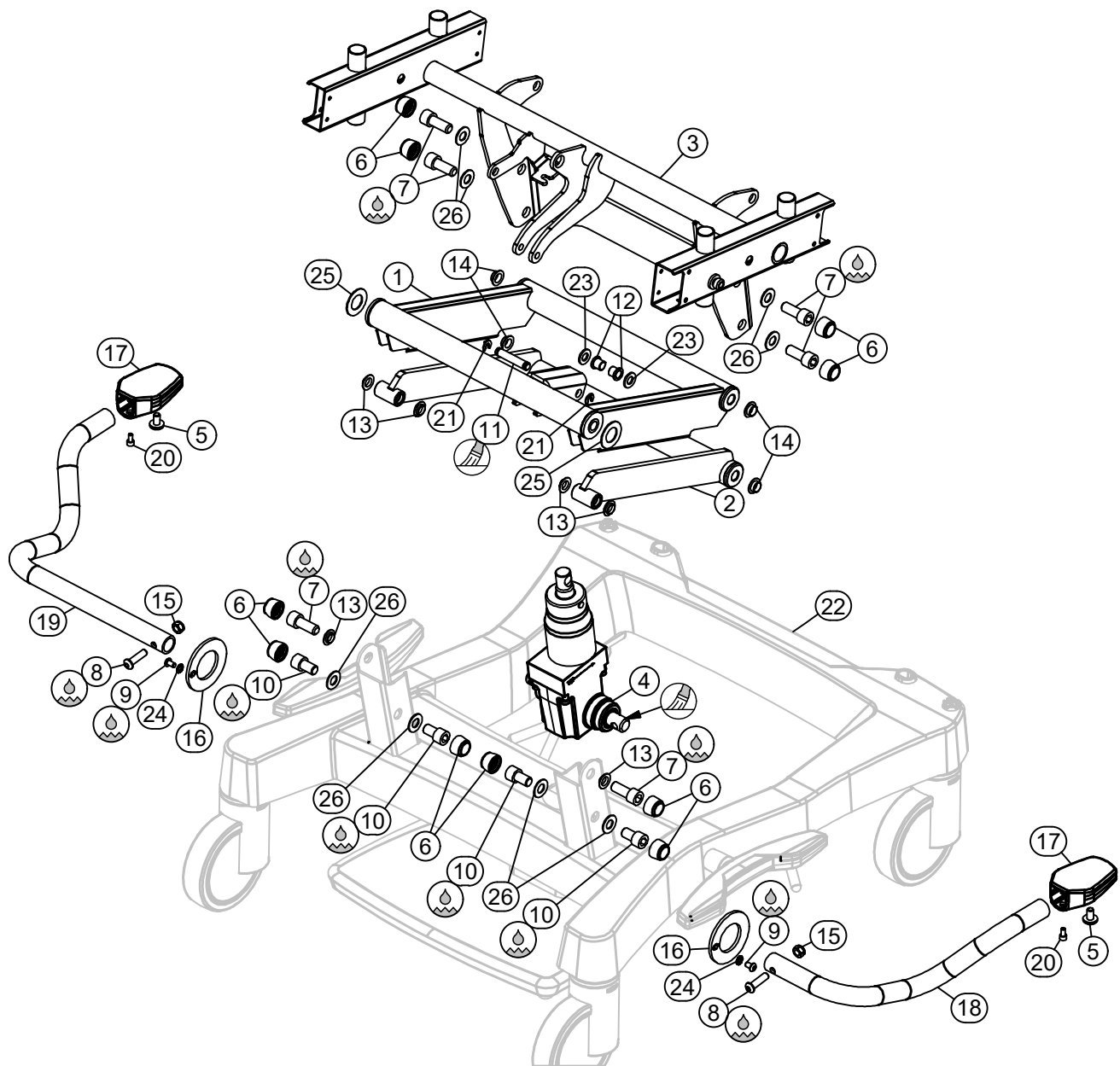


|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| <b>Description:</b><br>undercarriage + HL system Avento HL 202005                                                                                                               |  | <b>Designer:</b><br>Ionut I.                                                                   |   |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | <b>Date:</b> 25/05/2020<br><b>Drawing nr.:</b> T-039394                                        | <b>Scale:</b> 1 : 8<br><b>Revision nr.:</b> a                                                                                                                               |


|          |                            |   |
|----------|----------------------------|---|
| T-039377 |                            | a |
| Options  |                            |   |
| ID.      | Description                |   |
| A        | Castors 100 mm directional |   |
| B        | Castors 125 mm             |   |
| C        | Removable head part        |   |
| D        | IV rod holder R            |   |
| E        | IV rod holder L            |   |

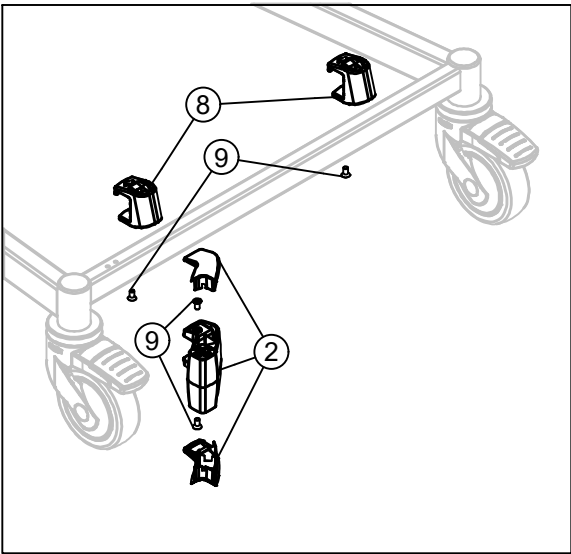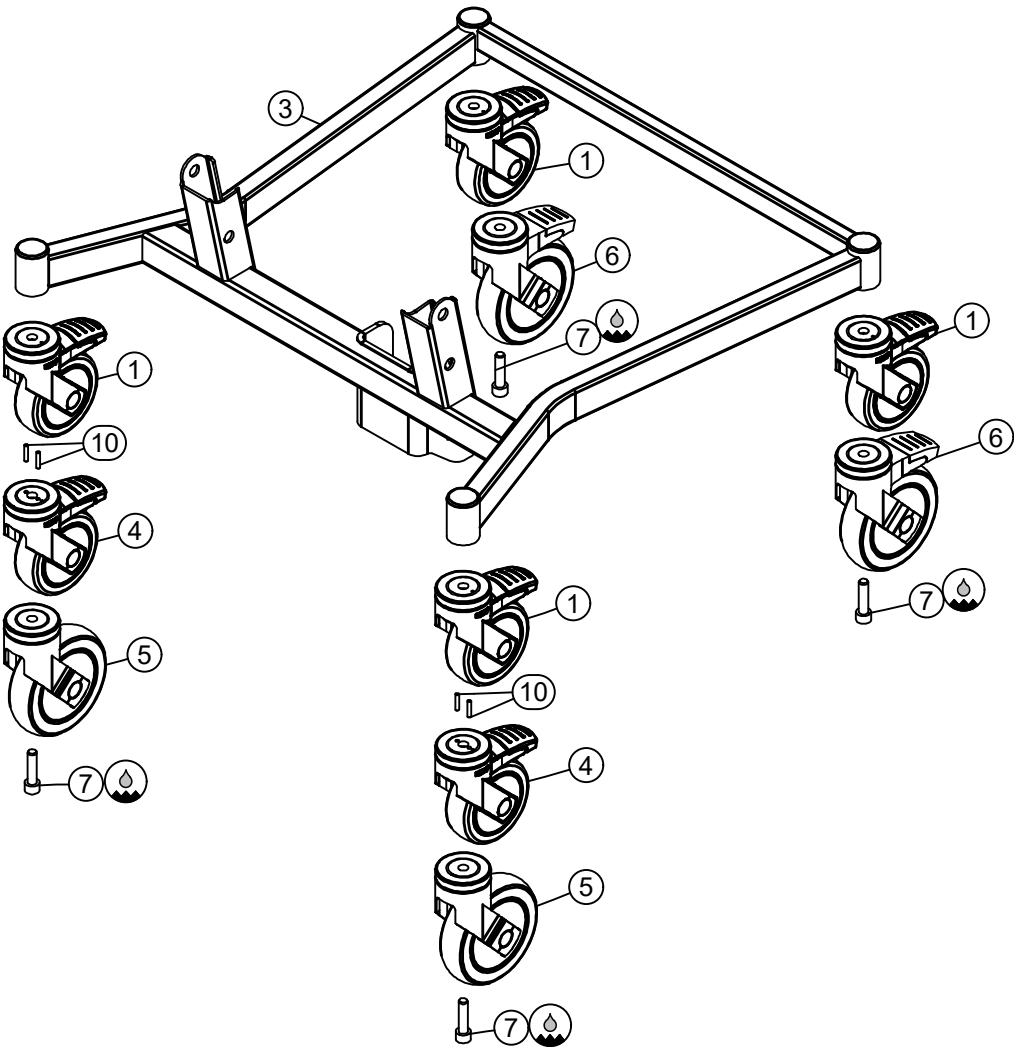


IV rod holder position

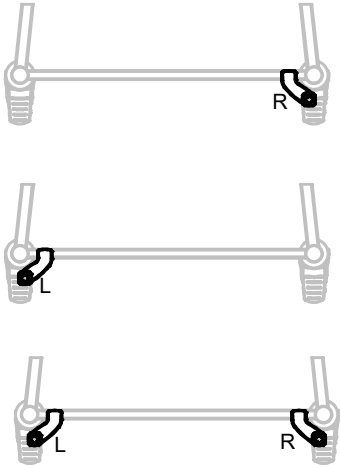


|    | ID       | Description                              | Q |
|----|----------|------------------------------------------|---|
| 1  | A-013148 | castor Levina-100-B-M10 A-2;B-2          | 4 |
| 2  | A-021494 | IV-rod holder Fero D+1;E+1               | 0 |
| 3  | A-023097 | carrier Avento HL without CB             | 1 |
| 4  | A-023685 | castor Levina-100-B-M10-DC A+2           | 0 |
| 5  | A-026153 | castor Levina-125-P30-M10 B+2            | 0 |
| 6  | A-026154 | castor Levina-125-B-P30-M10 B+2          | 0 |
| 7  | B-000339 | bolt Din912-M10x40x40-8.8 GV white       | 4 |
| 8  | P-020166 | fixation removable head part C+2         | 0 |
| 9  | S-023524 | screw Din7500-ME M5x10-St GV C+2;D+2;E+2 | 0 |
| 10 | ST021344 | dowel pin Din7-4x20-A1 A+4               | 0 |
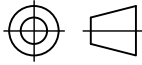
|                                                                                                                                                                                 |  |                                                                                                |                                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                       |
| Description:                                                                                                                                                                    |  | Designer:                                                                                      |  |
| undercarriage Avento HL - individual brake 202005                                                                                                                               |  | Ionut I.                                                                                       |                                                                                       |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 22/05/2020                                                                               | Scale: 1 : 8                                                                          |
|                                                                                                                                                                                 |  | Drawing nr.: T-039377                                                                          | Revision nr.: a                                                                       |

|    | ID       | Description                                     | Q  |
|----|----------|-------------------------------------------------|----|
| 1  | A-022112 | HL arm Avento HL                                | 1  |
| 2  | A-022116 | stabilisator arm Avento HL                      | 1  |
| 3  | A-022122 | main support Avento HL                          | 1  |
| 4  | A-022416 | hydraulic pump 10000N-IL202-SL80                | 1  |
| 5  | AD000435 | buffer                                          | 2  |
| 6  | AD003053 | grey cap M12                                    | 10 |
| 7  | B-003235 | bolt Din912-M12x30x20-8.8 GV white              | 6  |
| 8  | B-012047 | bolt Iso7380-M8x30x30-10.9 GV white             | 2  |
| 9  | B-018125 | bolt Iso7380-M6x10x10-10.9 GV white             | 2  |
| 10 | B-027583 | bolt Din912-M12x20x15-8.8 GV white              | 4  |
| 11 | DC027515 | axle 10x42                                      | 1  |
| 12 | GL017646 | bearing 101216-12                               | 2  |
| 13 | GL025719 | bearing 1214-04                                 | 6  |
| 14 | GI000620 | bearing 1214-06                                 | 4  |
| 15 | M-000189 | check nut Din985-M8-8-GV                        | 2  |
| 16 | P-024987 | hydraulic pump fixation ring                    | 2  |
| 17 | P-028260 | high-low pedal                                  | 2  |
| 18 | P-028763 | tube pedal L Avento HL without CB               | 1  |
| 19 | P-028765 | tube pedal R Avento HL without CB               | 1  |
| 20 | S-014995 | screw type TT12-M5x10-GV white                  | 2  |
| 21 | ST010662 | axle locking ring 14/7x1.3                      | 10 |
| 22 | T-039377 | undercarriage Avento HL individual brake 202005 | 1  |
| 23 | VL000469 | washer Din125A-M10-PA                           | 5  |
| 24 | VL002143 | washer Din6798A-M6                              | 2  |
| 25 | VL002364 | bearing flange 2036-015                         | 2  |
| 26 | VI000638 | bearing flange 1224-015                         | 8  |

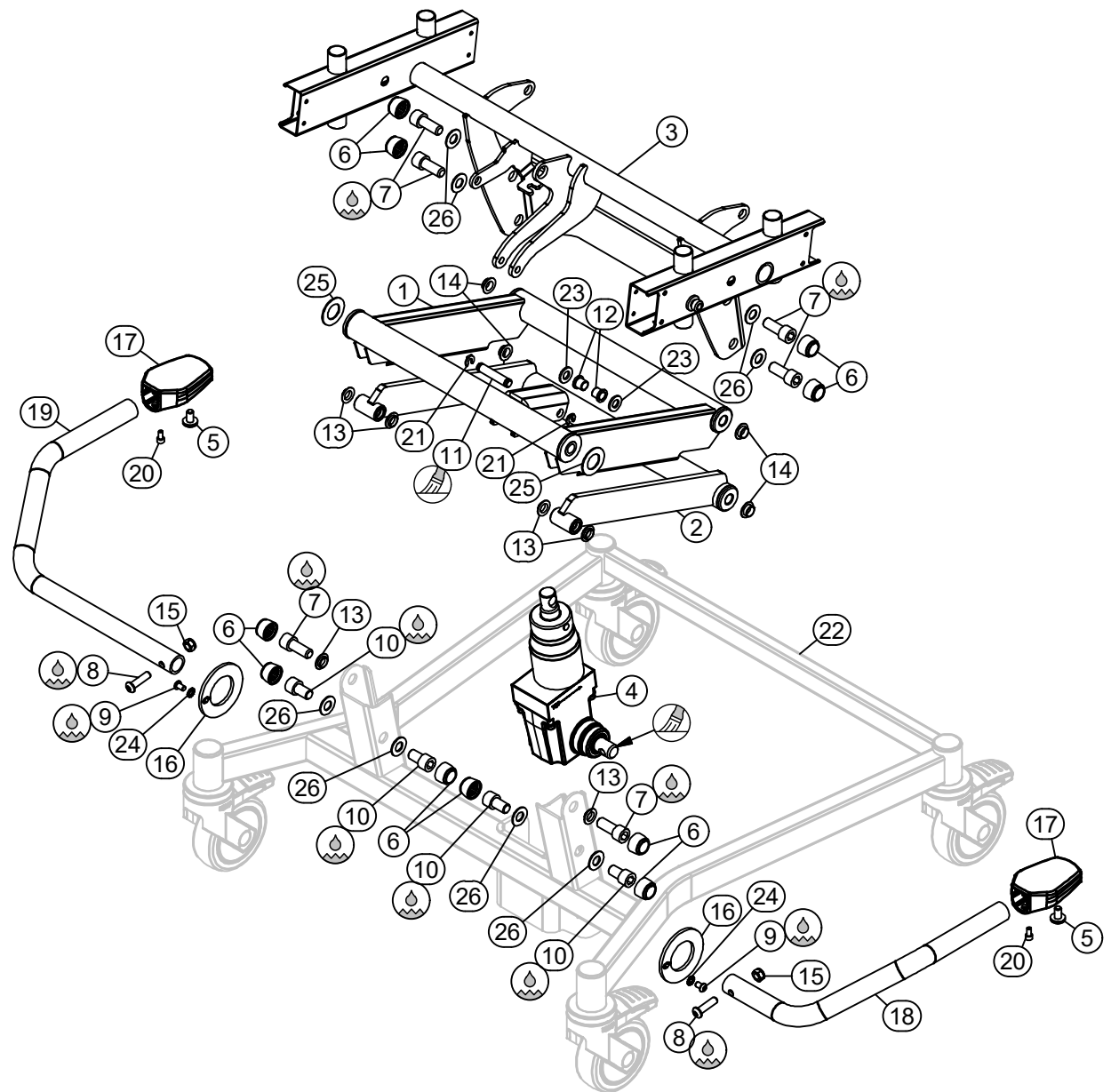


|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| Description:                                                                                                                                                                    |  | Designer:                                                                                      |   |
| undercarriage + HL system Avento HL individual brake 202005                                                                                                                     |  | Ionut I.                                                                                       |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 25/05/2020                                                                               | Scale: 1 : 8                                                                                                                                                                |
|                                                                                                                                                                                 |  | Drawing nr.: T-039395                                                                          | Revision nr.: a                                                                                                                                                             |

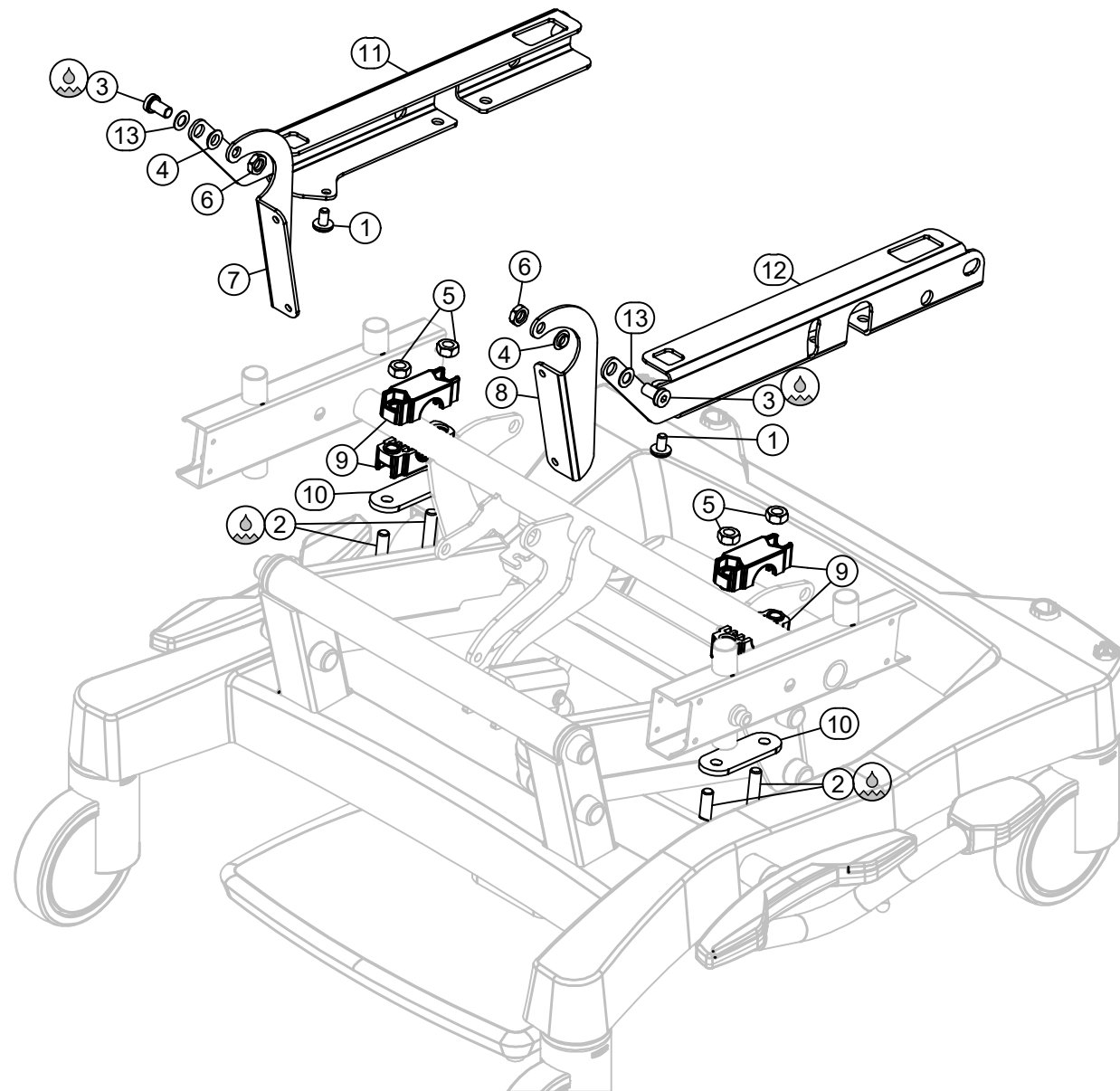

|    | ID       | Description                            | Q |
|----|----------|----------------------------------------|---|
| 1  | AD000435 | buffer                                 | 2 |
| 2  | B-000822 | bolt Iso7380-M10x50-10.9 GV white - LC | 4 |
| 3  | B-003317 | bolt-M10x18x12-GV white                | 2 |
| 4  | GL000618 | bearing 1012-04                        | 2 |
| 5  | M-000342 | nut Din934-M10-8.8                     | 4 |
| 6  | M-006850 | nut din439-M10-04 GV                   | 2 |
| 7  | P-005858 | support for leg support R              | 1 |
| 8  | P-005859 | support for leg support L              | 1 |
| 9  | P-005967 | plain bearing 28 - top                 | 4 |
| 10 | P-021862 | fortification plate                    | 2 |
| 11 | P-036187 | seatprofile R Avento HL 202005         | 1 |
| 12 | P-036188 | seatprofile L Avento HL 202005         | 1 |
| 13 | VL001873 | bearing flange 1018-010                | 2 |


|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| Description:                                                                                                                                                                    |  | Designer:                                                                                      |   |
| Avento HL - seat frame standard 202005                                                                                                                                          |  | Ionut I.                                                                                       |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 25/05/2020                                                                               | Scale: 1 : 6                                                                                                                                                                |
|                                                                                                                                                                                 |  | Drawing nr.: T-039396                                                                          | Revision nr.: a                                                                                                                                                             |



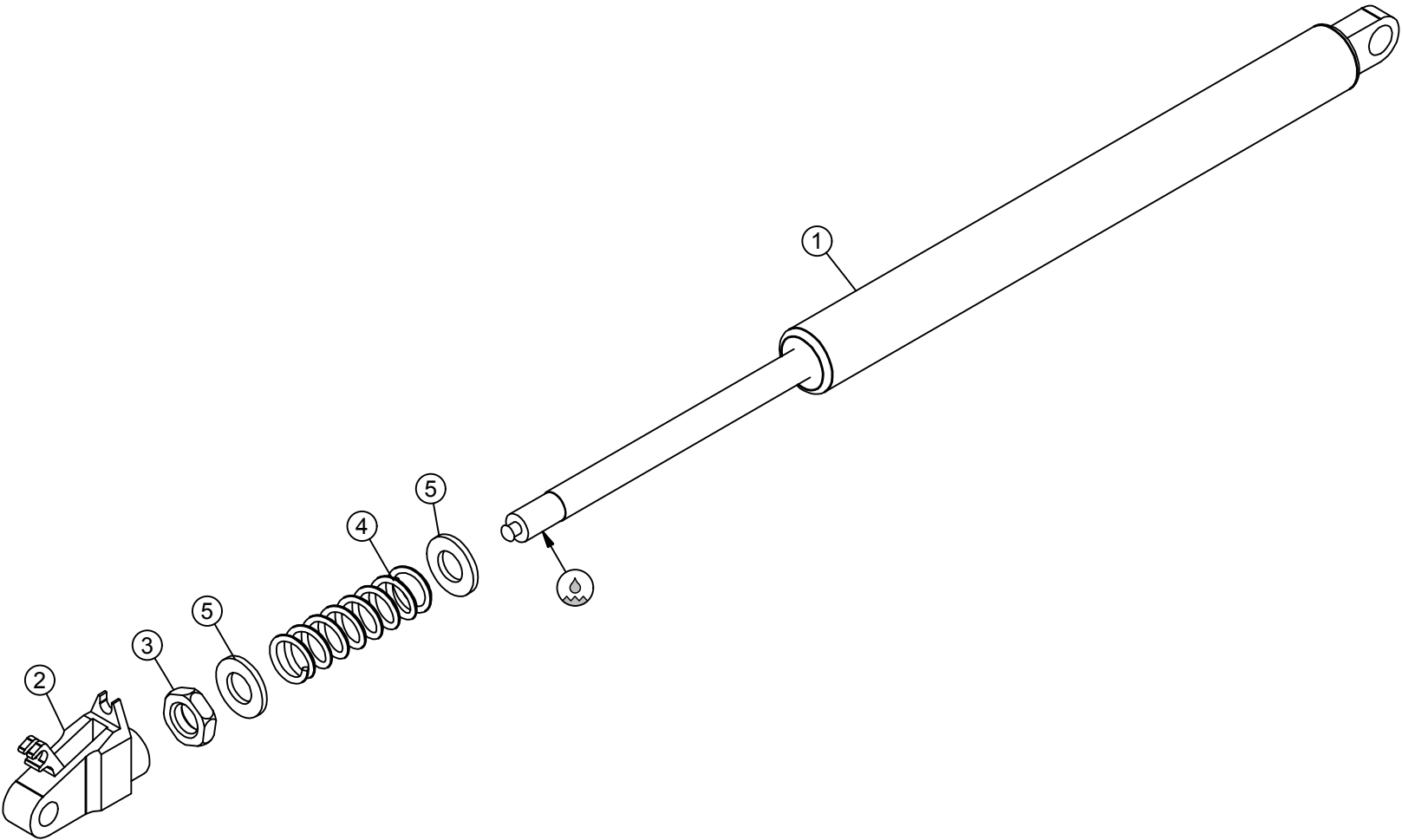

|   | ID       | Description                     | Q |
|---|----------|---------------------------------|---|
| 1 | A-022727 | gasspring 22B-SL85-IL283.5-900N | 1 |
| 2 | A-024087 | relase head 4401                | 1 |
| 3 | M-001368 | nut Din439-M10x1-GV             | 1 |
| 4 | P-027962 | spring H55-D16-Dr2              | 1 |
| 5 | VL000469 | washer Din125A-M10-PA           | 2 |

|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| Description:                                                                                                                                                                    |  | Designer:                                                                                      |   |
| gas spring Avento HL stand-up aid                                                                                                                                               |  | Sebastian Ola                                                                                  |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 16.09.2016                                                                               | Scale: 1 : 2                                                                                                                                                                |
|                                                                                                                                                                                 |  | Drawing nr.: T-030222                                                                          | Revision nr.: c                                                                                                                                                             |

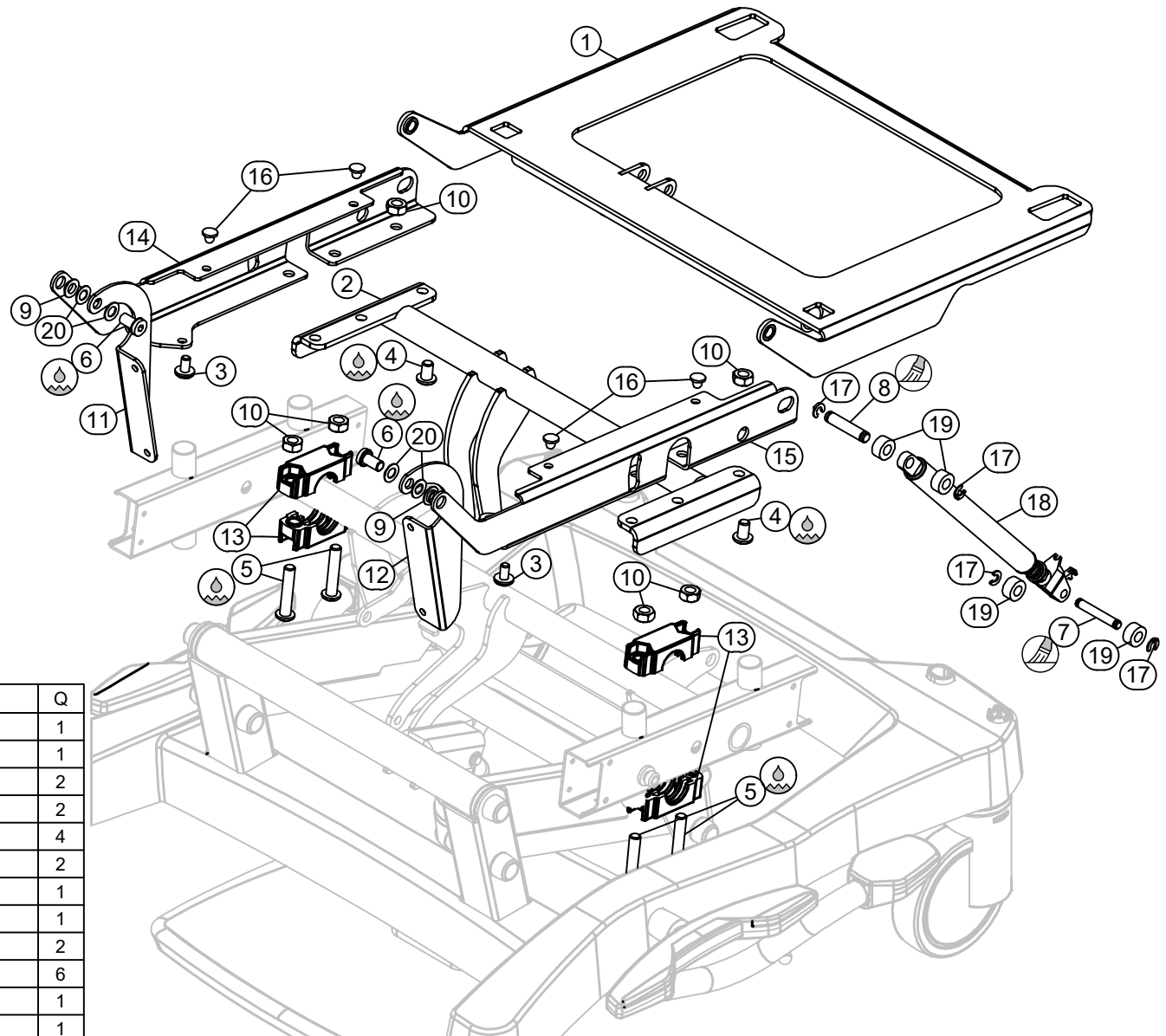

|    | ID       | Description                                  | Q |
|----|----------|----------------------------------------------|---|
| 1  | A-026447 | tilt frame stand-up aid Avento HL 201910     | 1 |
| 2  | A-026456 | support frame stand-up aid 201910            | 1 |
| 3  | AD000435 | buffer                                       | 2 |
| 4  | B-000816 | bolt Iso7380-M10x16x16-10.9 GV white         | 2 |
| 5  | B-000822 | bolt Iso7380-M10x50-10.9 GV white - LC       | 4 |
| 6  | B-003317 | bolt-M10x18x12-GV white                      | 2 |
| 7  | DC022156 | axle type 1-8x45                             | 1 |
| 8  | DC026581 | axle type 1-10x53                            | 1 |
| 9  | GL000618 | bearing 1012-04                              | 2 |
| 10 | M-000342 | nut Din934-M10-8.8                           | 6 |
| 11 | P-005858 | support for leg support R                    | 1 |
| 12 | P-005859 | support for leg support L                    | 1 |
| 13 | P-005967 | plain bearing 28 - top                       | 4 |
| 14 | P-034741 | seat profile R Avento HL stand-up aid 201910 | 1 |
| 15 | P-034742 | seat profile L Avento HL stand-up aid 201910 | 1 |
| 16 | ST000141 | buffer 14x2/8x7                              | 4 |
| 17 | ST010662 | axle locking ring 14/7x1.3                   | 4 |
| 18 | T-030222 | gas spring Avento HL stand-up aid            | 1 |
| 19 | VL001176 | synthetic washer 20x10x10                    | 4 |
| 20 | VL001873 | bearing flange 1018-010                      | 4 |



## Description:

Avento HL - seat frame with stand-up aid 202005


All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95

## Designer:

Ionut I.

Date: 27/05/2020

Drawing nr.: T-039426

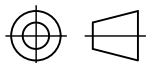


Scale: 1 :6


Revision nr.: a

| Options |                 |
|---------|-----------------|
| ID.     | Description     |
| A       | Stand-up aid    |
| B       | Reclining lock  |
| C       | Accessory hooks |
| D       | DIN rail        |


|    | ID       | Description                              | Q             |
|----|----------|------------------------------------------|---------------|
| 1  | A-022512 | plastic spring button- LA Avento         | 2             |
| 2  | A-026821 | DIN rail Avento                          | D+2 0         |
| 3  | B-000651 | bolt Din912-M6x16x16-8.8 GV white        | D+4 0         |
| 4  | DC027316 | armrest guiding Avento HL                | 4             |
| 5  | P-020160 | accessory hook vertical fixation         | C+4 0         |
| 6  | P-025201 | side cover Avento R                      | A-1;C-1;D-1 1 |
| 7  | P-025318 | side cover Avento L                      | A-1;C-1;D-1 1 |
| 8  | P-026003 | inside cover plate Hospital              | A-1;B-1 2     |
| 9  | P-028787 | side cover Avento stand-up aid R         | A+1 0         |
| 10 | P-028788 | side cover Avento stand-up aid L         | A+1 0         |
| 11 | P-030141 | inside cover plate Avento - lock         | A+1;B+1 0     |
| 12 | P-032247 | side cover Avento L options              | C+1;D+1 0     |
| 13 | P-032252 | plate accessory hooks Avento             | C+2 0         |
| 14 | P-032253 | side cover Avento R options              | C+1;D+1 0     |
| 15 | P-035452 | fixation plate B DIN rail Avento         | D+2 0         |
| 16 | P-035453 | fixation plate A DIN rail Avento         | D+2 0         |
| 17 | P-035665 | side cover Avento stand-up aid L options | A+C+1;A+D+1 0 |
| 18 | P-035666 | side cover Avento stand-up aid R options | A+C+1;A+D+1 0 |
| 19 | R-000461 | rivet Alu/St 4.8x10                      | D+8 0         |
| 20 | R-002201 | rivet Alu/St 4.8x14                      | C+4 0         |
| 21 | S-018531 | screw PLASTITE 7985-4x13-GV              | 8             |
| 22 | S-028464 | screw Din7500 CE-M5x10                   | 8             |

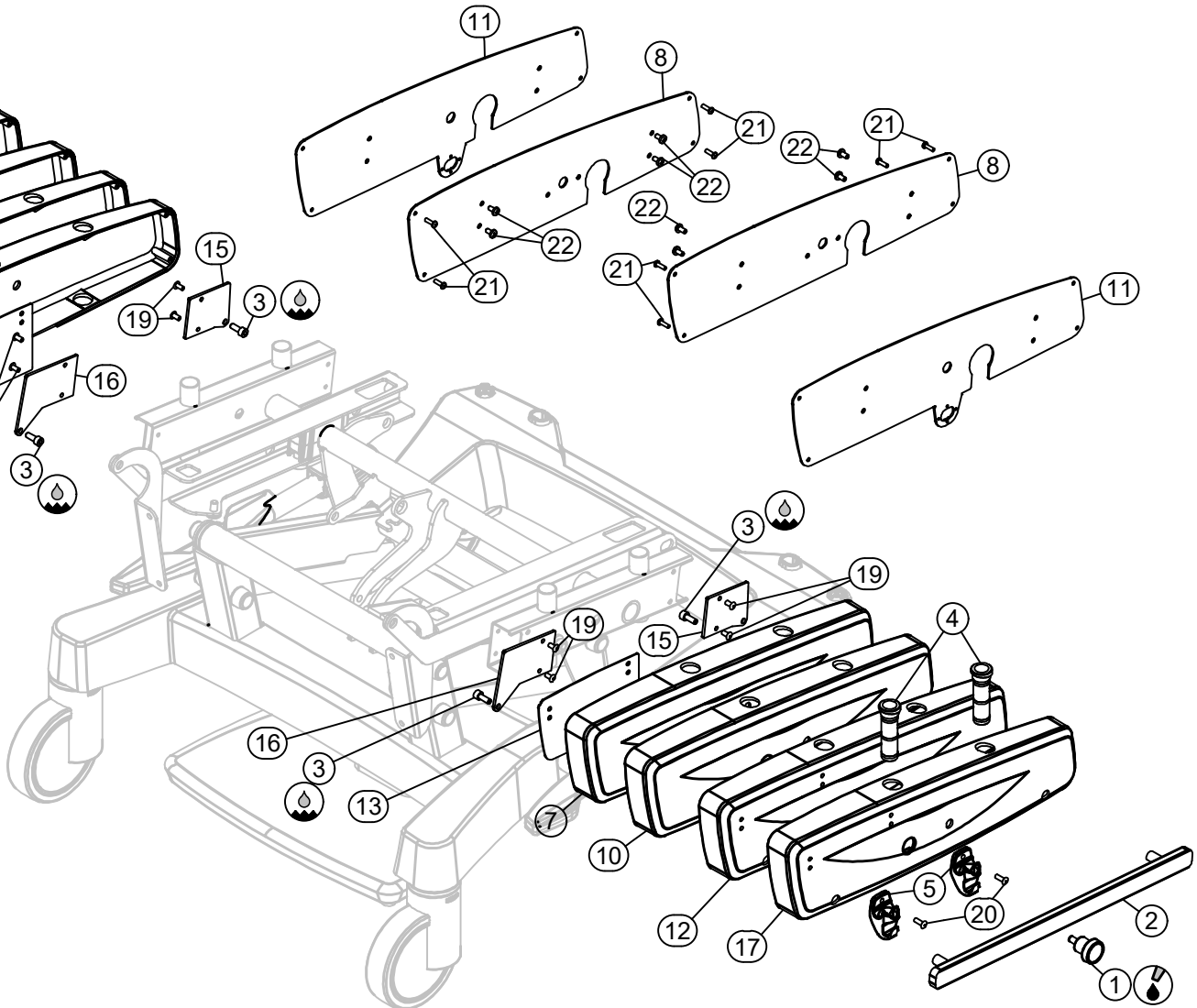


|                                                                                                                                                                                 |  |                                                                                                       |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|-------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  <b>Haelvoet</b> |                                                                                                                                                                             |
| <b>Description:</b><br>sides Avento Hospital 202005                                                                                                                             |  | <b>Designer:</b><br>Ionut I.                                                                          |   |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | <b>Date:</b> 29/05/2020<br><b>Drawing nr.:</b> T-039446                                               | <b>Scale:</b> 1 : 8<br><b>Revision nr.:</b> b                                                                                                                               |

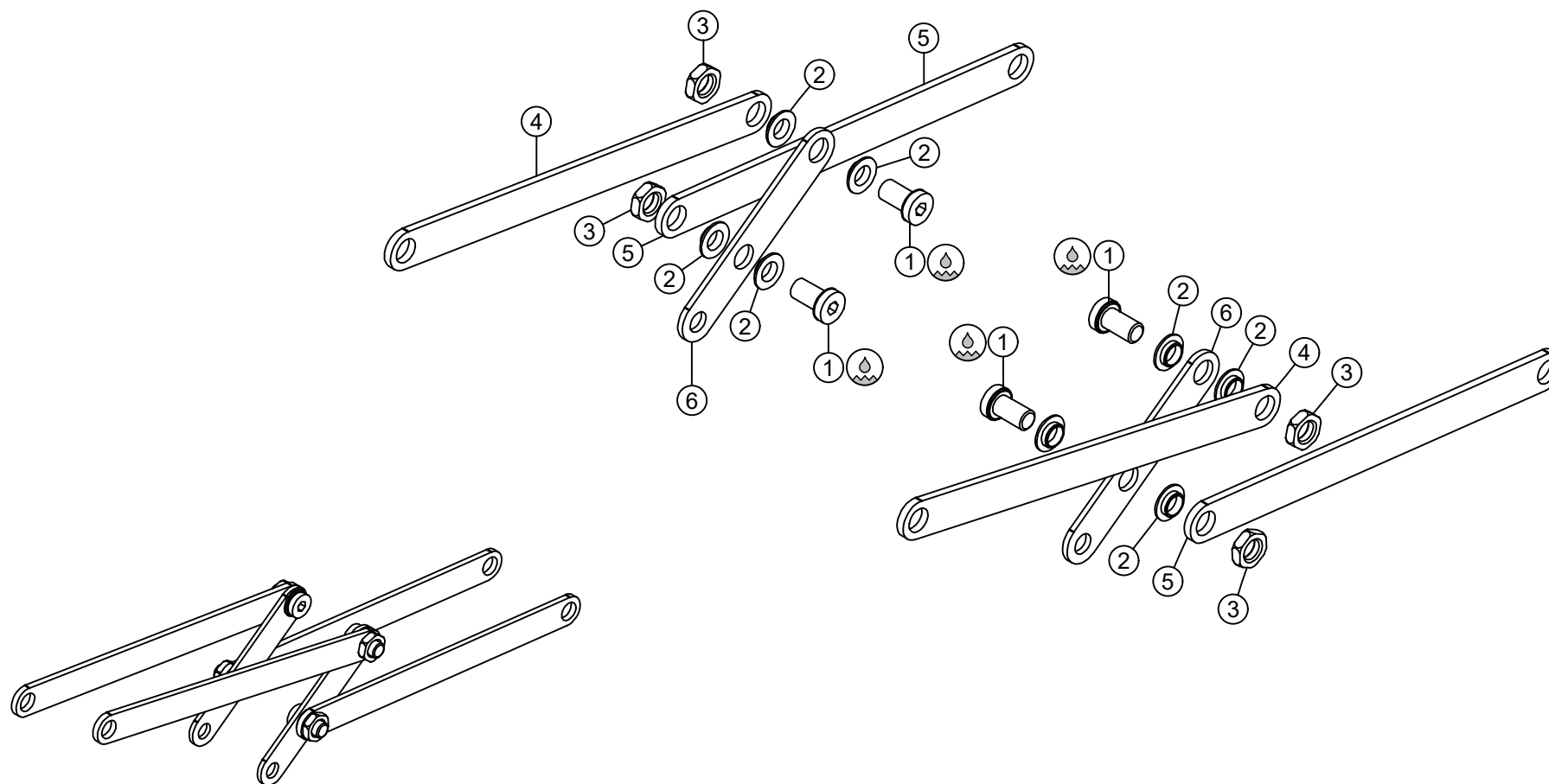


|   | ID       | Description             | Q |
|---|----------|-------------------------|---|
| 1 | B-003317 | bolt-M10x18x12-GV white | 4 |
| 2 | GL000618 | bearing 1012-04         | 8 |
| 3 | M-006850 | nut din439-M10-04 GV-LC | 4 |
| 4 | P-005057 | push fork leg support   | 2 |
| 5 | P-005423 | push fork backrest      | 2 |
| 6 | P-005824 | tilting fork            | 2 |
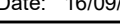
|                                                                                                                |                                                                                       |                                                                                       |
|----------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|                                                                                                                | Scale: 1 : 3                                                                          |  |
|                                                                                                                | Designer: Sebastian Ola                                                               |                                                                                       |
|                                                                                                                | Date: 16/09/2016                                                                      |                                                                                       |
| Description:<br>recliner mechanism forks Ergo-line                                                             |  |                                                                                       |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV |                                                                                       |                                                                                       |

|                                                 |  |  |
|-------------------------------------------------|--|--|
| L. Bekaertstraat 8 - 8770 Ingelmunster, Belgium |  |  |
| Tel.: +32(0)51 48 66 95 Fax.:+32(0)51 48 73 19  |  |  |

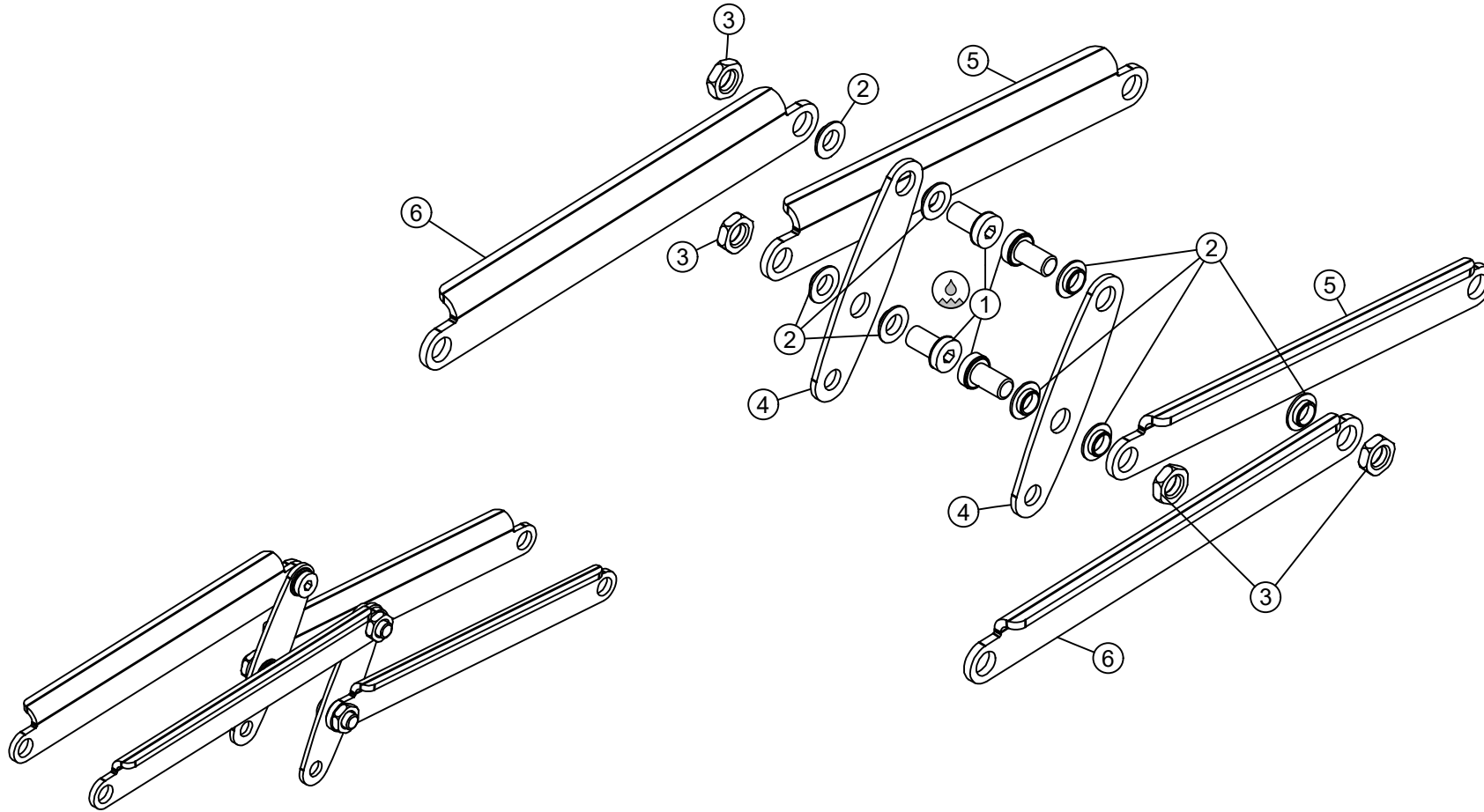


|   | ID       | Description                         | Q |
|---|----------|-------------------------------------|---|
| 1 | B-003317 | bolt-M10x18x12-GV white             | 4 |
| 2 | GL000618 | bearing 1012-04                     | 8 |
| 3 | M-006850 | nut din439-M10-04 GV-LC             | 4 |
| 4 | P-021980 | tilting fork 200 kg                 | 2 |
| 5 | P-035313 | push fork backrest 200 kg 202001    | 2 |
| 6 | P-035314 | push fork leg support 200 kg 202001 | 2 |
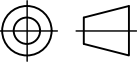
|                                                                                                                                                                                 |  |                                                                                                |                                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                       |
| <b>Description:</b><br>recliner mechanism forks 200 kg                                                                                                                          |  | Designer:<br>Bogdan Onica                                                                      |  |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 11/10/2018<br>Drawing nr.: T-034633                                                      | Scale: 1 : 3<br>Revision nr.: b                                                       |


T-039463

a


Reclining lock on right side

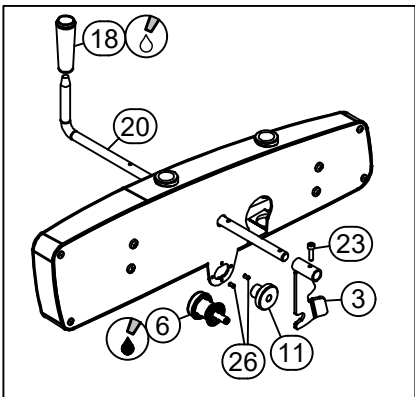

Stand-up aid lock on left side

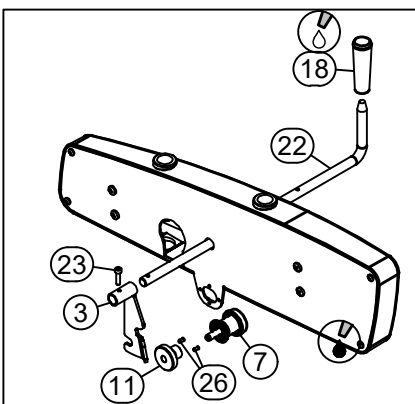


Manual adjustable legrest

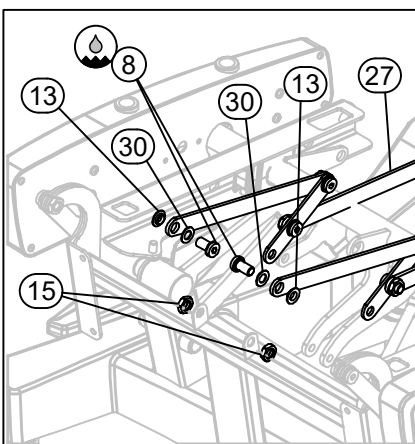

| Options |                               |
|---------|-------------------------------|
| ID.     | Description                   |
| A       | Stand-up aid                  |
| B       | Reclining lock                |
| C       | Manual adjustable legrest (*) |

(\*) Possible only with Avento fixed height

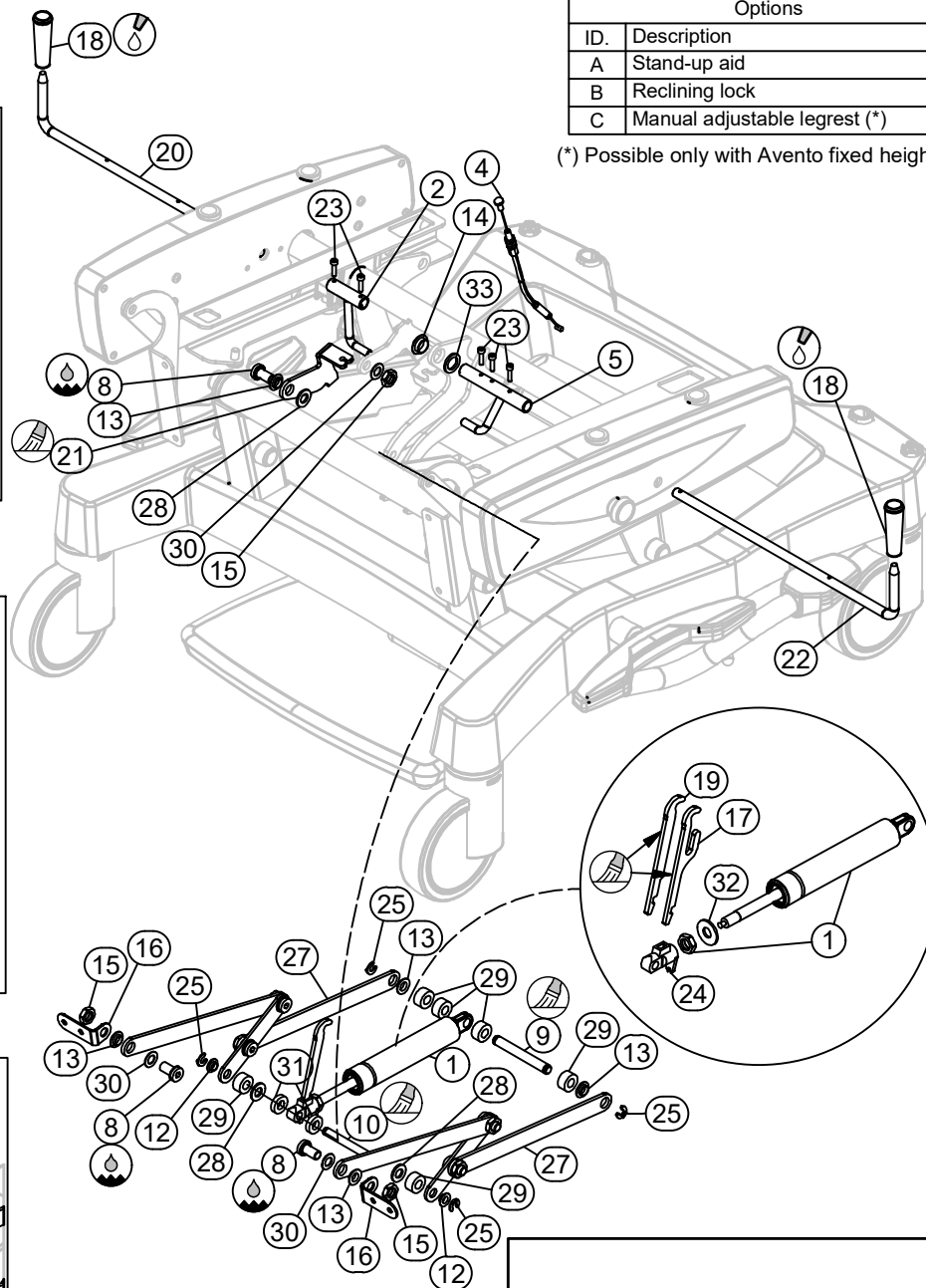

|    | ID       | Description                               | Q         |
|----|----------|-------------------------------------------|-----------|
| 1  | A-003302 | gas spring 28BD-SL50-IL187-700N           | 1         |
| 2  | A-022168 | activation cam stand-up aid               | A+1 0     |
| 3  | A-022670 | reclining locking cam                     | A+1;B+1 0 |
| 4  | A-022751 | activation cable Avento HL stand-up aid   | A+1 0     |
| 5  | A-023204 | camshaft                                  | 1         |
| 6  | A-027405 | spring button lock mechanism red RAL3026  | B+1 0     |
| 7  | A-027406 | spring button lock mechanism blue RAL5017 | A+1 0     |
| 8  | B-003317 | bolt-M10x18x12-GV white                   | A+1 2     |
| 9  | DC002587 | axle type 1-10x72                         | 1         |
| 10 | DC006037 | axle type 1-8x64                          | 1         |
| 11 | DC029334 | fixation bush locking spring button       | A+1;B+1 0 |
| 12 | GL000611 | bearing 0810-04                           | 2         |
| 13 | GL000618 | bearing 1012-04                           | A+1 4     |
| 14 | GL005737 | bearing 1416-06                           | 1         |
| 15 | M-006850 | nut din439-M10-04 GV                      | A+1 2     |
| 16 | P-005430 | pushing hinge leg support                 | C-2 2     |
| 17 | P-005840 | control cam Fero Ergo-line                | A-1 1     |
| 18 | P-011411 | handle Fero                               | 2         |
| 19 | P-025055 | control cam Avento HL                     | A+1 0     |
| 20 | P-025072 | control handle Fero with stand-up aid     | 1         |
| 21 | P-027578 | cable cam stand-up aid Avento HL          | A+1 0     |
| 22 | P-027608 | control handle Fero                       | 1         |
| 23 | S-015930 | screw Din7500(TT12)-M4x16-GV              | A+3;B+1 3 |
| 24 | ST005073 | release head M10 dia8                     | 1         |
| 25 | ST010662 | axle locking ring 14/7x1.3                | 4         |
| 26 | ST029828 | cylindrical pin Din6325 3m5x6 St          | A+2;B+2 0 |
| 27 | T-030224 | recliner mechanism forks Fero Ergo-line   | 1         |
| 28 | VL000469 | washer Din125A-M10-PA                     | A+1 2     |
| 29 | VL001176 | synthetic washer 20x10x10                 | 6         |
| 30 | VL001873 | bearing flange 1018-010                   | A+1 2     |
| 31 | VL004120 | synthetic washer 20x10x5                  | 2         |
| 32 | VL014993 | synthetic washer 25x10x2                  | 1         |
| 33 | VL029858 | washer Din125A-M14.5-PA                   | 1         |

Description:

Avento Hospital - recliner mechanism 202006


All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95

Designer:

Ionut I.

Date: 03/06/2020

Drawing nr.: T-039463

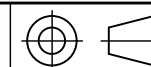


Scale: 1 : 8

Revision nr.: a



|          |                               |   |
|----------|-------------------------------|---|
| T-039465 |                               | a |
| Options  |                               |   |
| ID.      | Description                   |   |
| A        | Reclining lock                |   |
| B        | Manual adjustable legrest (*) |   |

(\*) Possible only with Avento fixed height

Reclining lock on right side

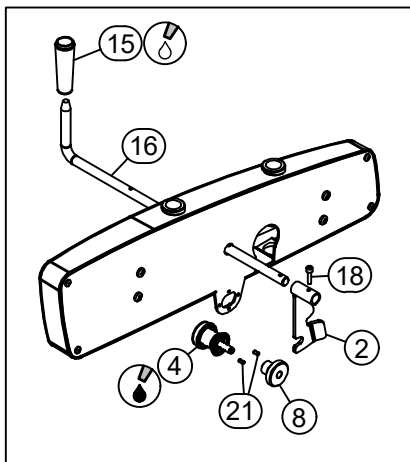

Manual adjustable legrest

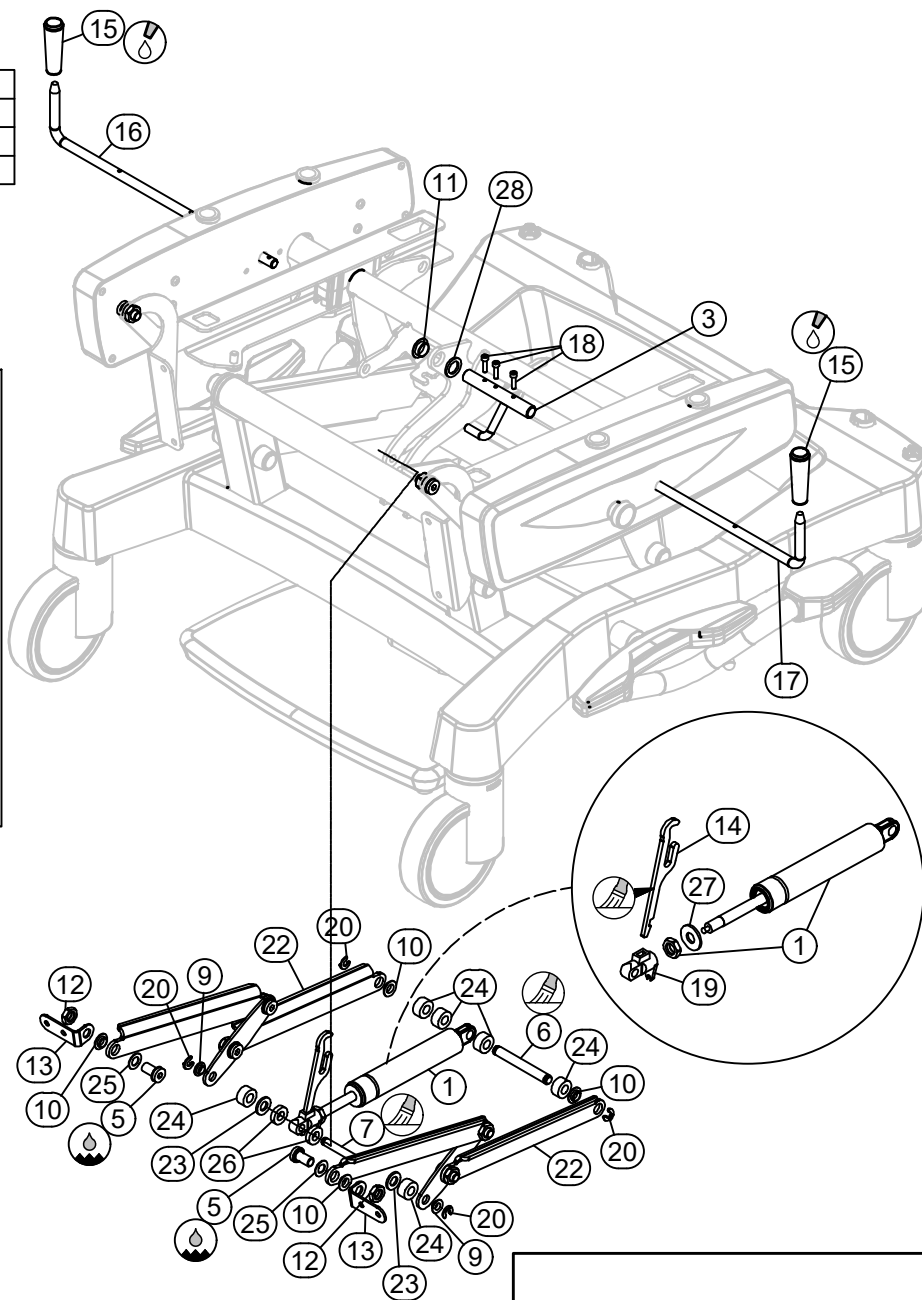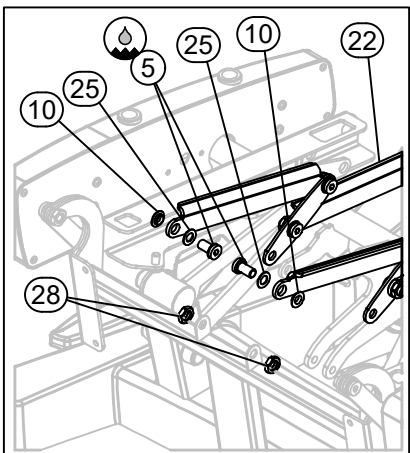


|    | ID       | Description                              | Q     |
|----|----------|------------------------------------------|-------|
| 1  | A-003302 | gas spring 28BD-SL50-IL187-700N          | 1     |
| 2  | A-022670 | reclining locking cam                    | A+1 0 |
| 3  | A-023204 | camshaft                                 | 1     |
| 4  | A-027405 | spring button lock mechanism red RAL3026 | A+1 0 |
| 5  | B-003317 | bolt-M10x18x12-GV white                  | 2     |
| 6  | DC002587 | axle type 1-10x72                        | 1     |
| 7  | DC006037 | axle type 1-8x64                         | 1     |
| 8  | DC029334 | fixation bush locking spring button      | A+1 0 |
| 9  | GL000611 | bearing 0810-04                          | 2     |
| 10 | GL000618 | bearing 1012-04                          | 4     |
| 11 | GL005737 | bearing 1416-06                          | 1     |
| 12 | M-006850 | nut din439-M10-04 GV                     | 2     |
| 13 | P-005430 | pushing hinge leg support                | B-2 2 |
| 14 | P-005840 | control cam Fero Ergo-line               | 1     |
| 15 | P-011411 | handle Fero                              | 2     |
| 16 | P-025072 | control handle Fero with stand-up aid    | 1     |
| 17 | P-027608 | control handle Fero                      | 1     |
| 18 | S-015930 | screw Din7500(TT12)-M4x16-GV             | A+1 3 |
| 19 | ST005073 | release head M10 dia8                    | 1     |
| 20 | ST010662 | axle locking ring 14/7x1.3               | 4     |
| 21 | ST029828 | cylindrical pin Din6325 3m5x6 St         | A+2 0 |
| 22 | T-034633 | recliner mechanism forks 200 kg          | 1     |
| 23 | VL000469 | washer Din125A-M10-PA                    | 2     |
| 24 | VL001176 | synthetic washer 20x10x10                | 6     |
| 25 | VL001873 | bearing flange 1018-010                  | 2     |
| 26 | VL004120 | synthetic washer 20x10x5                 | 2     |
| 27 | VL014993 | synthetic washer 25x10x2                 | 1     |
| 28 | VL029858 | washer Din125A-M14.5-PA                  | 1     |

|                                                                                                                                                                                 |  |                                                                                                |                                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                       |
| Description:                                                                                                                                                                    |  | Designer:                                                                                      |  |
| Option: Avento Hospital - recliner mechanism 200 kg 202006                                                                                                                      |  | Ionut I.                                                                                       |  |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 03/06/2020                                                                               | Scale: 1 : 8                                                                          |
|                                                                                                                                                                                 |  | Drawing nr.: T-039465                                                                          | Revision nr.: a                                                                       |

| Options |                                         |
|---------|-----------------------------------------|
| ID.     | Description                             |
| A       | backrest with removable head rest       |
| B       | shoulder rests Fero                     |
| C       | removable head rest with shoulder rests |


|   | ID       | Description                                                                   | Q |
|---|----------|-------------------------------------------------------------------------------|---|
| 1 | A-020534 | backrest Fero <span>A/B/C-1</span>                                            | 1 |
| 2 | A-020547 | backrest Fero with removable head section <span>A+1</span>                    | 0 |
| 3 | A-020581 | backrest Fero with shoulder rests <span>B+1</span>                            | 0 |
| 4 | A-020782 | neck rest Fero                                                                | 1 |
| 5 | A-021251 | backrest Fero with removable head section and shoulder rests <span>C+1</span> | 0 |
| 6 | A-023569 | fixation back                                                                 | 1 |
| 7 | B-000787 | bolt Din912-M10x20x20-8.8 GV white                                            | 4 |
| 8 | VL000338 | washer Din125A-M10-GV                                                         | 4 |

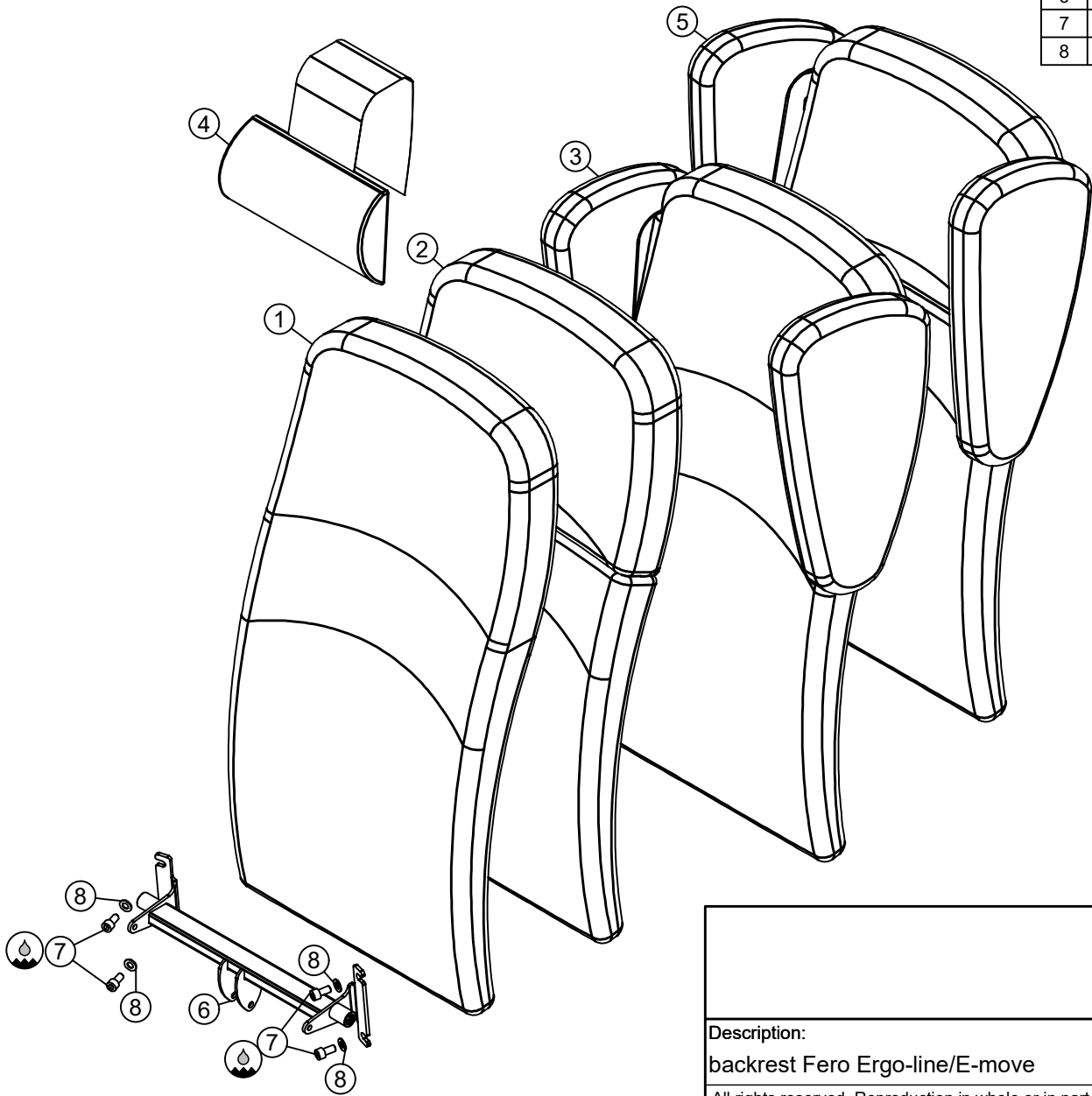

|                                                                                                                                                                                 |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  Haelvoet                                                                                  |                                                                                                                                                                             |
| Description:<br>backrest Fero Ergo-line/E-move                                                                                                                                  |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |                                                                                                                                                                             |
| Designer:<br>Bogdan Onica                                                                                                                                                       |   |
| Date: 09/04/2020                                                                                                                                                                | Scale: 1 : 10                                                                                                                                                               |
| Drawing nr.: T-039043                                                                                                                                                           | Revision nr.: a                                                                                                                                                             |




|          |                           |   |
|----------|---------------------------|---|
| T-039466 |                           | a |
| Options  |                           |   |
| ID.      | Description               |   |
| A        | fixation brace            |   |
| B        | support dinner tray Elice |   |

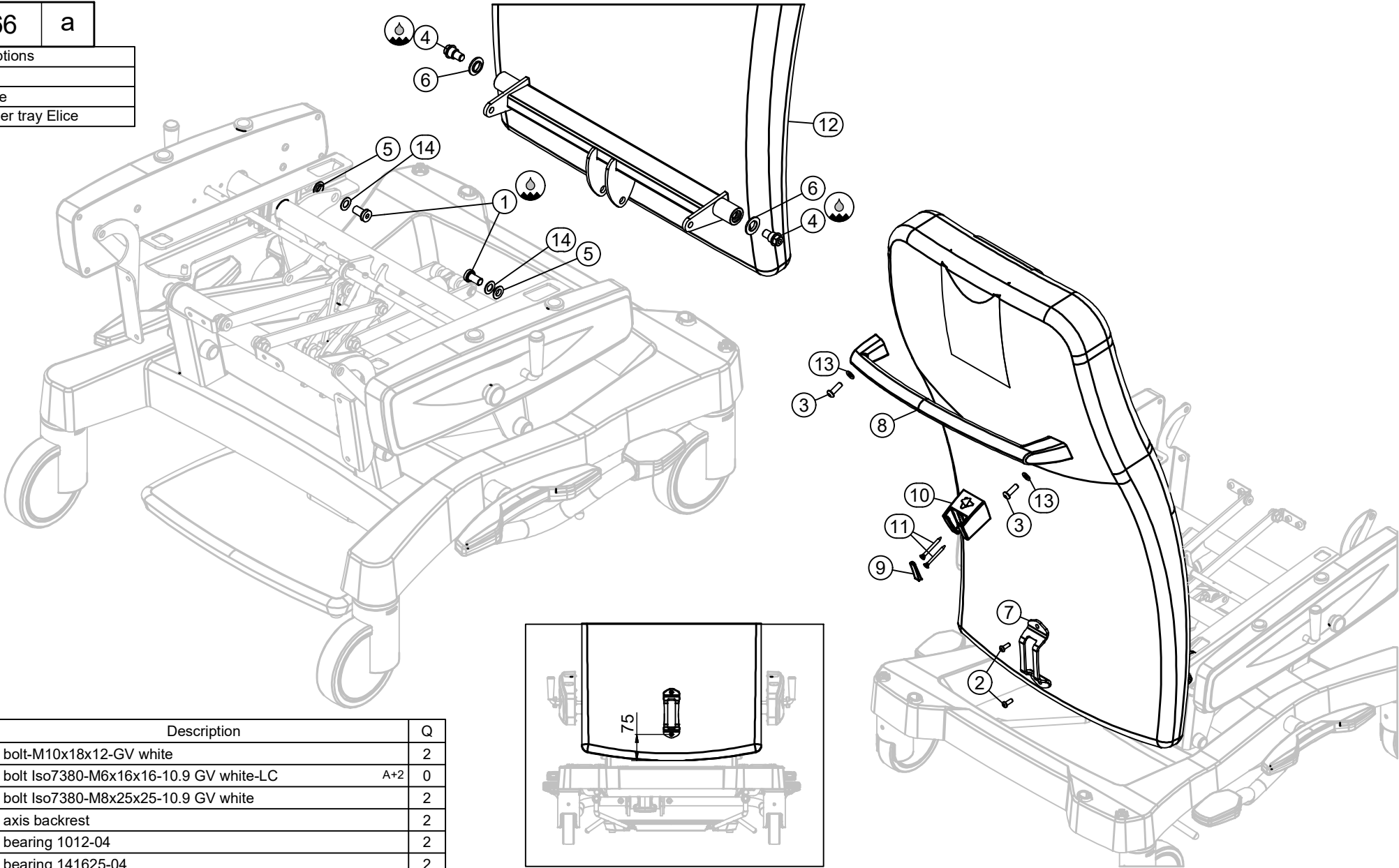

|    | ID       | Description                                       | Q     |
|----|----------|---------------------------------------------------|-------|
| 1  | B-003317 | bolt-M10x18x12-GV white                           | 2     |
| 2  | B-005873 | bolt Iso7380-M6x16x16-10.9 GV white-LC            | A+2 0 |
| 3  | B-016275 | bolt Iso7380-M8x25x25-10.9 GV white               | 2     |
| 4  | DC029455 | axis backrest                                     | 2     |
| 5  | GL000618 | bearing 1012-04                                   | 2     |
| 6  | GL029787 | bearing 141625-04                                 | 2     |
| 7  | P-016223 | fixation brace Fero                               | A+1 0 |
| 8  | P-022121 | push bar Fero                                     | 1     |
| 9  | P-027408 | cap support dinner tray Elice                     | B+1 0 |
| 10 | P-027409 | support dinner tray Elice                         | B+1 0 |
| 11 | S-028666 | screw with sunken head and drilled head-4,5x50-GV | B+2 0 |
| 12 | T-039043 | backrest Fero Ergo-line/Emove                     | 1     |
| 13 | VL000337 | washer Din125A-M8-GV                              | 2     |
| 14 | VL001873 | bearing flange 1018-010                           | 2     |

|                                                                                                                                                                                 |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  <b>Haelvoet</b>                                                                           |                                                                                                                                                                             |
| <b>Description:</b><br>backrest Avento 202006                                                                                                                                   |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |                                                                                                                                                                             |
| <b>Designer:</b><br>Ionut I.                                                                                                                                                    |   |
| <b>Date:</b> 03/06/2020<br><b>Drawing nr.:</b> T-039466                                                                                                                         | <b>Scale:</b> 1 : 7<br><b>Revision nr.:</b> a                                                                                                                               |

| Options |                |
|---------|----------------|
| ID.     | Description    |
| A       | Reclining lock |
| B       | Stand-up aid   |


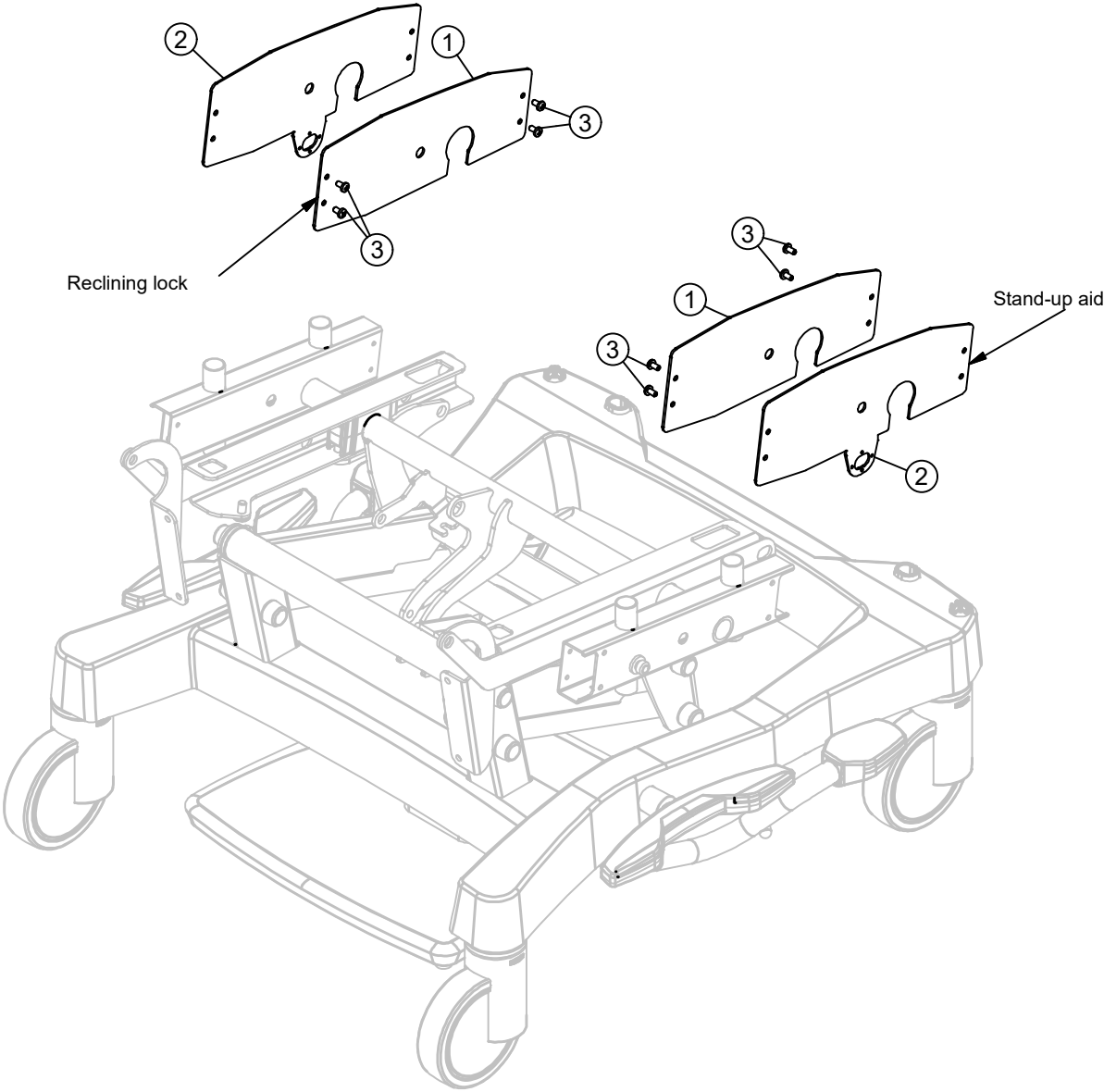


|   | ID       | Description                                | Q |
|---|----------|--------------------------------------------|---|
| 1 | P-029014 | inside cover Avento HL Care A-1;B-1        | 2 |
| 2 | P-029040 | inside cover Avento HL Care + lock A+1;B+1 | 0 |
| 3 | S-028464 | screw Din7500 CE-M5x10                     | 8 |

|                                                                                                                                                                                 |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  Haelvoet                                                                                  |                                                                                                                                                                             |
| <b>Description:</b><br>sides Avento Care 202006                                                                                                                                 |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |                                                                                                                                                                             |
| Designer:<br>Ionut I.                                                                                                                                                           |   |
| Date: 05/06/2020                                                                                                                                                                | Scale: 1 :7                                                                                                                                                                 |
| Drawing nr.: T-039481                                                                                                                                                           | Revision nr.: a                                                                                                                                                             |


| Options |                               |
|---------|-------------------------------|
| ID.     | Description                   |
| A       | Stand-up aid                  |
| B       | Manual adjustable legrest (*) |

(\*) Possible only with Avento fixed height

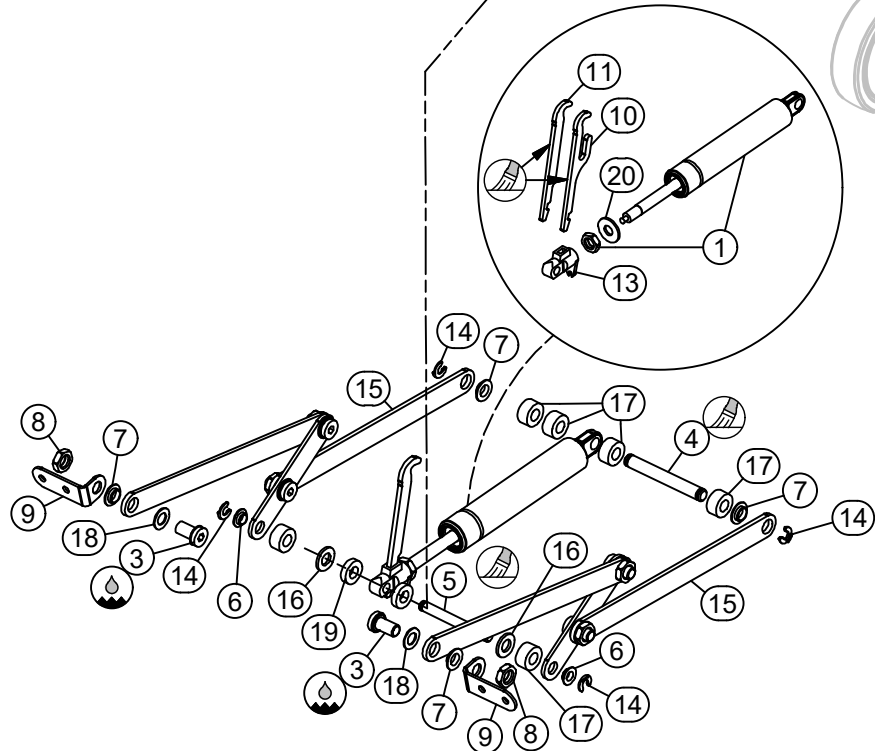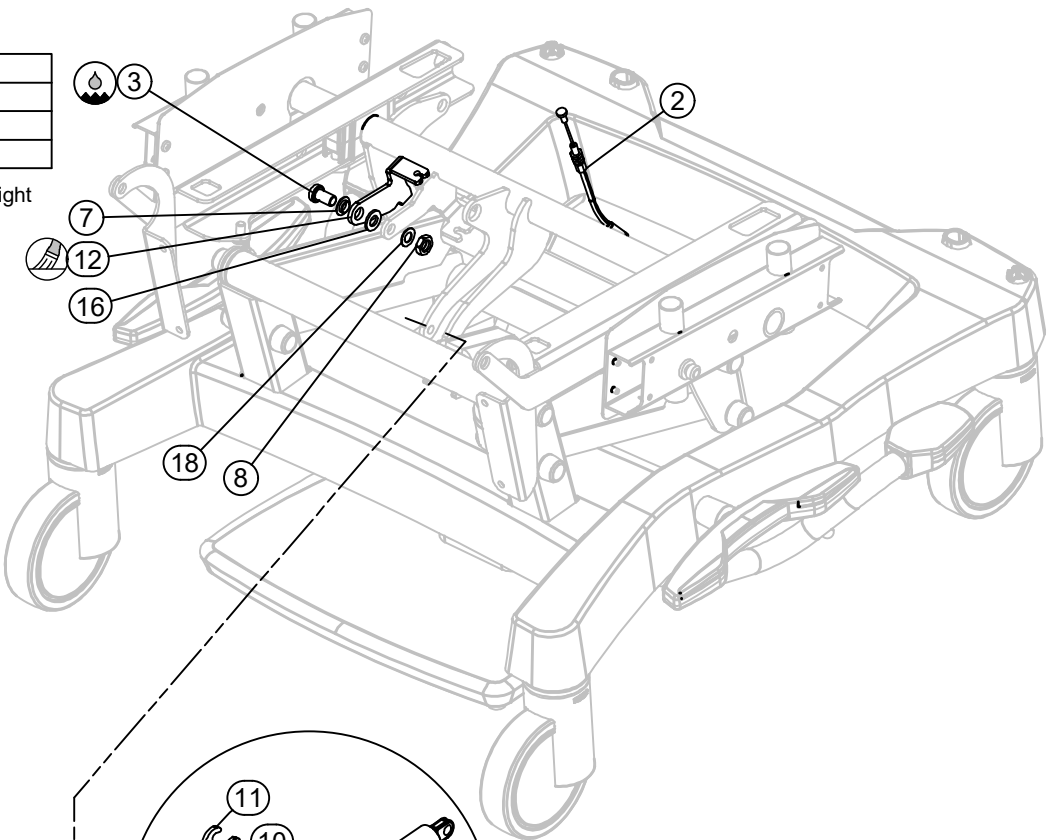

|    | ID       | Description                             | Q     |
|----|----------|-----------------------------------------|-------|
| 1  | A-003302 | gas spring 28BD-SL50-IL187-700N         | 1     |
| 2  | A-022751 | activation cable Avento HL stand-up aid | A+1 0 |
| 3  | B-003317 | bolt-M10x18x12-GV white                 | A+1 2 |
| 4  | DC002587 | axle type 1-10x72                       | 1     |
| 5  | DC006037 | axle type 1-8x64                        | 1     |
| 6  | GL000611 | bearing 0810-04                         | 2     |
| 7  | GL000618 | bearing 1012-04                         | A+1 4 |
| 8  | M-006850 | nut din439-M10-04 GV                    | A+1 2 |
| 9  | P-005430 | pushing hinge leg support               | B-2 2 |
| 10 | P-005840 | control cam Fero Ergo-line              | A-1 1 |
| 11 | P-025055 | control cam Avento HL                   | A+1 0 |
| 12 | P-027578 | cable cam stand-up aid Avento HL        | A+1 0 |
| 13 | ST005073 | release head M10 dia8                   | 1     |
| 14 | ST010662 | axle locking ring 14/7x1.3              | 4     |
| 15 | T-030224 | recliner mechanism forks Fero Ergo-line | 1     |
| 16 | VL000469 | washer Din125A-M10-PA                   | A+1 2 |
| 17 | VL001176 | synthetic washer 20x10x10               | 6     |
| 18 | VL001873 | bearing flange 1018-010                 | A+1 2 |
| 19 | VL004120 | synthetic washer 20x10x5                | 2     |
| 20 | VL014993 | synthetic washer 25x10x2                | 1     |

Manual adjustable legrest

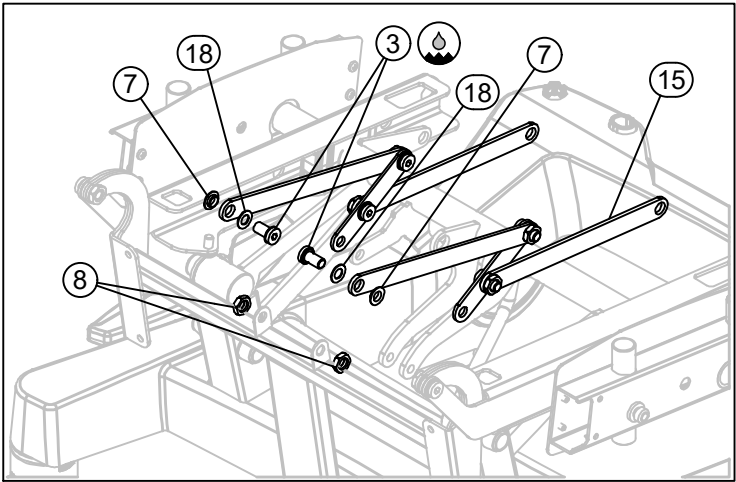

|                                                                                                                                                                                 |  |                                                                                                           |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|-----------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  <div>Haelvoet</div> |                                                                                                                                                                             |
| Description:                                                                                                                                                                    |  | Designer:                                                                                                 |   |
| Avento Care - recliner mechanism 202006                                                                                                                                         |  | Ionut I.                                                                                                  |                                                                                                                                                                             |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 5/06/2020                                                                                           | Scale: 1 :7                                                                                                                                                                 |
|                                                                                                                                                                                 |  | Drawing nr.: T-039482                                                                                     | Revision nr.: b                                                                                                                                                             |


T-039483

a



| Options |                               |
|---------|-------------------------------|
| ID.     | Description                   |
| A       | Manual adjustable legrest (*) |

(\*) Possible only with Avento fixed height

|    | ID       | Description                     | Q     |
|----|----------|---------------------------------|-------|
| 1  | A-003302 | gas spring 28BD-SL50-IL187-700N | 1     |
| 2  | B-003317 | bolt-M10x18x12-GV white         | 2     |
| 3  | DC002587 | axle type 1-10x72               | 1     |
| 4  | DC006037 | axle type 1-8x64                | 1     |
| 5  | GL000611 | bearing 0810-04                 | 2     |
| 6  | GL000618 | bearing 1012-04                 | 4     |
| 7  | M-006850 | nut din439-M10-04 GV            | 2     |
| 8  | P-005430 | pushing hinge leg support       | A-2 2 |
| 9  | P-005840 | control cam Fero Ergo-line      | 1     |
| 10 | ST005073 | release head M10 dia8           | 1     |
| 11 | ST010662 | axle locking ring 14/7x1.3      | 4     |
| 12 | T-034633 | recliner mechanism forks 200 kg | 1     |
| 13 | VL000469 | washer Din125A-M10-PA           | 2     |
| 14 | VL001176 | synthetic washer 20x10x10       | 6     |
| 15 | VL001873 | bearing flange 1018-010         | 2     |
| 16 | VL004120 | synthetic washer 20x10x5        | 2     |
| 17 | VL014993 | synthetic washer 25x10x2        | 1     |

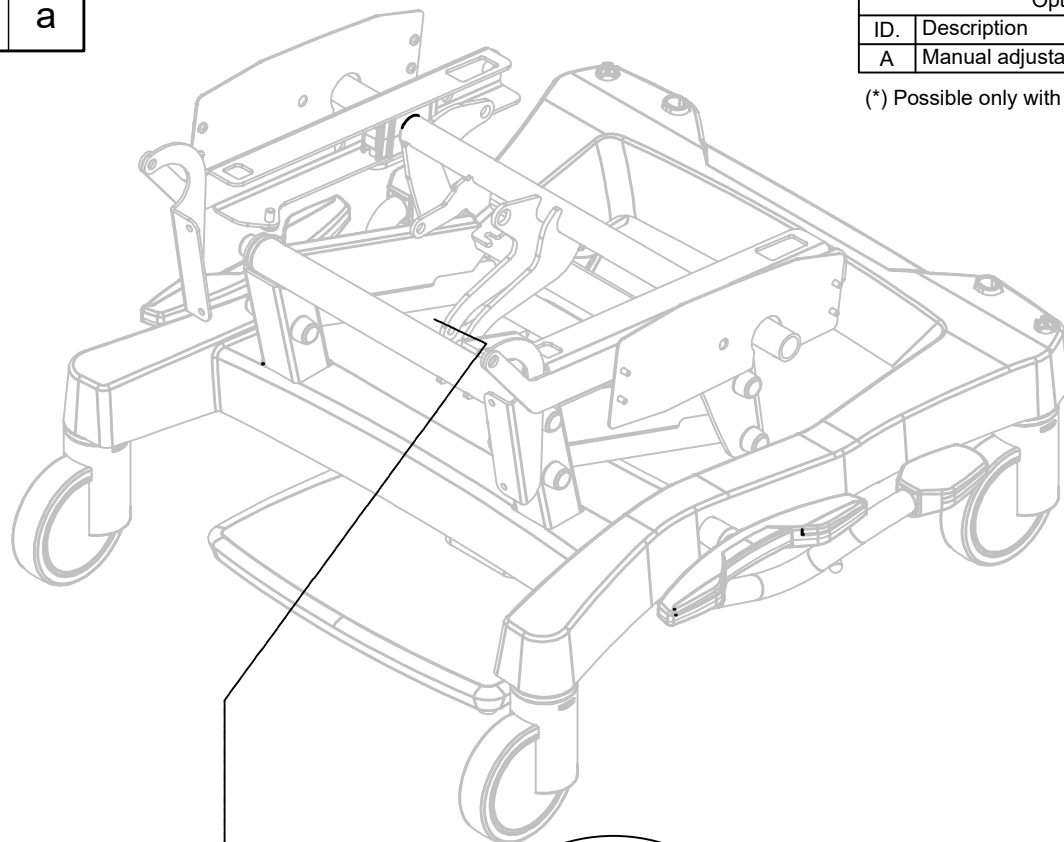

## Manual adjustable leg rest

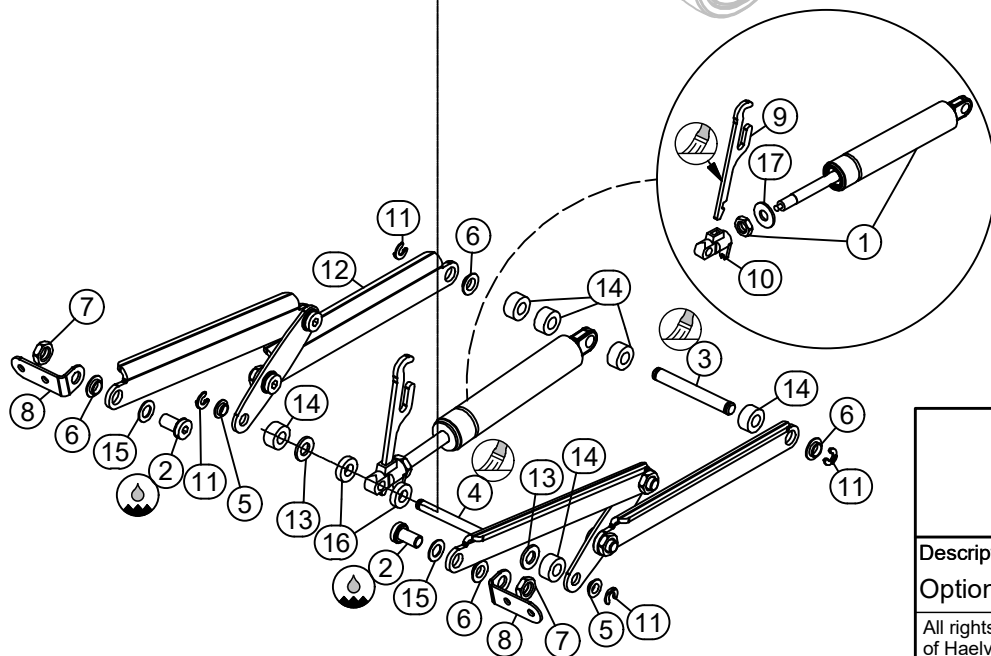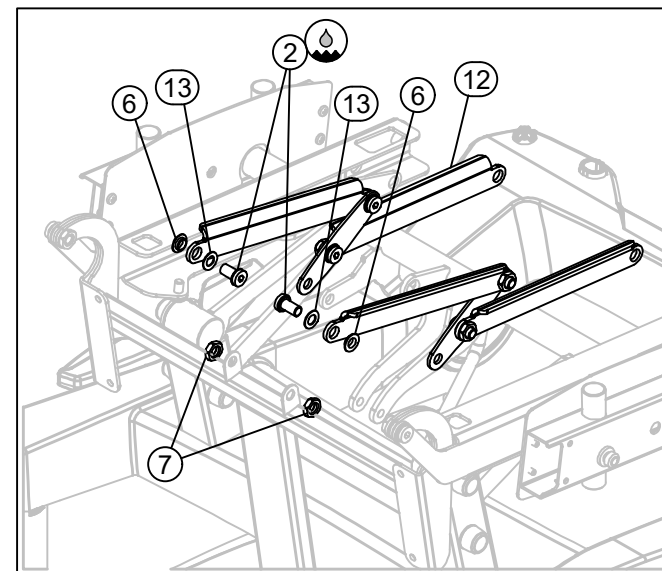

## Description:

Option: Avento Care - recliner mechanism 200 kg 202006


All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95

## Designer:

Ionut I.

Date: 05/06/2020

Drawing nr.: T-039483


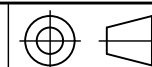


Scale: 1 : 7


Revision nr.: a

| T-039485 |                           | a |
|----------|---------------------------|---|
| Options  |                           |   |
| ID.      | Description               |   |
| A        | Stand-up aid              |   |
| B        | Reclining lock            |   |
| C        | Fixation brace            |   |
| D        | Support dinner tray Elice |   |



|    | ID       | Description                                       | Q         |
|----|----------|---------------------------------------------------|-----------|
| 1  | A-022168 | activation cam stand-up aid                       | A+1 0     |
| 2  | A-022512 | plastic spring button- LA Avento                  | 2         |
| 3  | A-022670 | reclining locking cam                             | A+1;B+1 0 |
| 4  | A-023204 | camshaft                                          | 1         |
| 5  | A-027405 | spring button lock mechanism red RAL3026          | B+1 0     |
| 6  | A-027406 | spring button lock mechanism blue RAL5017         | A+1 0     |
| 7  | B-003317 | bolt-M10x18x12-GV white                           | 2         |
| 8  | B-005873 | bolt Iso7380-M6x16x16-10.9 GV white-LC            | C+2 0     |
| 9  | B-016275 | bolt Iso7380-M8x25x25-10.9 GV white               | 2         |
| 10 | DC005842 | handle                                            | 2         |
| 11 | DC028996 | armrest guiding Avento HL Care                    | 4         |
| 12 | DC029334 | fixation bush locking spring button               | A+1;B+1 0 |
| 13 | DC029455 | axis backrest                                     | 2         |
| 14 | GL000618 | bearing 1012-04                                   | 2         |
| 15 | GL005737 | bearing 1416-06                                   | 1         |
| 16 | GL029787 | bearing 141625-04                                 | 2         |
| 17 | P-016223 | fixation brace Fero                               | C+1 0     |
| 18 | P-022121 | push bar Fero                                     | 1         |
| 19 | P-025072 | control handle Fero with stand-up aid             | 1         |
| 20 | P-027408 | cap support dinner tray Elice                     | D+1 0     |
| 21 | P-027409 | support dinner tray Elice                         | D+1 0     |
| 22 | P-027608 | control handle Fero                               | 1         |
| 23 | P-028290 | wooden side cover L Avento HL                     | 1         |
| 24 | P-028291 | wooden side cover R Avento HL                     | 1         |
| 25 | S-004232 | screw Din7982C-4.2x25-GV                          | 4         |
| 26 | S-015930 | screw Din7500(TT12)-M4x16-GV                      | A+3;B+1 3 |
| 27 | S-028666 | screw with sunken head and drilled head-4,5x50-GV | D+2 0     |
| 28 | ST029828 | cylindrical pin Din6325 3m5x6 St                  | A+2;B+2 0 |
| 29 | T-039043 | backrest Fero Ergo-line/Emove                     | 1         |
| 30 | VL000337 | washer Din125A-M8-GV                              | 2         |
| 31 | VL001873 | bearing flange 1018-010                           | 2         |
| 32 | VL029858 | washer Din125A-M14.5-PA                           | 1         |

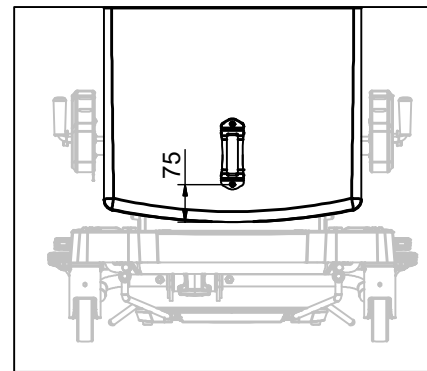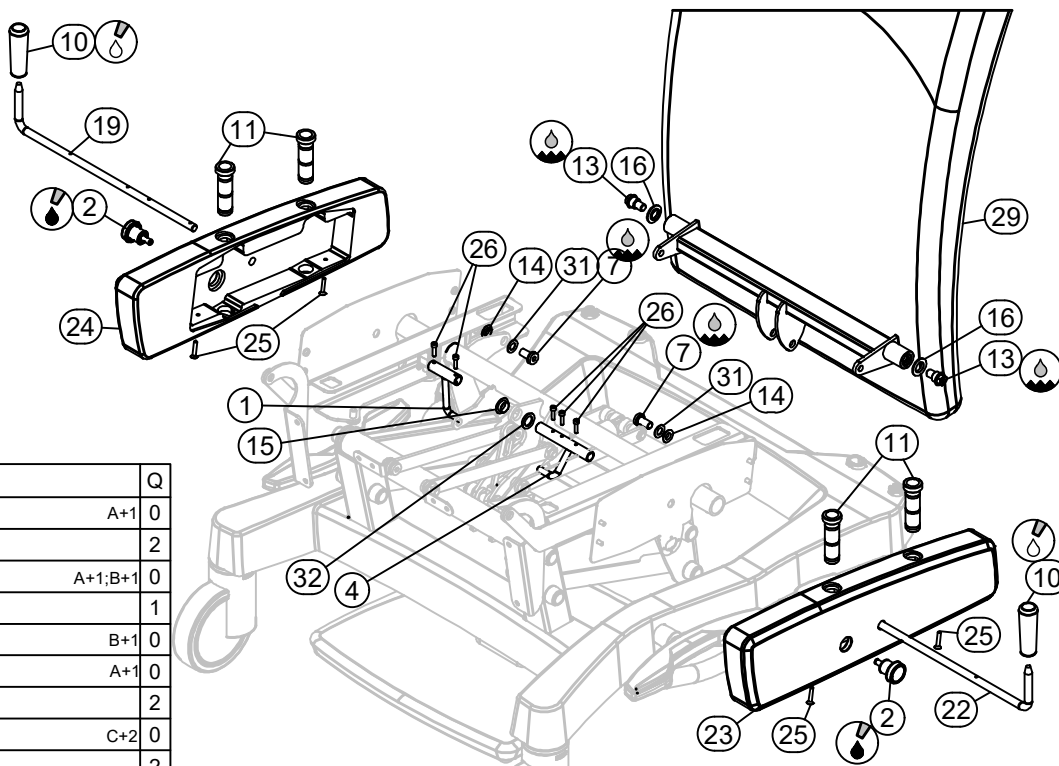

Reclining lock on right side


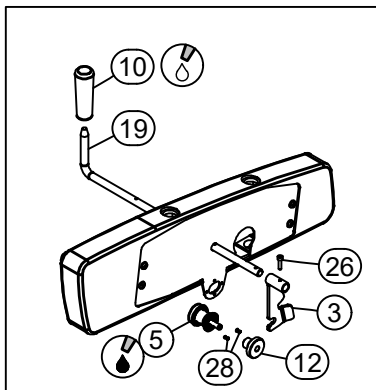


Stand-up aid lock on left side

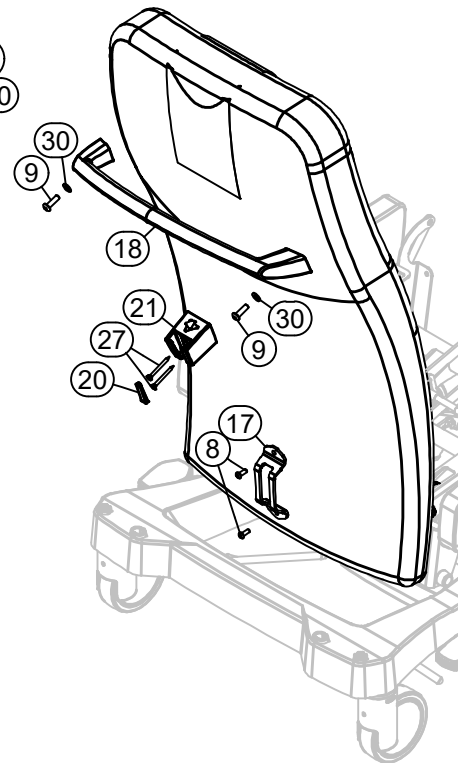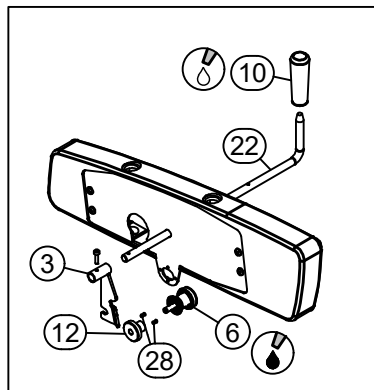

|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| <b>Description:</b><br>Avento Care back and sides 202006                                                                                                                        |  | <b>Designer:</b><br>Ionut I.                                                                   |   |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | <b>Date:</b> 05/06/2020<br><b>Drawing nr.:</b> T-039485                                        | <b>Scale:</b> 1 : 10<br><b>Revision nr.:</b> a                                                                                                                              |

|          |               |   |
|----------|---------------|---|
| T-016924 |               | d |
| Options  |               |   |
| ID.      | Description   |   |
| A        | Epoxy support |   |

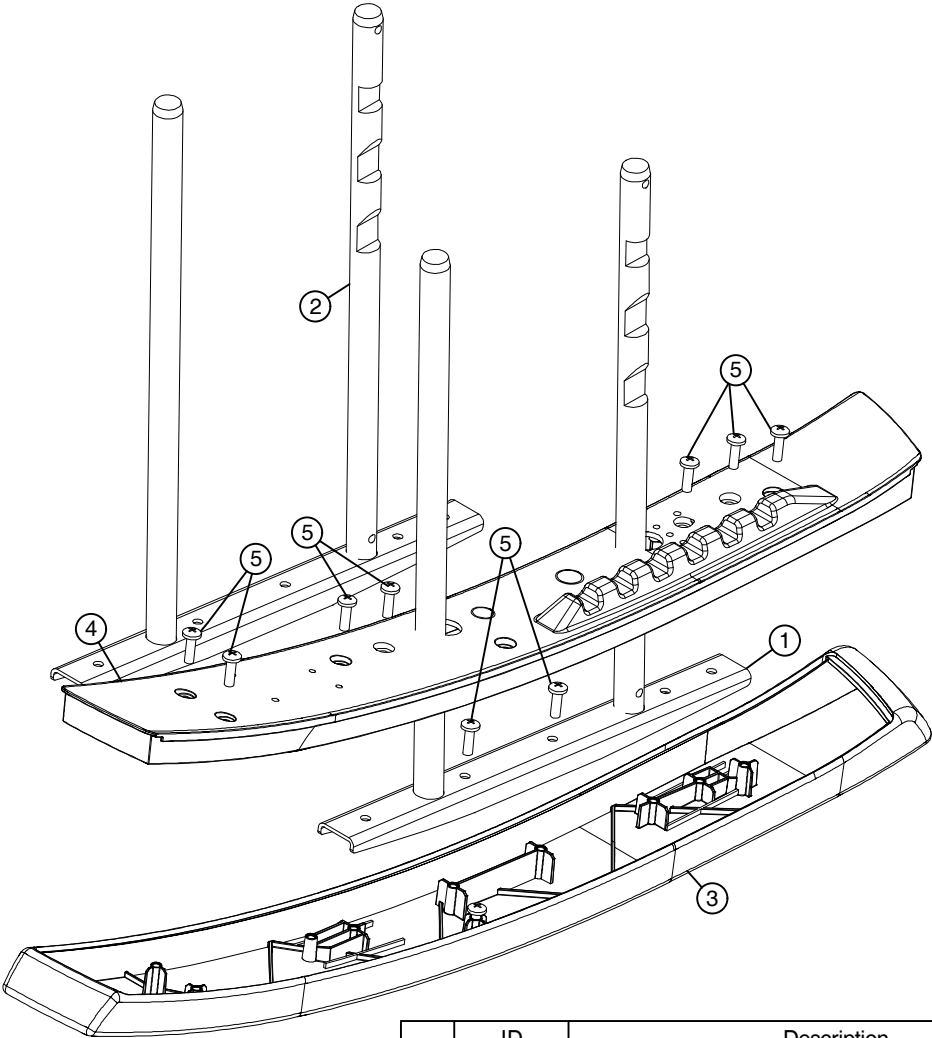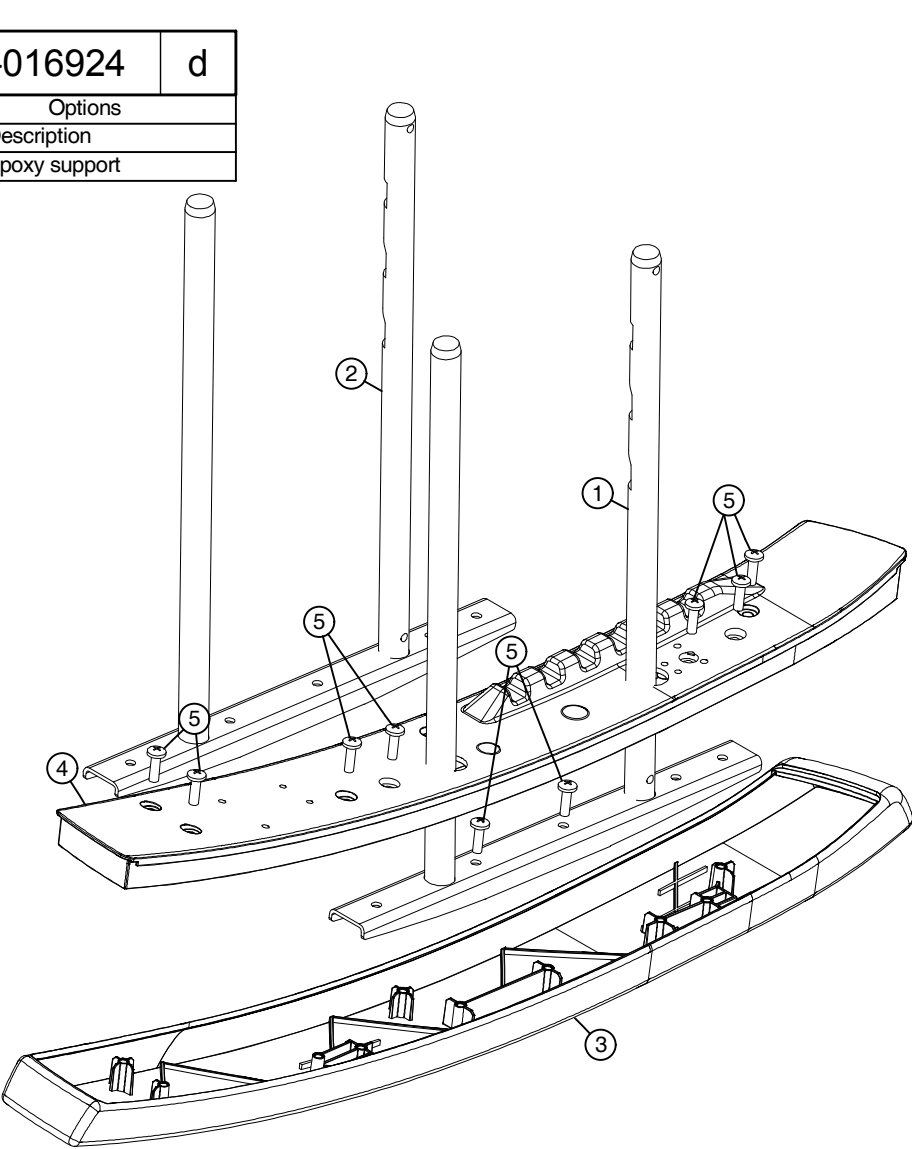


|   | ID       | Description                                   | Q     |
|---|----------|-----------------------------------------------|-------|
| 1 | A-014801 | support adjustable synthetic armrest chrome L | A-1 1 |
| 2 | A-025395 | support adjustable synthetic armrest epoxy L  | A+1 0 |
| 3 | P-018845 | cover synthetic armrest Fero L                | 1     |
| 4 | P-018846 | box adjustable synthetic armrest Fero L       | 1     |
| 5 | S-019308 | screw PLASTITE 7985-5x16-GV                   | 10    |

|   | ID       | Description                                   | Q     |
|---|----------|-----------------------------------------------|-------|
| 1 | A-014297 | support adjustable synthetic armrest chrome R | A-1 1 |
| 2 | A-025393 | support adjustable synthetic armrest epoxy R  | A+1 0 |
| 3 | P-018287 | cover synthetic armrest Fero R                | 1     |
| 4 | P-018288 | box adjustable synthetic armrest Fero R       | 1     |
| 5 | S-019308 | screw PLASTITE 7985-5x16-GV                   | 10    |



|                                                                                                                                                                                 |  |                                                                                                       |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|-------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  <b>Haelvoet</b> |                                                                                                                                                                             |
| <b>Description:</b><br><b>assembly synthetic adjustable armrests Fero</b>                                                                                                       |  | <b>Designer:</b><br>David V.                                                                          |   |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | <b>Date:</b> 23/04/2012<br><b>Drawing nr.:</b> T-016924                                               | <b>Scale:</b> 1 : 4<br><b>Revision nr.:</b> d                                                                                                                               |

R

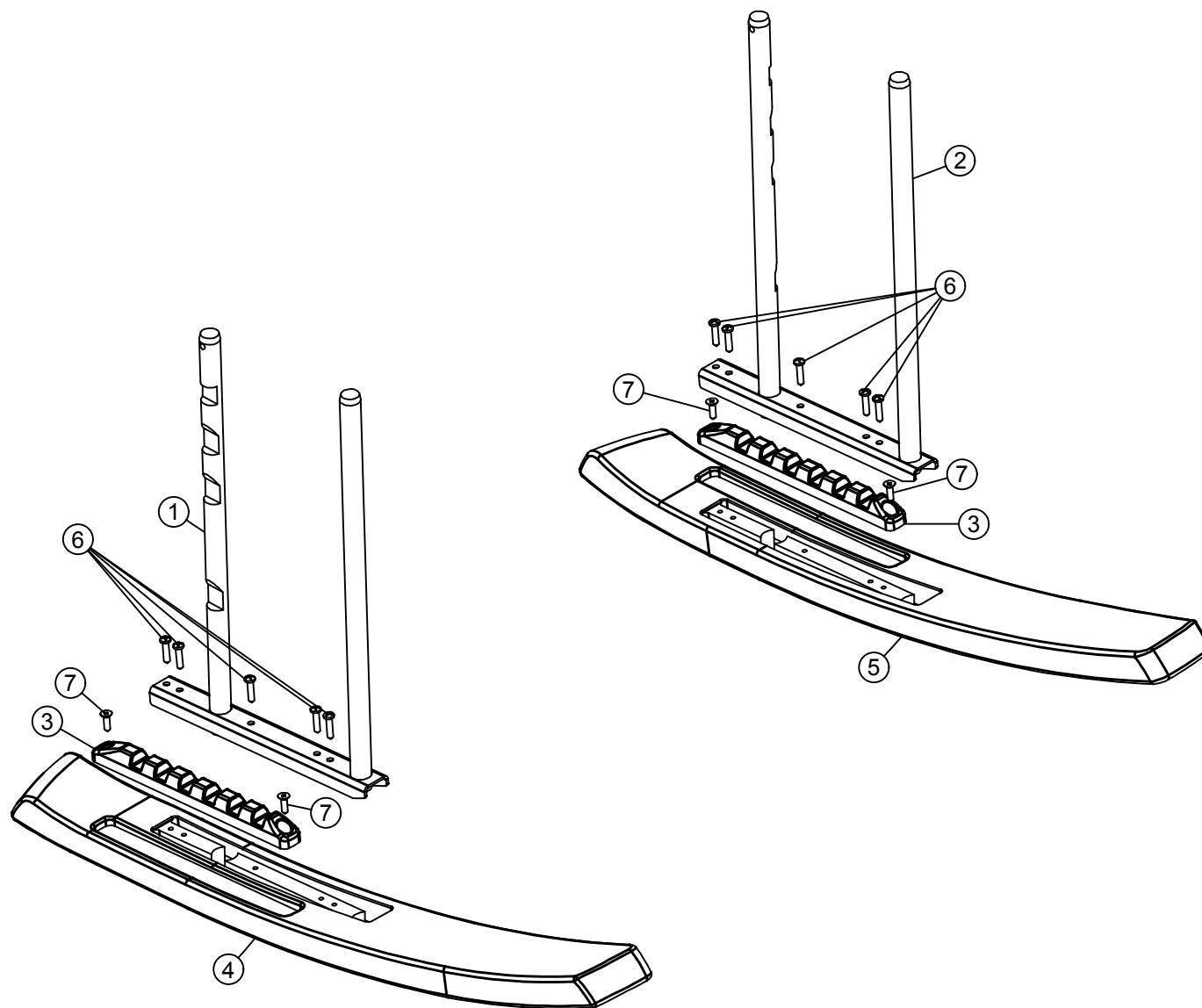

|   | ID       | Description                              | Q  |
|---|----------|------------------------------------------|----|
| 1 | A-023491 | support adjustable armrest Fero/Avento R | 1  |
| 2 | A-023495 | support adjustable armrest Fero/Avento L | 1  |
| 3 | P-005705 | dinner tray fixation                     | 2  |
| 4 | P-028295 | adjustable armrest Fero/Avento R         | 1  |
| 5 | P-028305 | adjustable armrest Fero/Avento L         | 1  |
| 6 | S-000071 | screw Din7505B-4.5x20-GV                 | 10 |
| 7 | S-003179 | screw Din7505A-4.5x16-GV                 | 4  |





|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| <b>Description:</b><br>adjustable armrests Fero/Avento                                                                                                                          |  | Designer:<br>Ionut I.                                                                          |   |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 20/11/2015<br>Drawing nr.: T-028442                                                      | Scale: 1 : 5<br>Revision nr.: b                                                                                                                                             |

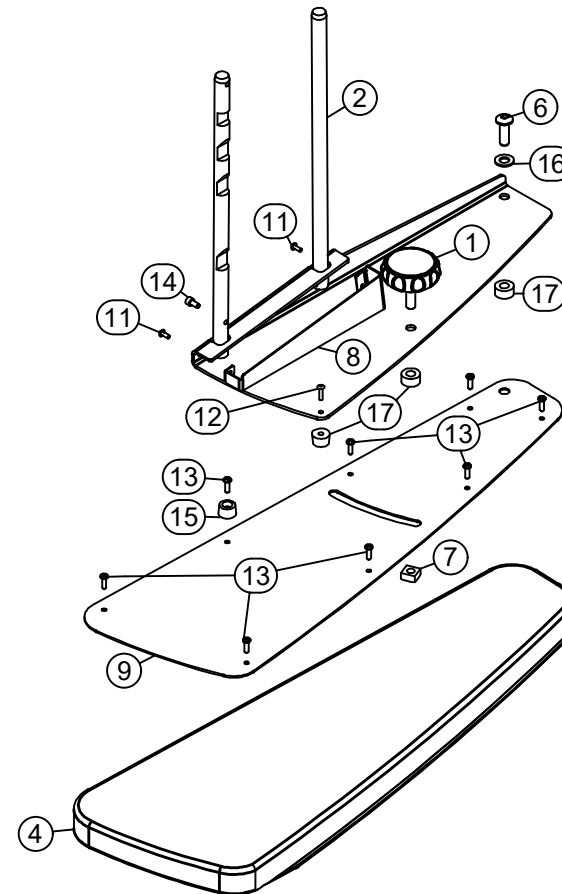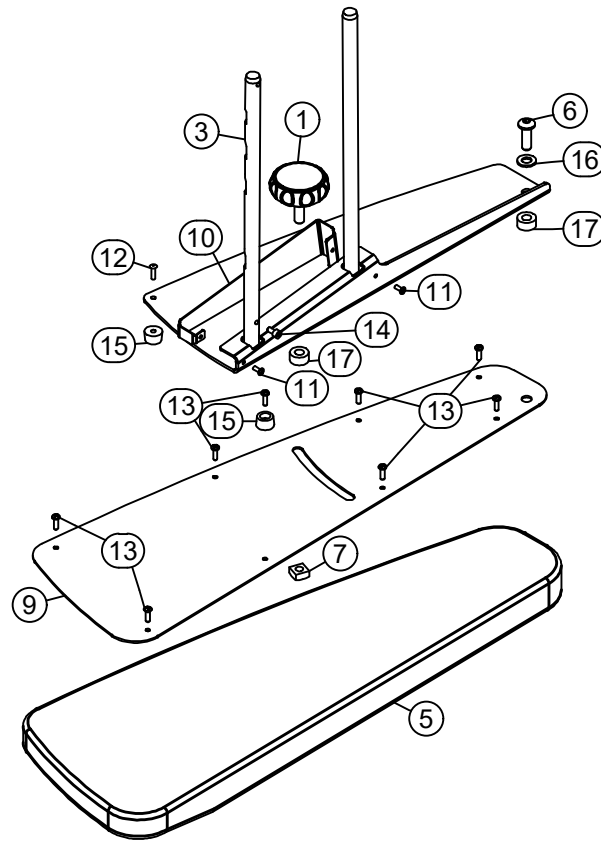


|    | ID       | Description                           | Q  |
|----|----------|---------------------------------------|----|
| 1  | A-000505 | clamping handle                       | 2  |
| 2  | A-012155 | support large swivel armrest R        | 1  |
| 3  | A-012165 | support large swivel armrest L        | 1  |
| 4  | A-021675 | swivelling broadened armrest R        | 1  |
| 5  | A-021680 | swivelling broadened armrest L        | 1  |
| 6  | B-000818 | bolt Iso7380-M10x30x30-10.9 GV white  | 2  |
| 7  | M-013706 | square nut Din557-M10-GV white        | 2  |
| 8  | P-013676 | cover large swivel armrest R          | 1  |
| 9  | P-013677 | protection plate large swivel armrest | 2  |
| 10 | P-013709 | cover large swivel armrest L          | 1  |
| 11 | R-000459 | rivet Alu/St 3.9x10                   | 4  |
| 12 | R-000460 | rivet Alu/St 3.9x16                   | 2  |
| 13 | S-000376 | screw Din7505B-4.5x16-GV              | 16 |
| 14 | S-014995 | screw type TT12-M5x10-GV white        | 2  |
| 15 | ST005281 | buffer 19/11                          | 4  |
| 16 | VL000469 | washer Din125A-M10-PA                 | 2  |
| 17 | VL001176 | synthetic washer 20x10x10             | 4  |

|                                                                                                                                                                                 |  |                                                                                                |                                                                                                                                                                             |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                                                                                                             |
| <b>Description:</b><br>swivelling broadened armrests                                                                                                                            |  | Designer:<br>Ionut I.                                                                          |   |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 25/03/2020<br>Drawing nr.: T-038876                                                      | Scale: 1 : 8<br>Revision nr.: a                                                                                                                                             |




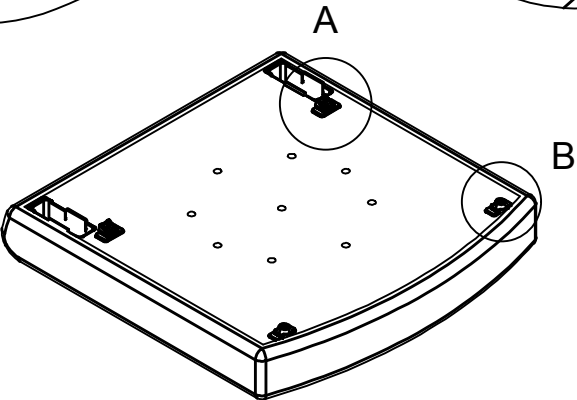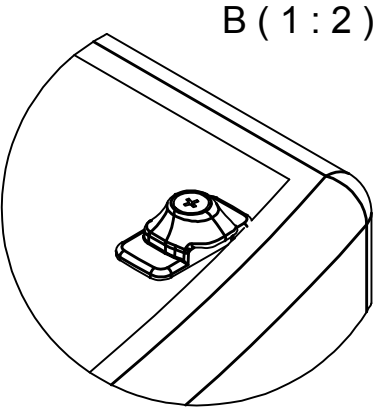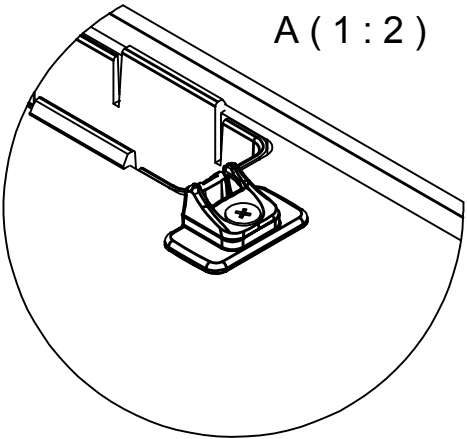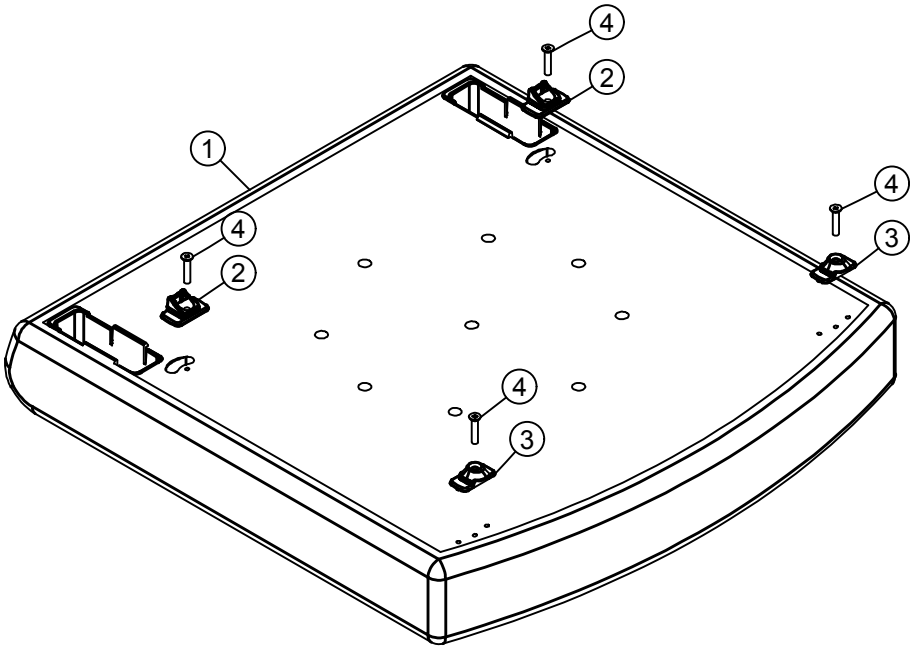


|   | ID       | Description                         | Q |
|---|----------|-------------------------------------|---|
| 1 | A-025991 | seat Fero Ergo-line upholstery 2019 | 1 |
| 2 | P-029694 | seat fixation front                 | 2 |
| 3 | P-029700 | seat fixation back                  | 2 |
| 4 | S-030477 | screw PLASTITE 965-5x25-GV          | 4 |
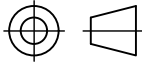
|                                                                                                                                                                                 |  |                                                                                                |                                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  Haelvoet |                                                                                       |
| <b>Description:</b><br>seat Fero Ergo-line snap-on 2019                                                                                                                         |  | Designer:<br>Lucas V.                                                                          |  |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | Date: 20/12/2017<br>Drawing nr.: T-036674                                                      | Scale: 1 : 6<br>Revision nr.: a                                                       |

|          |                               |   |
|----------|-------------------------------|---|
| T-039467 |                               | a |
| Options  |                               |   |
| ID.      | Description                   |   |
| A        | Swivelling broadened armrests |   |
| B        | Wooden lowerable armrests     |   |

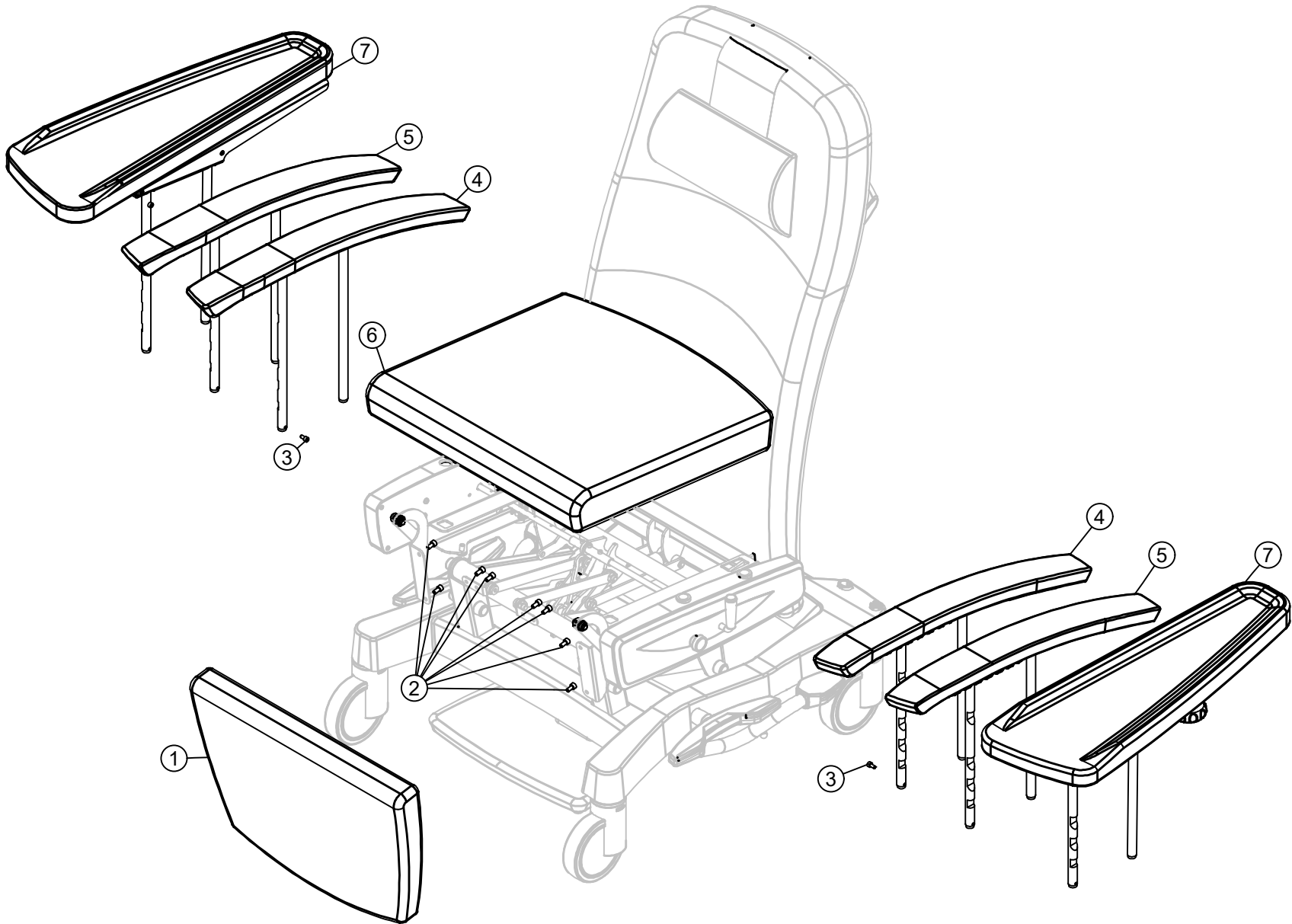

|   | ID       | Description                              | Q         |
|---|----------|------------------------------------------|-----------|
| 1 | A-026611 | leg rest Fero Ergo-line 201912           | 1         |
| 2 | B-000650 | bolt Din912-M6x12x12-8.8 GV white        | 8         |
| 3 | S-014995 | screw type TT12-M5x10-GV white           | 2         |
| 4 | T-016924 | synthetic lowerable armrests Fero/Avento | A-1;B-1 1 |
| 5 | T-028442 | wooden lowerable armrests Fero/Avento    | B+1 0     |
| 6 | T-036674 | seat Fero Ergo-line snap-on 2019         | 1         |
| 7 | T-038876 | swivelling broadened armrests            | A+1 0     |
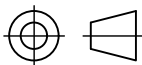
|                                                                                                                                                                                 |  |                                                                                                       |                                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|-------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|                                                                                                                                                                                 |  |  <b>Haelvoet</b> |                                                                                       |
| <b>Description:</b><br><b>leg &amp; seat &amp; arm Avento HL</b>                                                                                                                |  | <b>Designer:</b><br>Ionut I.                                                                          |  |
| All rights reserved. Reproduction in whole or in part is prohibited without the written consent of Haelvoet NV - L. Bekaertstraat 8 8870 Ingelmunster - Tel: +32 (0)51 48 66 95 |  | <b>Date:</b> 03/06/2020<br><b>Drawing nr.:</b> T-039467                                               | <b>Scale:</b> 1 : 10<br><b>Revision nr.:</b> a                                        |