

Imitation leather

Cleaning instructions

Cleaning and disinfection of a wooden Fero seat

Cleaning: Use a soft cloth - that is moistened with cold or tepid water - and a mild cleaning product or an all-purpose cleaner to clean the seat. Clean the seat afterwards with a moist cloth (without cleaning products) and wipe the seat carefully after the entire cleaning process.

Avoid:

- Alcoholic cleaning products
- Corrosion-stimulating or aggressive cleaning products
- Cleaning products containing harmful substances
- Scourers
- The composition of the products may not affect the structure or surface of the wooden and synthetic parts. Moreover, the polyester epoxy coating may not be affected as well.

Always read the product information and follow the directions of the cleaning products and disinfectants.

Disinfection: A good chemical disinfection of the seat can only be obtained by cleaning the seat thoroughly. Take the clinical picture of the patient and the potential presence of infected seat parts into account, and adjust the number of disinfections accordingly. Only qualified and trained personnel, familiar with the effects and use of disinfectants, are allowed to disinfect the seat. Always wear appropriate working clothes, since disinfectants may cause irritations. Always follow the directions of the used product.

- Always use cold or tepid water to dilute the product. Do not use warm water, as this causes vapour. Always seal the dilution.
- Do not use alcoholic products to disinfect large surfaces.
- Avoid skin contact and always wear gloves.
- Always check if the correct quantity has been used.
- Always make sure that there is enough ventilation during and/or after the disinfection.
- Always use a cloth or a rag to disinfect. Do not use spray products (risk of inhalation!).

We would like to refer to the website of the Robert-Koch-Institut (www.rki.de) to choose the correct disinfectant. Keep a disinfection journal for each seat and note down when and why the disinfection has taken place. Also write down which disinfectant and which quantity has been used, and do not forget to mention the name and signature of the cleaning person.



General information

Check the following items before cleaning or disinfecting the seat:

- Activate the seat brakes.
- The gas spring should be fully retracted, to prevent that water arrives in the gas spring.
- Do not abundantly sprinkle the seat with water. It is absolutely forbidden to use a high-pressure cleaner.
- Make sure the personnel has appropriate outfits and material (water-proof aprons and gloves, correct cleaning products, ...)
- All metal protective layers have a high abrasive resistance. Nonetheless, a metal component can have scratches that expose the underlying layer. Always repair this damage to avoid corrosion.

After having cleaned and disinfected the seat, it is important to disinfect your hands before going to another area or room.

Haelvoet NV cannot be held liable for any damage, injuries and impairments resulting from improper use of cleaning products or disinfectants.

